NOW ONE MASK

Delivers 24% to 90% Oxygen!



ONE MASK DELIVERS 24% TO 90% OXYGEN

24* Nasal Cannula

40* (up to 6 L/min)*

*AARC Clinical Practice Guideline Reprinted from RESPIRATORY CARE (Respir Care 2002; 47(6):717-720)

35* Simple Mask 50* (5-10 L/min)*

40* Partial Rebreathing Mask 70* (6-10 L/min)*

60* Non-rebreathing Mask 80* (min10 L/min)*

24*

24*

"From Admission to Discharge"
IDEAL FOR EMS, HOSPITAL AND HOMECARE

OxyMask's logical advantages:


- Phthalate-Free
- The only O2 therapy device to deliver FiO2 of 24% 90%
- Can be utilized to deliver High Flow O2 therapy to flush
- Zero probability of creating intrinsic PEEP on flow rates > 15 LPM
- · Zero probability of CO2 rebreathing on low flow rates
- · Allows open communication with your patient
- · Designed for nose and mouth breathers
- · Convenient swivel elbow to reposition tubing
- Nasal Gastric tube can be threaded through OxyMask
- · Allows for suctioning & oral care through mask
- · Open mask allows for delivery of other respiratory therapies
- · Reduces probability of aspiration

Cost saving advantages:

- Reduces number of O2 devices per patient
- · Simplifies and reduces inventory
- · Reduces PVC waste
- Reduces manpower due to simple FiO2 / flow changes
- · Reduces medical gas consumption
- Achieve higher FiO₂ on lower flow rates

Proven to Deliver Effective Heliox Therapy

OxyMask O₂ % Guidelines

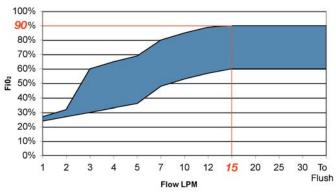





OxyMASK Product # OM-1125-8

OxyPLUS Product # OP-1125-8 25% larger than OxyMask

OxyMASK Product # OM-1425-8 with Dual Far Flastics

OxyPLUS Product # OP-1425-8 25% larger than OxyMask with Dual Ear Elastics

Flowrate	Fi02	
1 LPM 2 LPM 3 LPM 4 LPM 5 LPM 7 LPM 10 LPM 12 LPM ≥15 LPM	24% - 27% 27% - 32% 30% - 60% 33% - 65% 36% - 69% 48% - 80% 53% - 85% 57% - 89% 60% - 90%	Actual FiO ₂ will vary. Breath rate, depth of breathing and distance of the diffuser from the face will affect performance. Actual flow rate required should be prescribed in conjunction with the use of a pulse oximeter.



705-726-9383

email: smedic@southmedic.com

www.southmedic.com www.oxyarm.com
50 Alliance Blvd., Barrie, ON., Canada L4M 5K3
ISO 13485 EDI 406000117

1st Open Oxygen System™ We are the experts! Patent Pending

OXYMASK IS A REGISTERED TRADEMARK