



---

# 13 POWER LIFT

---

User's Manual




---

ILIVINGUSA

## BRIEF INSTRUCTION FOR I3 POWER LIFT

What you get from the main carton:

- ◇ The lift
- ◇ Control cable
- ◇ Elastic band x2strips

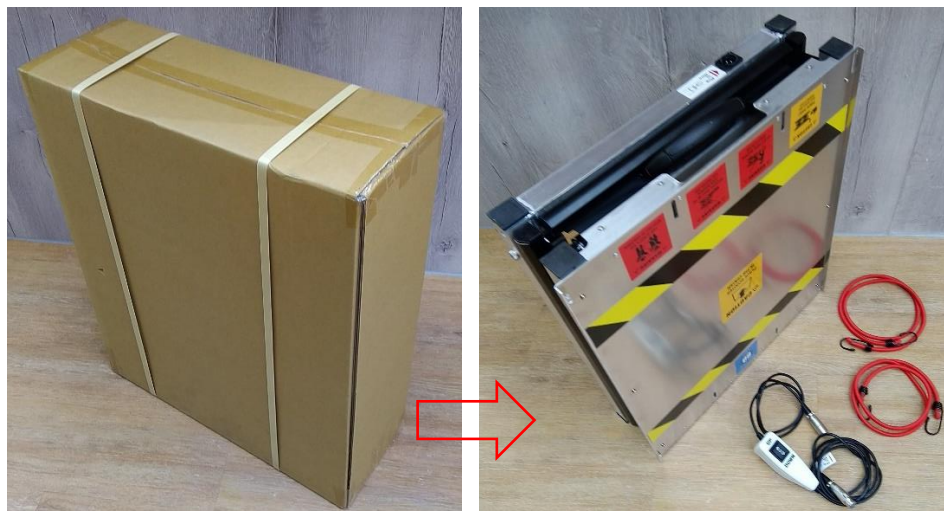

How to use the power lift to lift your i3 scooter:

**1. Place the lift on a flat and solid ground.**

Note: the handle may drop outward and protrude outside, this is a correct position for the handle. At this position, the handle will not obstruct the up and down motion of the lift. Moreover, this position will be easy for you to grip the handle.



**2. Put your i3 onto the lift platform.**


Note: make sure that the lift is at its lowest position. The lowest position of the lift platform has a height of about 14cm (5.5") from ground.

Note: the scooter should place at the center position of the platform, i.e. within the yellow/black strips.



3. Plug the control cable into the lift receptor socket.

Note: there are 2 plugs on the control cable, you can never mix up the two or wrongly plugged. The one with a longer cable is for the lift.

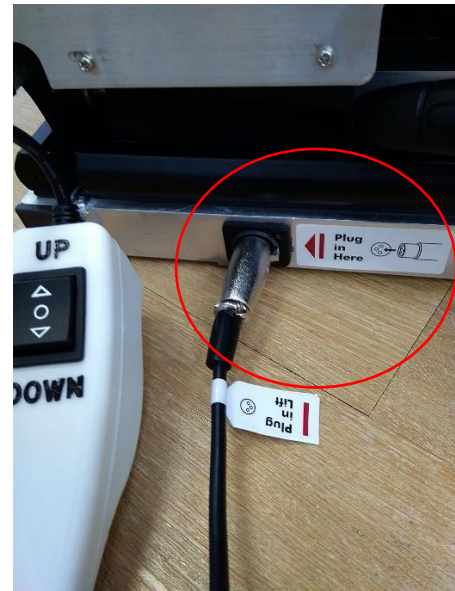

4. Plug the control cable into the charge port of the scooter.

Note: the plug with a shorter cable is for the scooter.

Note: the lift gets its power from the scooter battery.

Note: the scooter can supply power to the lift via the charge port. Make sure the scooter battery is fully charged for the lift operation.




5. Press the up arrow to raise the lift.

Note: you can stop the lift at any up raising position, just stop pressing the up arrow.

Note: the lift will auto stop at its highest position. You control switch will then stop and no more operation.

**WARNING:** there are curtain around the 4 side of the lift to prevent unawareness of being pinched and hurt. The curtain should not be removed and replace new one if damaged.


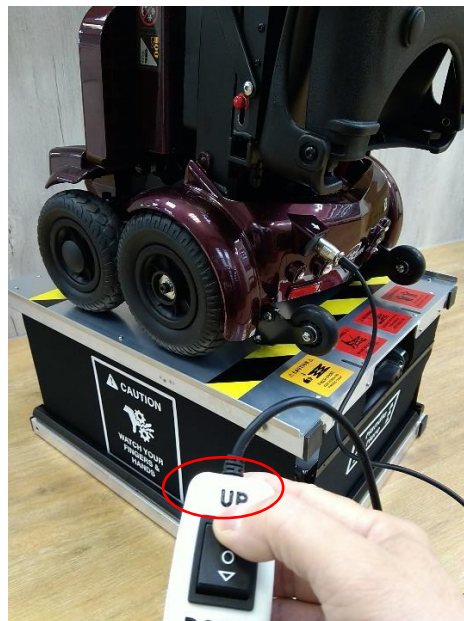


6. Press the down arrow to lower the lift.

Note: you can stop the lift at any position, just release the switch.

Note: it is a good practice that every time you finish using the lift, always low the lift to its lowest position for easy carry and storage.

CAUTION: somehow, due to removal of scooter from platform, the lift will rebound upward due to weight removal. You can still lower the lift to its lowest position by replug the control cable to the scooter charge port and drive the lift to its lowest position.

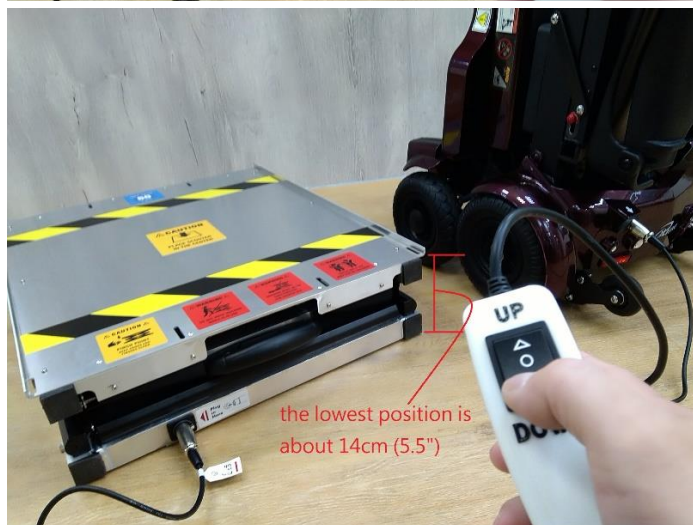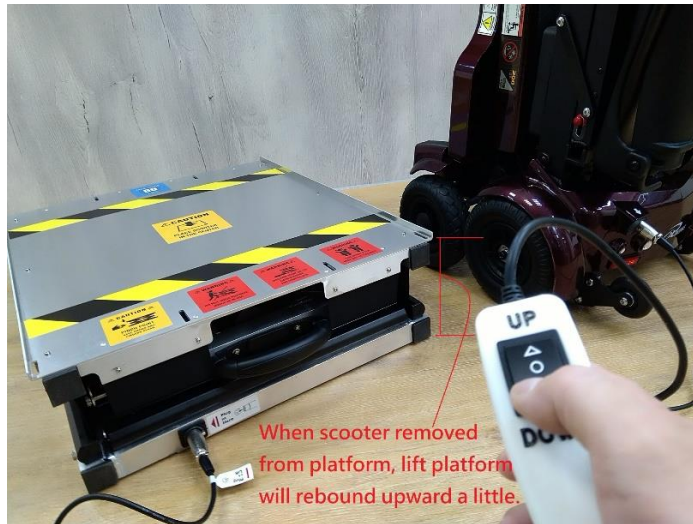

7. Note: we have included 2 elastic band for you to secure the scooter when it is placed on the platform during upraising and down ward operation of the lift.

It is a good practice to always secure the scooter when it is on the platform.



8. Optional items: you can have an extra battery and a portable charge port unit. The lift can drive by the optional kit



## How to set up the ramp

1. What you get from the carton: 2 foldable ramp



2.

- ❖ There are slot holes located at the side of the platform. These slots are for the ramp hooks to attach.
- ❖ Extend the ramp, note the "L" shape hooks at one end of the ramp.
- ❖ Just place the hook into the slot
- ❖ Attach both ramp and ready for use.
- ❖ **WARNING:**
- ❖ Ramp can be use ONLY when the lift is at its lowest position. Do Not follow this warning may cause serious damage to the lift and user being hurt.



Use the ramp to push the scooter onto the platform and into a SUV

1. To drag, to draw, to push your scooter, you have to put the rear axel into free wheel mode. Push the free wheel handle into "N" neutral position. Now you can drag or push the scooter easy and light.




2.  
❖ Put the lift near by the back trunk of your SUV.  
❖ Hook up the ramp.  
❖ Locate the scooter as shown



3.  
Push the scooter along the ramp and up to the platform of the lift



4.  
❖ Located the scooter at the center of the platform.  
❖ Tie down the scooter by elastic band.  
❖ Remove the ramps

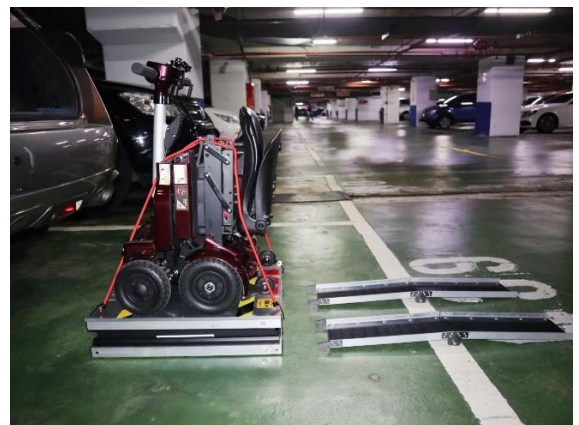

5. Connect the control cable



6. Raise the lift to a height level (approx.) to the back trunk of the SUV.



7. Attach the ramp at the other side of the Lift and bridge the lift to the trunk



8. Push the scooter into the trunk.

Caution: Disconnect the control cable before you push the scooter.



9.

- ✧ Now the scooter is in the trunk.
- ✧ Remove the ramp and reconnect the control cable.




10. Lower the lift to its fold up lowest position.



11.

Pack all items into the trunk and ready to go.

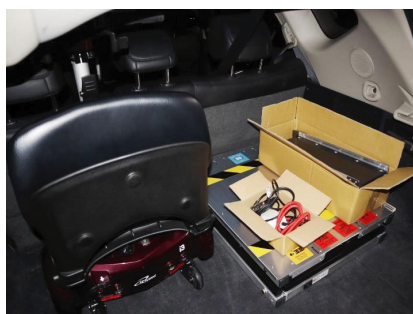

## Warranty Information

This product comes with a 1-YEAR LIMITED COMPONENT WARRANTY. If your unit does not appear to be working properly, please contact our service center by calling 1-800-317-1688. Prior to your call, we encourage you to visit our service related website [www.ilivingusa.com](http://www.ilivingusa.com) for troubleshooting tips and service instructions if needed.

### Warranty Information

Register your product at our website:



Or visit [iLivingUSA.com/register-product](http://iLivingUSA.com/register-product)

### Feedback

Love it? Help us make the product more for you.



Let us know with a customer review.

Please visit: <https://www.amazon.com/review/review-your-purchases#>

At iLiving USA, we are committed to bringing top quality products to our customers.

iLIVING USA

9293 Beatty Dr · Sacramento

CA 95826

Tel: 1-800-317-1688

Email: [service@ilivingusa.com](mailto:service@ilivingusa.com)



Like us on Facebook: <https://www.facebook.com/ilivingusa/>



Follow us on Twitter: @iLIVINGUSA

Check out our website: [www.ilivingusa.com](http://www.ilivingusa.com)