



**ILG-668 Sliding and Swivel Aluminum
Transfer Bench**

OWNER'S MANUAL

PLEASE READ AND

SAVE THESE INSTRUCTIONS



IMPORTANT SAFETY INSTRUCTIONS

Read instructions carefully before assembling your Shower Seat. It is important that you observe all safety instructions to help prevent personal injury.

DO NOT use this equipment without first reading and understanding the instructions contained in this user manual. If you are unable to understand the instructions or need help with assembly, please contact iLiving USA Support Team at **800.317.1688**.

BEFORE INSTALLATION



- 1. Flip Up Foam Armrest
- 2. PU Seat
- 3. Height adjustable foot tube
- 4. Tip
- 5. PU backrest
- 6. Rotating System

INSTALLATION INSTRUCTIONS

Step 1

Attach backrest to the frame, by depressing push buttons and sliding the backrest pipes into the tubes located on each side and on the back of the seat.



Step 2

Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs. Make sure that push button “snap” correctly through holes and are stable and adjusted the same

Step 3

Using the armrests, press on the button and lift armrests backward.



Step 4

- (1) Pulling the lever and turn the seat to face the user, then lock the seat into position by lowering the lever and keep swiveling the seat until it locks in position.
- (2) Unlock the seat by pulling the lever.
- (3) When users are seated in the chair, lock the seat and ensure that it is locked into position and do not rotate freely.



Step 5

By pulling the red lever to have the sliding function.



Warranty Information

This product comes with a 1-YEAR LIMITED COMPONENT WARRANTY. If your unit does not appear to be working properly, please contact our service center by calling 1-800-317-1688. Prior to your call, we encourage you to visit our service-related website www.ilivingusa.com for troubleshooting tips and service instructions if needed.

Register your product at our website:



Or visit [iLivingUSA.com/register-product](https://www.ilivingusa.com/register-product)

Feedback

Love it? Help us make the product more for you.

Let us know with a customer review



Please visit: <https://www.amazon.com/review/review-your-purchases#>

At iLivingUSA, we are committed to bringing top quality products to our customers.

iLIVING USA

860 Mahler Rd

Burlingame, CA 94010

Tel: 1-800-317-1688

Email: service@ilivingusa.com



Like us on Facebook: <https://www.facebook.com/ilivingusa/>



Follow us on Twitter: @iLIVINGUSA

Check out our website: www.ilivingusa.com