

9889 Garrymore Ln Missoula, MT 59808 888-687-3552 | +1-406-549-0769 www.aquacreek.com



EZ Series Hard Seat Option

PART #: F-0320HSO-4C 400 LB. [181 kg] MAXIMUM WEIGHT CAPACITY

FOR USE WITH THE EZ LIFT,
POWER EZ LIFT & SUPER POWER EZ LIFT

MANDATORY

LEAVE THIS MANUAL WITH LIFT OWNER

Read and follow all instructions.

Lift safety can only be ensured if the lift is installed and operated according to these instructions.

- NEVER permit children to play on or around the lift
- Do not allow children to use the lift without adult supervision
- NEVER use the lift with a dry pool

READ CAREFULLY

Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.


It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.

Hard Seat Installation

To install the hard seat option onto an EZ Series lift the sling seat must first be removed. To do this, simply unhook the hanger bar from the lift and store the hanger bar, chains and sling seat together.

Note: Actual product may vary slightly from images below.

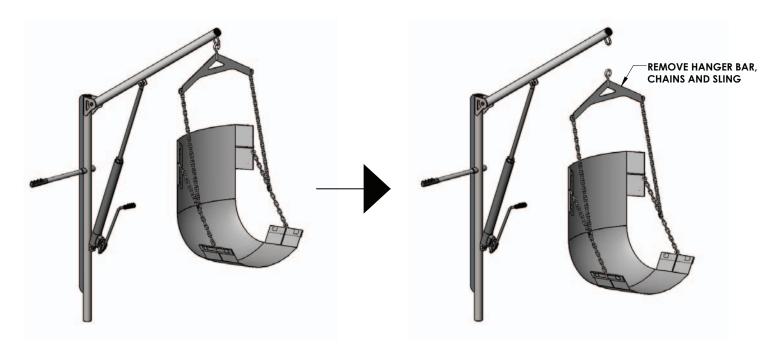

Once the sling seat has been removed, add the quick link to the u-bolt that is attached to the lifts boom arm. Then, attach the 4-point hanger bar to the quick link followed by connecting the chains and hard seat to the 4-point hanger bar. All pools and spas are different so the chains may have to be adjusted slightly to fit specific applications and/or personal needs.

