

THE LEADERS IN RECREATION, FITNESS AND ABILITY!

9889 Garrymore Ln Missoula, MT 59808 888-687-3552 | +1-406-549-0769 www.aquacreek.com



## Ultra Solar Charger Lift Cover

PART #: F-450SLUCS (Blue) PART #: F-450SLUCS-T (Tan) PART #: F-450SLUCS-G (Gray)

**MANDATORY** LEAVE THIS MANUAL WITH LIFT OWNER

Read and follow all instructions. Lift safety can only be ensured if the lift is installed and operated according to these instructions.

- NEVER permit children to play on or around the lift
- Do not allow children to use the lift without adult supervision
- NEVER apply direct water pressure to the electronics
- NEVER use the lift with a dry pool

## READ CAREFULLY

Check entire box and all packing materials for parts. Before beginning assembly, read the instructions and identify parts using the figures and parts listed in this document.

It is critical that all parts be carefully inspected prior to installation. If any damage occurred in transit, Aqua Creek Products, LLC must be notified within three days of receipt of unit.

Proper installation cannot be overstressed, as an improper installation voids Aqua Creek's warranty and may affect the safety of the user.

## Ultra Solar Charger Cover Installation

The Spa Lift Ultra Cover fits over the Ultra lift in the fully raised position. The photo below shows the cover on the lift, and note that the solar charger goes through a slit in the back of the cover. Make sure the cover is turned the right way before putting it over the lift.



2