

OPERATION GUIDE

01

Note: If the hearing aid is silent or the sound becomes low due to the blockage of the sound outlet, please clean the sound outlet.

2. Why scream

Howling is a manifestation of the normal working of the hearing aid. The hearing aid and the ear canal are not completely consistent, causing the sound from the receiver to be received by the microphone again. The sound forms a loop and is continuously amplified, which will cause howling.

05

Before wearing

02

Please use a cotton swab to clean the ear canal before wearing. If the cerumen is too much, it will block the sound outlet of the product and cause no sound.

How to wear

1. Choose earplugs that fit your ear canal size.

2. Install the selected earplugs and the machine

3. Wear the hearing aid as shown in the figure, and rotate the hearing aid to the ear canal as shown by the arrow in the figure

4. Adjust the appropriate volume program
※ When adjusting the volume, do not cover the pickup port

Note: Be sure to choose an earplug that fits the size of your ear canal, and wear it in the correct direction, and try to fit the ear as good as possible. Incorrect wearing will cause howling. The volume and program can be adjusted after the hearing aid is worn! ! !

Problems encountered in wearing

1. There is no sound when wearing?
If there is no sound:
①Check whether the volume program is adjusted to suit your hearing
②Check whether the hearing aid has electricity
③Whether there is no episode in the ear canal, and the earwax is blocking the sound of the hearing aid.

04

Turn ON

After the hearing aid is taken out, it will automatically turn on after 10S and you will hear 2 beeps.

06

※ Please avoid covering the hole when operating the device
※ The first charge should be guaranteed to reach 10–12 hours

Turn OFF

After use, put the hearing aid directly back into the charging compartment to automatically shut down. The product does not support manual shutdown

07

※ Low battery prompt: four fast beeps
※The chip of the charging box has an accurate detection function, and the earphone is taken out for 3 seconds and then returned to the warehouse for accurate charging.

Volume control

08

As shown in the figure: Touch the center of the shell to adjust the volume, there are 5 levels of volume, and the volume level is 1—5 circular adjustment;
A. Tap once, the hearing aid will prompt a "beep" and increase the volume by one level. When three beeps are heard, it means that the volume has reached the maximum level (each tap prompts a "beep")
※ When adjusting the volume, please do not cover the sound pickup hole on it

Recharge

Charging compartment
Flashing green = charging
Steady green light = charging is complete
Hearing aid
Steady blue light = charging
The blue light is off = charging is complete

Hearing aids can be used for 35 hours at a time
The charging compartment can charge a single hearing aid 4–5 times

Charging

Charging completed

09

Charging the battery

1. When charging the hearing aid, it can be charged through the power adapter or through the USB interface
2. Hearing aid charging: first turn off the power of the hearing aid, and then charge it. The first charge should ensure sufficient charging time (not affected by the indicator light), and a charging time of 10–12 hours should be guaranteed
3. If the hearing aid is not used for a long time, the battery self-discharge will cause the battery to be seriously insufficient. It should be charged, and the charging time should reach 12 hours. Even if the charging indicator shows that the charging is complete, the charging should not be stopped (guarantee to reach 12 hours)

10

21-day wear chart

THE JOURNEY TO LISTENING

Unlike vision correction, which simply wears glasses, hearing correction is a rather detailed process. Once you start your hearing correction journey, you will experience countless "new sounds", and your brain needs to adapt for a period of time. In just a few weeks, you will be shocked by the sound that you have never experienced before.

If you have any questions, we will serve you wholeheartedly.

12

THE FIRST WEEK

Some sounds may be too noisy when wearing a hearing aid for the first time, such as the buzzing of the refrigerator, the wrinkling of paper, or the sound of your footsteps. This is normal, and your brain is relearning how to prioritize these sounds that you don't hear very often. Sometimes you may not like the noise, but please don't give up wearing.

Exercise

Listen carefully to a song you are familiar with or watch a TV game show, and guess the answer without looking at the subtitles.

THE SECOND WEEK

The more you wear the hearing aid, the more you can adapt to the sound environment. You even can ask people around you not to speak loudly to you anymore, because you can easily communicate with people.

13

THE THIRD WEEK

After a few weeks, you will feel more energetic because you no longer need to spend a lot of energy to listen what others say. The sound will become clearer and clearer, and you will once again enjoy the sounds that surround you in your life, such as the melodious singing of birds in the backyard, the cheerful laughter of children, and the rustling of leaves in the wind.

Participate

Have a meal in a restaurant with friends, watch a movie in a theater, join a family gathering, and no longer have to bother to speculate about missed information.

15

One-Year Warranty Card

Product Model _____ User Name _____
Date of Purchase _____ Contact Way _____
Contact Address _____

Date	Repairman	Repair Record

We provide a one-year warranty. For any problem about the product quality, please fill in the warranty card, take photos and Email us: (contact@warranty@warranty.com) We will reply to you at the first time to solve your problem.

1)The products provided by the company, non-human damage the implementation of a one-year warranty, after-sale within a month due to product quality problems, unconditional and timely replacement.
2) After the warranty period, the company will follow up from time to time, free of charge to assist in daily maintenance, responsible for hiding maintenance depending on the cost of materials.
3) For quality or maintenance issues, please contact the local distributors.
4)The following circumstances do not enjoy warranty services:
a. artificial damage caused by failure to use, maintain and keep the products not accordance with the instructions for use
b. damage caused by the dismantling of the machine by technicians not related to the company
c. damage caused by force majeure

Note: Due to the continuous optimization of products, parameters may change without notice, the actual appearance of the product shall prevail. The right of interpretation belongs to the company.

16