

# Sockaid™



**NOW AVAILABLE ON PRESCRIPTION**

Call on 01509 501720

Product Code 600000 Pip Code 319-4016



AltiMed Ltd, Sullington Road, Shepshed,  
Loughborough, Leicestershire LE12 9JJ  
Telephone: 01509 501720 Fax: 01509 501721  
Email: [enquiries@altimed.co.uk](mailto:enquiries@altimed.co.uk)  
Website: [www.altimed.co.uk](http://www.altimed.co.uk)

Jan 2011

Sockaid is a trademark of AltiMed Ltd. AltiMed is a part of Uigo Ltd.

**MAKE LIGHT WORK OF FITTING SOCKS AND STOCKINGS**

**PUT  
YOUR  
FOOT  
IN IT!**

**S**ockaid takes away the struggle of fitting surgical stockings and socks. It's so easy to use. Simply peel the garment over the Sockaid and then peel it off onto the limb. Sockaid has been designed for those who have difficulty fitting socks and compression hosiery.



- Easy to use
- Can be used in or out of bed
- Lightweight and robust
- Easy to follow instructions
- Low cost

For best results follow these instructions:

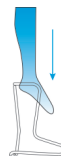


Figure 1



Figure 2

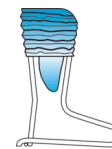


Figure 3

**Figure 1** Place Sockaid on a firm surface and lower garment into it.

**Figure 2** Peel the top of the garment outwards and stretch it over the top rim.

**Figure 3** Pull the top of the garment down the outside of Sockaid and continue peeling the garment onto the appliance, until the heel of the stocking is over the back of the device. There should be just enough room to slip the toes into the stocking.

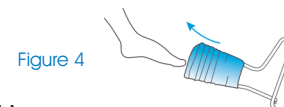


Figure 4

**Figure 4** For surgical stockings-

Seat the wearer safely. Introduce their toes into the open garment, and gently push Sockaid towards them, following the line of their leg until the stocking is completely fitted.

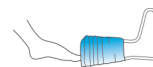


Figure 4a



Figure 4b

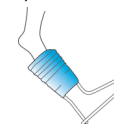


Figure 4c

**Figures 4a-c** For socks and stockings-

Seat the wearer safely. Place the appliance on the floor as shown in **4a** and introduce the toes into the garment. **4b** Draw the foot back carrying the appliance until it reaches an angle as shown in **4c**. Let the foot move forward and as it does so, the garment is automatically fitted to the foot.