




# Zetuvit® Plus

Maximum absorbency.  
Uncompromised comfort.

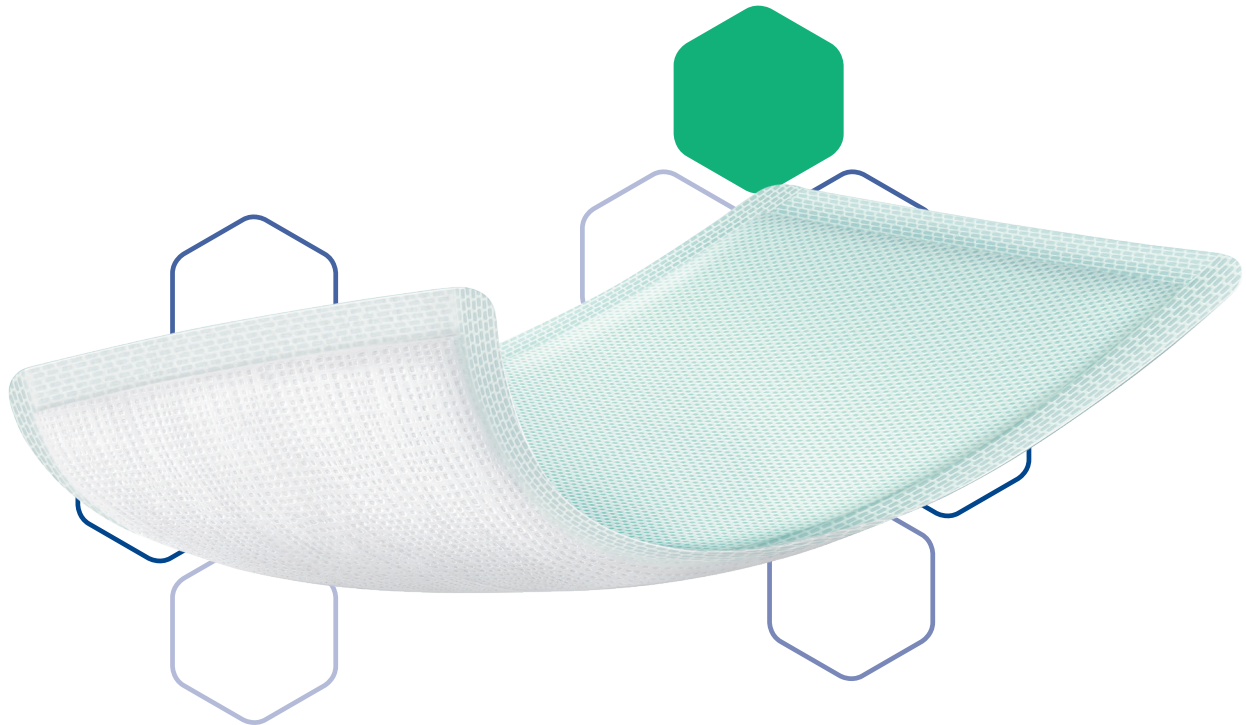


**Zetuvit® Plus** manages a variety of moderately to heavily exuding wounds easily and efficiently.

# Your solution for comfortably managing wound exudate, layer by layer.

Zetuvit Plus superabsorbent dressing is especially suitable for the treatment of moderately to heavily exuding wounds. The four layer dressing construction promotes wound healing by allowing excessive exudate to pass quickly in to a superabsorbent core. Exudate forms a gel inside the dressing resulting in odor control and a barrier to bacteria. The high absorption capacity of the core along with an air permeable outer layer helps reduce the frequency of dressing changes, saving clinician time and expense.


## Four layer construction for optimal performance.

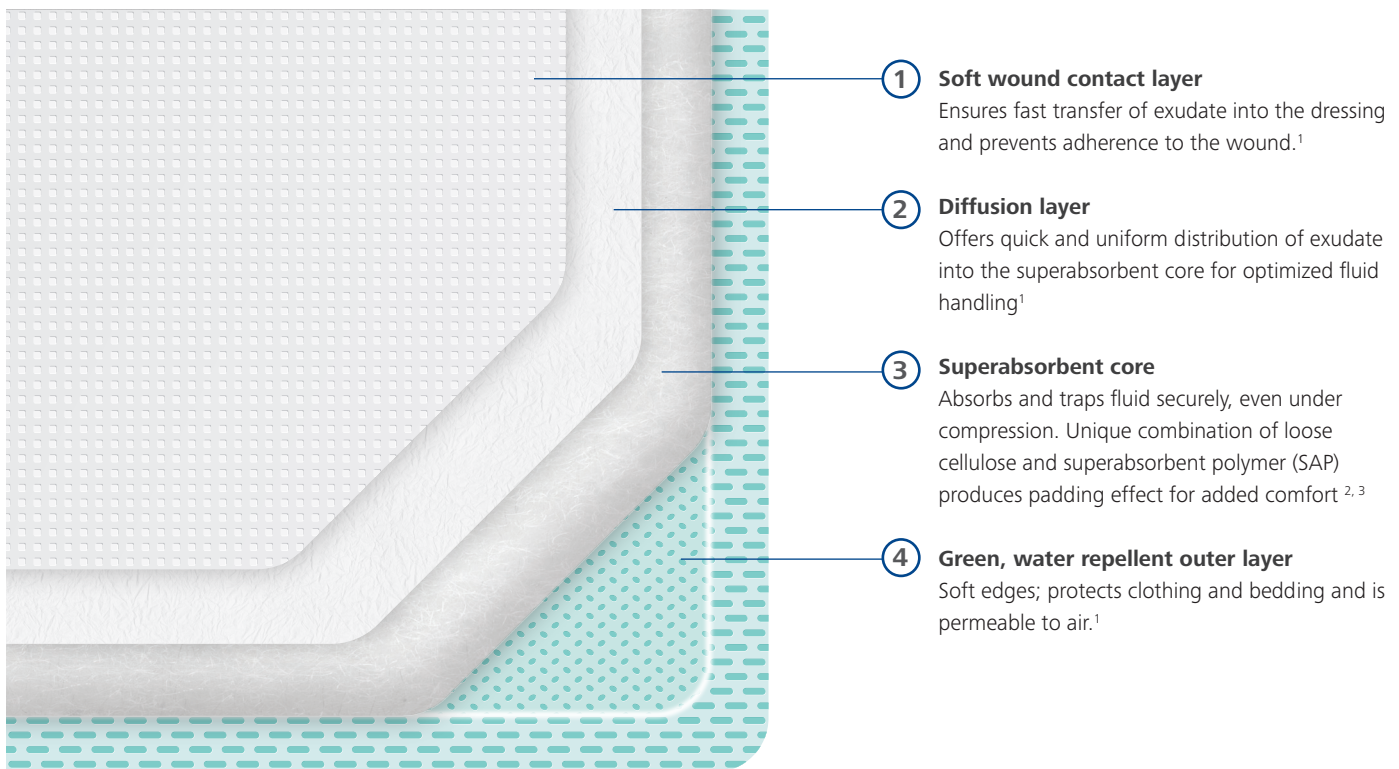

**Excellent patient comfort**  
and satisfaction

Patients report improved quality of life by being able to return to previous activities. Able to bind odor.<sup>2,4</sup>

### Zetuvit® Plus Patient Assessment <sup>2</sup>

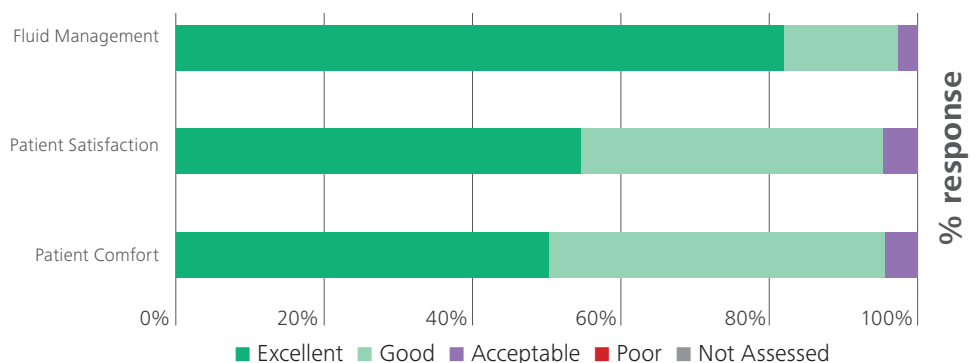




Superabsorbent core supports **fast absorption** even under compression.

Offers excellent absorption and retention when compared to leading competitive products.


### Absorption Levels Blood Substitute Solutions<sup>5</sup>

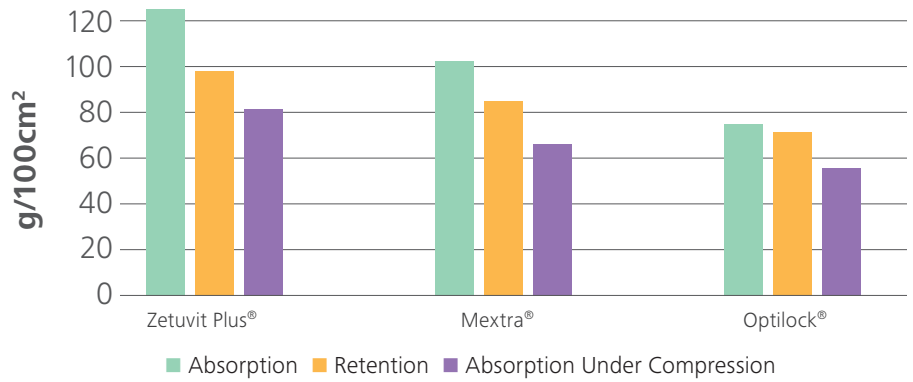

Supports **improved wound healing**

Patients showed a reduction in wound size after treatment and a significant decrease in periwound maceration (65%).


### Changes in Periwound Skin Condition<sup>2</sup>

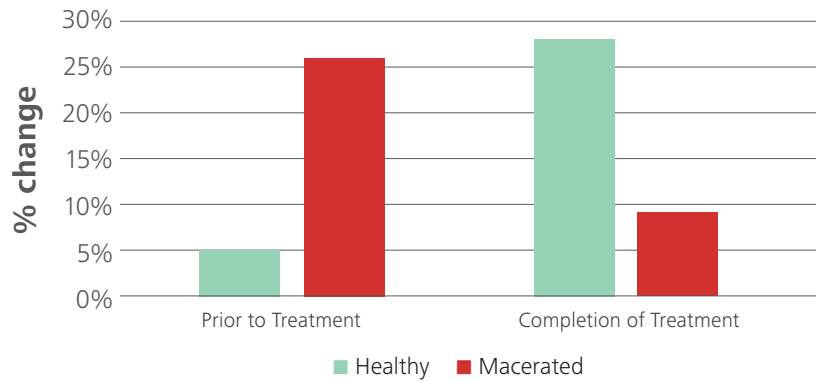

**High clinician satisfaction** and ease of use

Great confidence from nursing staff that dressing effectively manages exudate and is easy to apply and remove.

### Zetuvit<sup>®</sup> Plus Clinician Assessment<sup>2</sup>

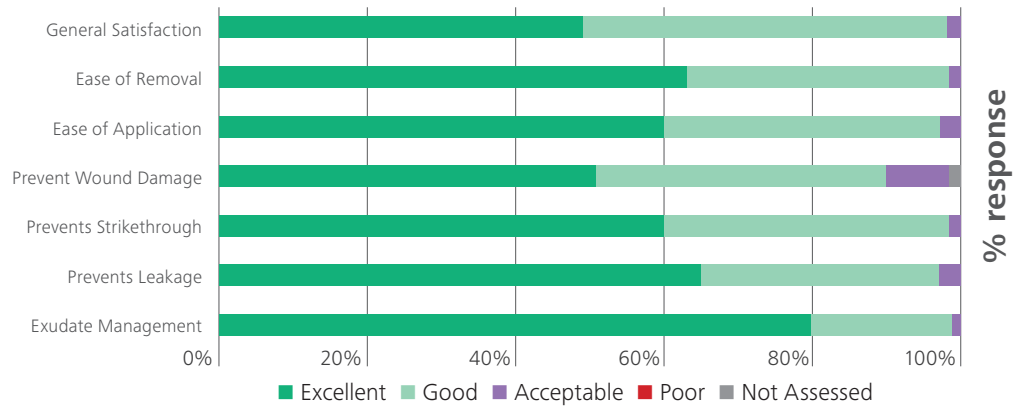

### Frequency of Dressing Change<sup>2</sup>

	Before using Zetuvit Plus	During Zetuvit Plus Study
Several times a day	6	0
Twice a day	4	0
Once a day	<b>48</b>	8
Once every 2 days	18	36
Once every 3 days	20	<b>42</b>
Once every 4 days	0	4
Once every 5 days	0	4
Weekly (7 days)	0	2
Other	0	2

\* Data not available for two patients before study.



**Cost-effective** and time saving

Reduced frequency of dressing changes from 6 to 2.7 times per week.



## Zetuvit® Plus

Order Number	Dressing Size	Dressing Per Box
413110	4" x 4" / 10cm x 10cm	10
413111	4" x 8" / 10cm x 20cm	10
413112	6" x 8" / 15cm x 20cm	10
413113	8" x 10" / 20cm x 25cm	10
413715	8" x 16" / 20cm x 40cm	10

Contact your local HARTMANN representative, or visit us at [www.hartmannusa.com](http://www.hartmannusa.com) for more information.

### References

- <sup>1</sup> Data on file (specification number P.6.1203)
- <sup>2</sup> Barrett, S. et al. (2018). An observational study of a superabsorbent polymer dressing evaluated by clinicians and patients. Journal of Wound Care vol 27, no. 2, February 2018
- <sup>3</sup> G. Kammerlander: Superabsorber in der Wundbehandlung unter Lupe, medical special 03/2013
- <sup>4</sup> Data on file: Davies, LO, Rippon MG and Westgate SJ. (2017) Odour Sequestration Properties of Superabsorbent Dressings Perfectus Paper 2017: Poster presented at Wounds UK. Harrogate, UK
- <sup>5</sup> Data on file: EN\_Benchmark Superabsorbants – Version finale Groupe 2015

Mextra® is a registered trademark of Mölnlycke Health Care. Optilock® is a registered trademark of Medline Industries, Inc.

HARTMANN USA, Inc.  
Rock Hill, SC 29730  
1-800-243-2294

[www.hartmannusa.com](http://www.hartmannusa.com)

XLIT 2862 Rev. 1 (0320)



Going further  
for health