


Long-Lasting Comfort & Protection

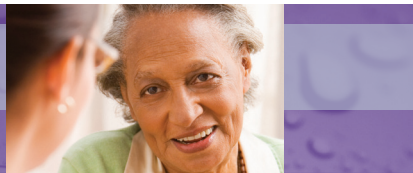

3M™ Cavilon™ Durable Barrier Cream

For the prevention of skin breakdown from bodily fluids and for moisturising skin. Cavilon Durable Barrier Cream is a concentrated, pH balanced, fragrance free moisturising barrier cream for intact or at risk skin. It is very concentrated, so a little goes a long way.

What is it used for?

- Preventing skin breakdown from bodily fluids, e.g. urine and faeces.
- Moisturising and conditioning dry skin.

How is Cavilon Durable Barrier Cream Applied?

- The area of skin to be applied must be clean and dry.
- Use very sparingly in small, pea-sized amounts, and gently smooth in to the area of skin affected.
- If too much is applied, the skin will feel oily.

How often should Cavilon Durable Barrier Cream be re-applied?

- When incontinence is a problem, it should be re-applied after every third or fourth incontinent episode.
- For moisturising dry skin, re-apply approximately every other day

Key points to remember

- It will not clog incontinence pads¹.
- It will still allow dressings and tapes to stick².
- It should not be used on infected skin.

References

1. Hart J. Assessment of the Incontinence Pad blocking potential of 3M Cavilon Durable Barrier Cream compared with Sudocream and Zinc & Castor Oil, Nursing Scotland. July/August 2002.
2. Campbell, K et al. Optimal Skin Care Solutions. A clinical product evaluation, 2000.

Skin and Wound Care Division

3M Australia Pty. Limited
ABN 90 000 100 096
Building A, 1 Rivett Road
North Ryde NSW 2113
Phone 1300 363 878
www.Cavilon.com.au

Skin and Wound Care Division

3M New Zealand Limited
94 Apollo Drive,
Rosedale 0632, Auckland
Freephone 0800 80 81 82
www.Cavilon.co.nz



Take Comfort in Proven Protection



3M™ Cavilon™ No Sting Barrier Film

For the **intervention** against damaged or broken skin. Cavilon No Sting Barrier Film is an alcohol-free liquid barrier that dries quickly to form a breathable, transparent protective film.

What is it used for?

- Protecting broken, tender skin without stinging.
- Protecting red or sore skin caused by incontinence.
- Protecting the skin around stoma sites.
- Protecting skin from damage caused by rubbing, friction or adhesives.
- Protecting skin from damage caused by oozing or discharge from wounds.

How is Cavilon No Sting Barrier Film Applied?


- The area of skin to be applied must be clean and dry.
- **Foam applicator or wipe** – apply an even coat of film to the entire area to be applied.
- **Pump spray** – hold the spray about 10-15cm away from the skin. Spray a smooth, even coating in a sweeping motion over the whole application area.
- Allow the area to dry for 30 seconds.

How often should Cavilon No Sting Barrier Film be re-applied?

- Normally, it should be re-applied every 48 to 72 hours (or every 2 to 3 days).
- If used for adhesion protection re-apply after every dressing or tape change.
- For severe incontinence or when skin is broken, reapply every 12 - 24 hours.

Key points to remember

- If Cavilon No Sting Barrier Film is applied to an area with skin folds or other skin-to-skin contact, make sure that the skin contact areas are separated and allow the coating to dry before returning to normal positions.
- Avoid using ointments or creams as they will stop Cavilon No Sting Barrier Film from working properly.
- Applying too many layers will make the skin feel stiff.

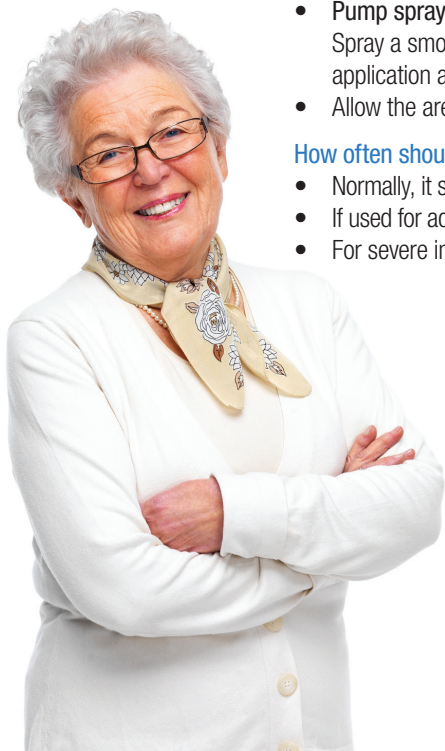

3M and Cavilon are trademarks of 3M.
Please recycle. © 3M 2013. All rights reserved.

