Smith & Nephew Record	
Item:	Ichrhopaste PIL (Global 4 language)
Code:	4959
This BSC/Asset:	42055 / 10601135
Prev BSC/Asset:	18505055 / 37948
Date:	24.04.2013
This Revision:	02

ADAMS DESIGN Record	
Job No:	TT-PAS.01.01
Dims:	210 x 42mm x 10pp
Colours: (Match to Pantone colors)	 ■ Black

(FR)

smith&nephew

ICHTHOPASTE°

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smith&nephew

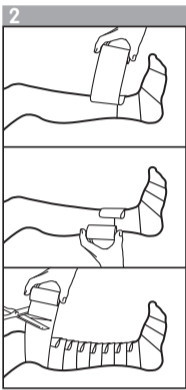
Illustration of the application of the Ichthopaste bandage.

UK Smith & Nephew Healthcare Limited, Healthcare House, 101 Hessle Road, Hull HU3 2BN. Tel: 01462 222200.
AUSTRALIA Smith & Nephew Pty Ltd., 315 Ferntree Gully Road, Mount Waverley, PO Box 242, Victoria 3149. Tel: (03) 8540 6777.
AUSTRIA Smith & Nephew GmbH,Concorde Business Park C3, A-2320 Schwechat. Tel: +43 (0)1 707 91 02.
BELGIUM Smith & Nephew N.V.-S.A., Hector Henneaulaan 366 - 1930 Zaventem. Tel: (02) 702 29 11.
CANADA Smith & Nephew Inc., 2280 Argenta Road, Mississauga, ON L5N 6H8. Tel: 1 800 463 7439.
CHINA Smith & Nephew Ltd, 12F Headquarters Building, 168 Xizang Road (M), Huangpu District, Shanghai 200001. Tel: (21) +86 21 2330 3000.
DENMARK Smith & Nephew A/S, Slotsmarken 14, 2970 Hørsholm, Tel: +45 45 80 61 00.
EASTERN EUROPE Smith & Nephew GmbH, Concorde Business Park C3, A-2320 Schwechat. Tel: +43 (0)1 707 91 02.
FRANCE Smith & Nephew SAS, 25 bd Alexandre Oyon 72019 LE MANS Cedex2. Tél: 02 43 83 23 23.
FINLAND Smith & Nephew Oy, Äyritie 12 C, 01510 Vantaa +358 207 866 300.
GERMANY Smith & Nephew GmbH, Paul-Dessau-Str. 8, Kontor O, 22761 Hamburg. Tel: 040 879744 4.
GREECE Costas A. Papaellinas (Hellas) SA, 26th km, Paeania - Markopoulou Ave, PO Box 182, 19400 Koropi. Tel: 662 6201-7.
HONG KONG Smith & Nephew Ltd., Unit 813-816, 8/F, Delta House, 3 On Yiu Street, Shatin, N.T. Tel: (852) 2648 7700.
INDIA Smith & Nephew Healthcare Pvt. Ltd., 203B, Dynasty Business Park, Andheri-Kurla Road, Andheri (East), Mumbai - 400059. Tel: +91 22-3308 3000 / 4005 5090.
INDONESIA PT Smith & Nephew Healthcare, Total Building, 5th Floor, JI. Let. Jend. S. Parman Kav 106A, Jakarta 11440. Tel: 21 5680521.
IRELAND Smith & Nephew Limited, Carrraig Court, George's Avenue, Blackrock, Co. Dublin. Tel: 01-217 0444.
ITALY Smith & Nephew S.r.l. - via De Capitani 2A, 20864 Agrate Brianza (MB). Tel: +39 039 60941.
JAPAN Smith & Nephew Wound Management KK, 2-4-1 Shiba Park, Minato-ku, Tokyo 105-0011 Japan. Tel: 03-5403-8930.
KOREA Smith & Nephew Ltd, 6th Floor, Kwangmyung Building, 237-2 Neung-dong, Kwangjin-ku, Seoul, Korea (143-180). Tel: 2343 60460.
MALAYSIA Smith & Nephew Healthcare Sdn Bdh, Menara Merais, 9th Floor, 1, Jalan 9/13, Petaling Jaya 46300, Selangor. Tel: 03-7958 7103.
MALTA Cherubino Ltd, Delf Building, Sliema Road, Gzira. Tel: 21 343270.
MEXICO Smith & Nephew, SA de CV, Patricio Sanz No. 442, Colonia del Valle, - Bloco C, 2625-445 Forle da Casa, Villa Franca de Xira. Tel: +351 214 460 650.
SINGAPORE Smith & Nephew Pte. Ltd., 1 Jalan Kilang, Timor, #08-03/05 Pacific Tech Centre, 159303. Tel: 270 0552.
SOUTH AFRICA Smith & Nephew Pharmaceuticals (Pty) Limited, 30 The Boulevard, West End Office Park, Westville, 3629. Tel: 031 7108111.
SPAIN Smith & Nephew SAU, Fructuos Gelabert, 2-4, 08970 Sant Joan Despi, Barcelona. Tel: 93 373 73 01.
SWEDEN Smith & Nephew AB, Box 143, Kråkorsgatan 20, 43122 Mölndal. Tel: 031-746 58 00.
SWITZERLAND Smith & Nephew AG, Glutz Blotzhelm-Str.1, CH-4502 Solothurn. Tel: 32 624-5660.
TAIWAN (R.O.C.) Smith & Nephew (Overseas) Ltd., Taiwan Branch, 9F-3, No. 130, Sec. 2, Jhongsiao East Road, Taipei City. Tel: 886 2 3322 5020.
THAILAND Smith & Nephew LTD, 344/3 Soi Rongrien Yupun, Rama IX Road, Kwang Bangapi, Khel Huay Kwang, Bangkok 10310. Tel: 2 719 6221-27.
TURKEY Taymed Sağlık Ürünleri Ticaret Ltd.Şti. Büyükdere Cad., Caýrıcırn Sok. Emlak Kredi Bloklan, A-1 Blok, D-10-11 Levent/Istanbul. Tel: +90 212 324 2190.
UNITED STATES Smith & Nephew, Inc., Wound Management Division, 970 Lake Carillon Drive, Suite 110, St. Petersburg, FL 33716.
Customer Care Centre: 1 800 876-1261 or 727 392-1261.
VENEZUELA Ace Sales Corp, 8085 NW 68th Street, Miami, Florida 33166 Tel: 305.835.0310.

(EN)

smith&nephew

ICHTHOPASTE°

Zinc Paste and Ichthammol Bandage BP

Single use. Store in a dry place (<25°C/77°F).

🚫 Do not use if pouch is opened or damaged.

Description

ICHTHOPASTE is a bandage made of open wove cloth impregnated with a paste containing 6.32% Zinc Oxide BP and 2% Ichthammol BP rolled onto a plastic core. It also contains Purified water BP, Glycerine BP, Gelatin BP, Emulsifying wax BP, Phenoxyethanol BP. Each bandage is presented in a wrapper and sealed in a polythene bag contained in a cardboard carton.

Mode of action

ICHTHOPASTE provides topical treatment. It provides a moist wound healing environment over a venous leg ulcer, helps to reduce skin irritation and soothes and protects surrounding skin. It will not dry out in use.

Indications

ICHTHOPASTE is indicated to assist the management of leg ulcers. Where venous insufficiency exists, the paste bandage should be used under graduated compression bandaging. ICHTHOPASTE is also suitable for use in the management of chronic eczema/dermatitis, where occlusion is indicated.

Contra-indications

Do not use in known cases of sensitivity or allergy to any of the ingredients.

Precautions

Where venous insufficiency exists, ICHTHOPASTE should be used under graduated compression bandaging, after first assessing the patient to exclude arterial disease. The use of Doppler ultrasound is recommended for this purpose. Compression bandaging should not be used on patients who have arterial disease (ABPI < 0.8) and is not recommended for use on diabetic patients with advanced small vessel disease. **Failure to detect reduced arterial flow can result in pressure necrosis, amputation, or even death. The risk of a patient having arterial as well as venous disease rises with age.** The skin of leg ulcer patients is easily sensitised to topical medications, including preservatives. Sensitisation should be suspected in patients particularly where there is deterioration of the surrounding skin. Such patients should be referred for specialist diagnosis, including patch testing. One of the functions of occlusive bandages is to increase absorption. Care should be taken, therefore, if it is decided to apply topical steroid, anaesthetic or retinoid preparations under the bandage, as their absorption may be significantly increased. This may lead to a shorter duration of effect of a topical anaesthetic product in addition to increasing the risk of sensitisation to these preparations. For topical use only. ICHTHOPASTE is a single use only product. If used on more than one patient, cross-contamination or infection may result. Once the packaging has been opened, unused material should not be retained for application at a later date.

Instructions for use

This product is ready for use upon removal from the sealed pouch and wrapper. There are two ways that the bandage can be applied:

- Beginning at the base of the toes, the bandage should be *loosely wrapped* around the foot, heel and around the leg in a spiral fashion, to just below the knee. Once applied, the bandage should then be smoothed and moulded around the leg.
- Beginning at the base of the toes, the bandage should be *loosely wrapped* around the foot and heel and then, whilst wrapping from the ankle, with every turn, the bandage should be folded back on itself in a pleat, at the front of the leg. This should be repeated up the leg until just below the knee. Compression bandaging may follow. Once ICHTHOPASTE has been applied, the leg should be covered by a bandage or dressing to prevent soiling to clothes.

Recommended duration of use adults and children

Change bandage as required, can be worn up to 7 days.

Additional information

Ichthammol (of itself, not related to the occlusive nature of the product) may increase the absorption of corticosteroids, heparin and chloramphenicol through the skin. Ichthammol has also been shown to synergise the activity of oxytetracycline and antagonise the activity of ampicillin when present together in various ointment bases.

Product Availability

Code	Size	Availability
4959	7.5cm x 6m	Carton of 1

Smith & Nephew Medical Limited, 101 Hessle Road, Hull HU3 2BN England
*Trade Mark of Smith & Nephew

©Smith & Nephew
Date of issue 04/2013
Made in China

NPN 01948466

CE
0086

42055 10601135