

RENASYS TOUCH – Quick Reference

(refer to the User Manual for full details on RENASYS TOUCH)




1. Switching the device between Continuous and Intermittent therapy

- To switch between Continuous and Intermittent therapy, ensure therapy is paused and press the therapy mode toggle on the home screen



Continuous therapy home screen

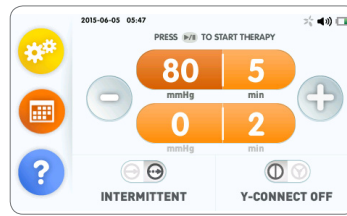

Intermittent therapy home screen

Therapy mode toggle


2. Starting therapy (Continuous)

Step 1: Ensure that the device is powered on and the Continuous therapy home screen is displayed

Step 2: Use the + or – buttons to select prescribed pressure setting

Step 3: Press the Start/Pause therapy button to start delivering therapy

When therapy is active the status indicator on top of the device illuminates green and the therapy indicator at the top of the screen will rotate orange

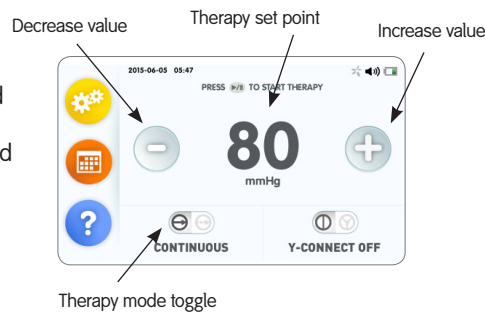

Therapy mode toggle



Start/Pause therapy

3. Starting therapy (Intermittent)

Step 1: Ensure that the device is powered on and the Intermittent therapy home screen is displayed

Step 2: Change the High and Low pressure settings and cycle times by selecting the setting and use the + or – to the prescribed pressure/time

Step 3: Press the Start/Pause therapy button to start delivering therapy

When therapy is active the status indicator on top of the device illuminates green and the therapy indicator at the top of the screen will rotate orange



All 4 settings can be adjusted within the following pressure ranges:

High Therapy
25, 40, 50, 60, 70, 80, 90, 100, 120, 140, 160, 180, 200mmHg

High Cycle times: 3, 5, 8, 10 minutes

Low Therapy
0, 25, 40, 50, 60, 70, 80, 90, 100, 120, 140, 160, 180mmHg

Low Cycle times: 2, 3, 5, 8, 10 minutes

4. Checking for a seal

Follow on screen seal checking instructions. Once seal is achieved the screen will display “seal achieved” and then display “delivering therapy”

Look for “raisin-like” appearance

Listen for a “hissing” sound, indicating a leak

Feel the dressing, which should be hard to the touch


Possible site of leak

- Wrinkle or crease
- Skin fold or crevice
- Where drain exits film (where applicable)

Intervention

Seal film edges with film dressing

If hole found in Transparent Film, patch with adhesive Transparent Film.

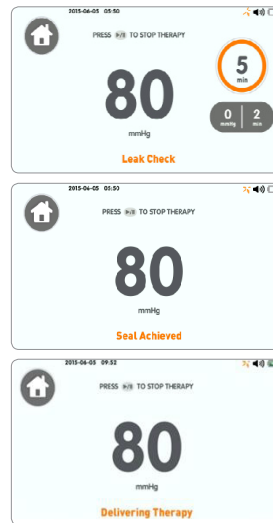

5. Locking the screen

The Lock function locks the touchscreen and Start/Pause therapy icon – Power icon is not locked. The status indicator continues to illuminate green to indicate the device is still delivering therapy




When delivering therapy, the screen will go dark. To view screen, press either Play or Lock pad button. In the event of an alarm, the device will automatically unlock and the Alarm screen will display. The device will automatically lock and enter Sleep mode after 5 minutes without user interaction.

6. Switching the device between Patient and Clinician mode

Step 1: Press the yellow settings button to access the settings screen




Step 2: Press the change mode option from the menu

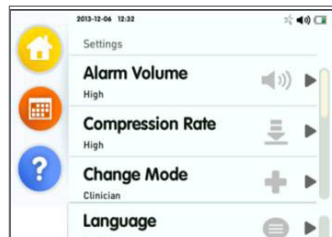

Step 3: Select which mode (Clinician/Patient) required



Step 4: Please enter the password (3141) and press the green tick to switch modes

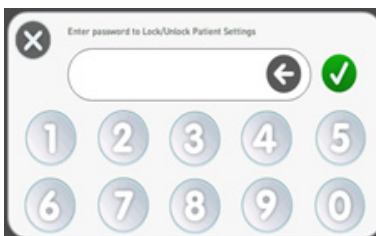

Step 5: Press yellow button on screen twice to return to the home screen



Supporting healthcare professionals for over 150 years

Smith & Nephew, Inc.
Fort Worth, TX
USA

www.smith-nephew.com
www.renasy-npwt.com

Customer Care Center
1-800-876-1261
T 727-392-1261
F 727-392-6914

QUESTIONS?

**CALL NPWT CLINICAL AND PRODUCT SUPPORT LINE,
VIA CUSTOMER CARE AT 1-800-876-1261**

For detailed product information, including the indications for use, contraindications, precautions and warnings, please consult the product's Instructions for Use (IFU) prior to use.