



# UrgoTul Absorb Border



## ADHESIVE FOAM DRESSING WITH SOFT-ADHERENT TLC

**Indication:** moderately to heavily exuding wounds

- Optimal healing result
- High absorption of exudate and drainage preventing any risk of maceration
- Waterproof silicone adhesive on the edges 
- Pain-free and atraumatic care

## UrgoTul Absorb Border

Supplied in boxes of individually pouched and sterile dressings, ready to use

Sizes	
	8 x 8 cm
	10 x 10 cm
	12 x 12 cm
	13 x 13 cm
	15 x 15 cm
	15 x 20 cm
	Sacrum 20 x 20 cm



# UrgoTul Absorb Border



## DESCRIPTION

■ **UrgoTul Absorb Border** is an adhesive lipido-colloid foam dressing with soft-adherent TLC (Technology Lipido-Colloid), an innovative Technology developed by the Laboratoires Urgo.

### COMPOSITION:

- A soft-adherent TLC layer (polymers and hydrocolloid

particles) combined with an absorbent polyurethane foam pad and a highly absorbent layer.

- A vapour permeable waterproof outer film with silicone adhesive on the edges.

**METHOD OF STERILIZATION:** Sterilized by ethylene oxide.

## PROPERTIES

- Soft-adherent TLC layer
- Optimal healing result
- Moist environment
- Pain-free dressing changes
- Fibroblasts proliferation
- Absorbent polyurethane foam pad and highly absorbent layer
- High fluid management, prevents maceration
- Suitable for use under compression bandaging
- Waterproof and silicone adhesive backing
- Soft and non-occlusive

- Very conformable to fit different wound shapes
- High vapour permeability to prevent maceration
- Good tolerance
- No need for secondary retention

### ■ Mode of action

• In contact with wound exudate, TLC forms a gel and creates a moist wound environment, beneficial for the healing process enabling the key cells involved in the repair process (fibroblasts, keratinocytes and macrophages) to exert their action.

## INDICATIONS

■ **UrgoTul Absorb Border** is indicated for moderately to heavily exuding acute wounds (such as burns, dermabrasions, traumatic wounds, postoperative wounds...) and chronic wounds (such as leg ulcers,

pressure ulcers, diabetic foot ulcers).

- The sacrum version is recommended for exuding wounds located in the sacral area (sacral pressure ulcers...).

## DIRECTIONS FOR USE

### METHOD OF USE

- Clean the wound as per local protocol.
- If an antiseptic is first used, carefully rinse the wound with normal sterile saline before applying **UrgoTul Absorb Border**.
- Dry the surrounding skin carefully.
- Remove the protective wings.
- Apply the soft-adherent side of **UrgoTul Absorb Border** to the wound (the adhesive silicone border should be at least 1 cm away from the wound).
- Smooth the dressing over the wound.
- Apply compression bandaging over the dressing when prescribed.
- To apply the sacrum version, place the dressing with the tip facing the sacral area.
- To remove **UrgoTul Absorb Border**, press down on the healthy skin, lift on the corner of the dressing and remove it carefully.
- **UrgoTul Absorb Border** dressing may be changed every 2 to 4 days, and left in place up to 7 days depending on the level of exudate and the condition of the wound.

### PRECAUTIONS

- As it includes a super-absorbent layer, the central pad should not be cut. However, the adhesive edges can be cut if necessary using sterile scissors to fit the dressing to different wound shapes.
- If clinical signs of local infection are noted, treatment can be changed to an antibacterial dressing from the Urgo range, dependent on clinical judgement.
- Any excess hair should be cut close to the skin to ensure good contact with the wound.
- In case of concomitant use with cream, an ointment, an emulsion, let the skin dry before the dressing application.
- Single use sterile individual packaging: re-using a single use dressing may lead to risks of infection.
- Do not re-sterilise the dressing.

### CONTRAINDICATION

- Do not use if there is a known sensitivity to **UrgoTul Absorb Border** or its components.