

PREMIUM LEAF SPRING AFO FOR DROP FOOT



APPLICATION INSTRUCTIONS

- FITTING THE AFO TO THE SHOE
 - If present, remove the insole from the shoe.
 - Adjust the shape of the footplate by grinding off excessive material.
 - Make sure the plate covers the entire surface of the shoe sole to prevent displacement in the shoe.
 - Ensure stability of the footplate on the shoe sole throughout the entire length of the shoe.
- CUSTOMIZING THE AFO
 - Insert the AFO into the shoe and have the patient don the AFO. If desired, temporarily attach the padding material.
 - Check for excessive pressure points and, if necessary, remove the AFO from the patient's leg and apply minimal heat (<90°) to reshape the AFO to relieve these points.
- FINISHING THE AFO
 - If desired, permanently attach the padding to the AFO.
 - The AFO is now ready for use.

WARNING

- The AFO is a supportive brace only and is not intended or guaranteed to prevent injury. SourceMED accepts no responsibility for any injury which might occur while using this product.
- With the use of this type of device, it is possible to receive various types of skin irritation and or pressure points. If you experience any unusual discomfort while using this products, consult your health care provider immediately.

Part#	Height	Leg	Size	Foot Plate
SM-110129	10.75"	Right	Small/Medium	9.5"
SM-110229	10.75"	Left	Small/Medium	9.5"
SM-110132	12"	Right	Large	11"
SM-110232	12"	Left	Large	11"

