SALJOL

Duschhocker Spa

Gebrauchsanleitung | User manual

SALJOL

www.saljol.de

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Preliminary introduction

Dear customer,

congratulations on your purchase: With the rotating shower stool Spa you shower safely and comfortably. We invite you to discover your new spa shower stool piece by piece on the following pages.

Familiarize yourself with the many functions and applications and use the tips. We wish you much pleasure with your spa.

Your team from Saljol

Do you have questions or even your own ideas?
Just let us know: hallo@saljol.de or +49 800 50 50 269

Content

1. Identification	14
2. Declaration of conformity	14
3. Display of safety instructions	15
4. Product information	15
4.1 Product overview	15
5. Instruction manual	16
6. Technical data	17
7. Cleaning and disinfection	18-19
8. Mounting the backrest	20
9. Warranty	21

1. Identification

Identification data:

Manufacturer: Saljol GmbH
Product: Spa
Type: Shower stool
Article designations: Shower stool Spa
Version: 2019

Manufacturer:
 Saljol GmbH
 Frauenstr. 32
 82216 Maisach
 Phone: +49 800 50 50 269
 fax: +49 8141 317 74 11
 hallo@saljol.de
 www.saljol.de

Information corresponds to the status at time of printing.
 Subject to technical changes.
 Technical drawings and photos may differ from reality.

© Copyright Saljol
 All rights reserved.
 Reprinting, even in extracts, only with permission
 of the company Saljol GmbH.

Formalities of the operating instructions:
 Version/Revision: 2019.1
 Creation date: 03/2019

2. Declaration of conformity

Compliant with Directive 93/42 EEC and tested to ISO 17966 - Personal hygiene aids to assist users. A CE declaration of conformity can be found on our homepage www.saljol.de

The photo shows the position of the two labels.

 <p>SALJOL</p> <p>Model: Spa shower chair for the bathroom Max. user weight: 150 kg Max. width S: 502 mm</p>	Saljol GmbH Frauenstraße 32 82216 Maisach 2020-01	 <p>Seat height: 460 mm Turning circle: 470 mm Max. load seat: 150 kg</p>
 <p>SALJOL</p> <p>Model: Spa shower chair for the bathroom Max. user weight: 150 kg Max. width M: 512 mm</p>	Saljol GmbH Frauenstraße 32 82216 Maisach 2020-01	 <p>Seat height: 500 mm Turning circle: 480 mm Max. load seat: 150 kg</p>
 <p>SALJOL</p> <p>Model: Spa shower chair for the bathroom Max. user weight: 150 kg Max. width L: 522 mm</p>	Saljol GmbH Frauenstraße 32 82216 Maisach 2020-01	 <p>Seat height: 540 mm Turning circle: 490 mm Max. load seat: 150 kg</p>

All illustrations, dimensions and price quotations, subject to printing errors. Tolerances, colour deviations and innovations within the scope of technical implementation cannot be excluded.

3. Presentation of safety instructions

The safety instructions are marked twice: By a symbol and by a signal word. The symbol informs you about the type of note. The signal word explains further details, for example, the severity of an imminent risk.

Caution

Potentially dangerous situation
(slight injury or damage to property)

Note

Application tips and particularly useful information.
Important Obligation to behave in a special way or to perform an activity for the safe handling of the shower stool.

Warning of specific risks

These are risks associated with the construction and function of the shower stool.

4. Product information

4.1 Product overview

5. Instruction manual for the Spa shower stool

Stainless steel and powder coating

Get to know the product

Place the Spa shower stool on the floor and take a seat for a first sitting test. Turn from left to right together with the shower stool while keeping both legs on the floor. If you are being cared for, hold on to the handles. The person caring for you can turn you gently. For all-round care.

Turn on the spot




The wheels of the shower stool are arranged in a circle and equipped with a non-slip rubber coating. Hard to reach parts of the body can therefore be easily cleaned and cleaned. to be achieved. You have the choice to place your feet comfortably on the frame or to place them on the floor for a secure feeling.

Seat width adjustment

The hygiene opening between the warmer reflecting seats of your Spa Shower- and hockers can be adapted to your physique without tools. To do this, place the stool on its side as described on the left and move the seats according to your wishes. The three seat widths 1, 2 and 3 engage perceptibly. Please note that both sides must be set to the same level.

Warning Risk of injury!

In case of slight dizziness, please hold on to the side handles.

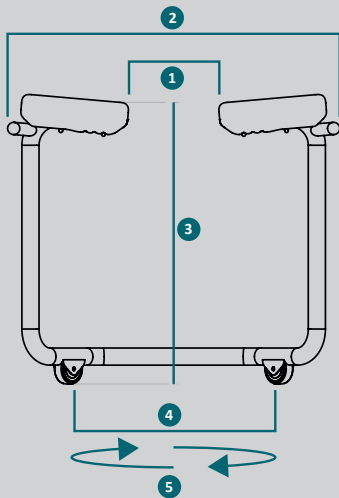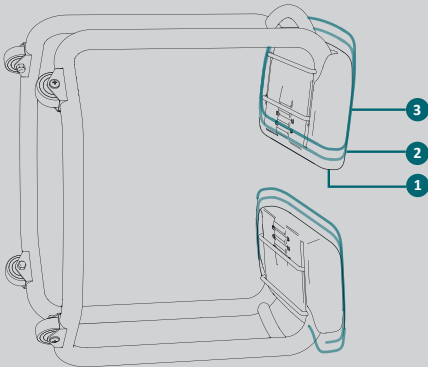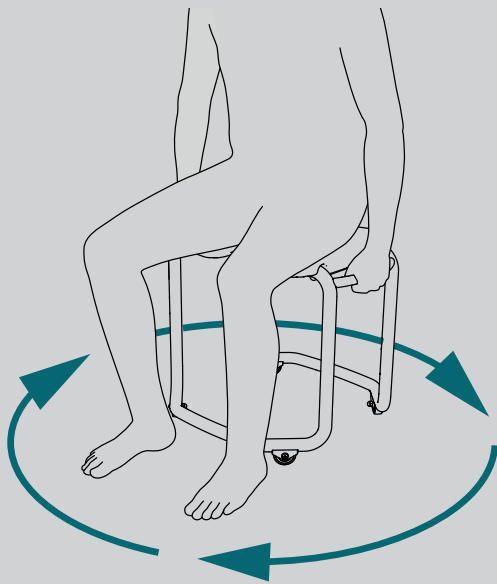	S	M	L
1. Hygiene cutout max.	108 mm	118 mm	128 mm
Hygiene cutout min.	48 mm	58 mm	68 mm
2. Total width max.			
from handle to handle	502 mm	512 mm	522 mm
3. Height	460 mm	500 mm	540 mm
4. Wheel distance	301 mm	311 mm	321 mm
5. Turning circle Average	470 mm	480 mm	490 mm
Wheels outside			

Maximum load capacity: 159kg

Appropriate use

The spa is intended exclusively as an aid to showering and for use in the bathroom for people with the following limitations:

- Uncertainty and/or inability to stand
- Diseases that affect the stability

Any other use is not permitted.

Version: stainless steel

The spa shower stool comes in a very common alloy; the 304 stainless steel alloy with 18% chrome and 8% nickel. Thanks to its high strength, excellent corrosion resistance and minimal carbon content, the 304 stainless steel alloy is ideal for wet applications.

Version: KTL and powder coating

Cathodic dip coating (CDC) is an electrochemical process in which the spa is coated in a bath of aqueous dip paint under a DC voltage of 3,000 amperes and 220 to 290 volts.

During powder coating, paint powder is electrostatically charged and applied to the material with a powder gun. A prerequisite is that the material to be coated is electrically conductive, the spa is made of steel. The paint coating is then baked in the baking oven at a temperature between 140° and 200° C. In advance, the material is thoroughly derusted, degreased, blasted, etc. - pre-treatment of the surface is very important for the quality of the powder coating.

6. Technical data

Size	S	M	L
Adjustable hygiene opening: Stage 1 Stage 2 Stage3	48 mm 78 mm 108 mm	58 mm 88 mm 118 mm	68 mm 98 mm 128 mm
Total width max. from handle to handle	502 mm	512 mm	522 mm
Seat height	460 mm	500 mm	540 mm
Height with backrest	768 mm	808 mm	848 mm
Wheelbase	301 mm	311 mm	321 mm
Turning circle Ø - Space requirement	470 mm	480 mm	490 mm
Load capacity / max. user weight	150 kg	150 kg	150 kg
Weight Spa stainless steel	3,8 kg	4 kg	4,2 kg
Weight Spa powder coated	4,6 kg	4,8 kg	5,1kg
Weight backrest	1,8 kg	1,8 kg	1,8 kg
Colour- / material variations	<ul style="list-style-type: none"> • Stainless steel • Powder coating, anthracite 		

7. Cleaning and disinfection

Your spa shower stool, with a highly polished or powder-coated surface and welded seams, cannot remain protected from harmful environmental influences despite the careful and high-quality selection of materials. However, regular care reduces the risk considerably and guarantees an optimal value retention and optical appearance for a long time. If this does not happen, blind spots can be the result.

The following applies to the stainless steel version:

Should small corrosion spots occur, it is usually so-called "flash rust" which affects the product. This occurs when metallic particles adhere to the stainless steel and slowly "eat into" it. Insufficient care accelerates this process.

This damage to the surface can be removed with a polish or with a fine grinding and polishing paste. We expressly point out that our warranty only applies if this care has been demonstrably carried out! For this stainless steel care we recommend the product of the company Durosol: Autosol Edelstahl Chromglanz/Metal Polish.

The product can be cleaned and disinfected with commercially available products.

i Note: Disinfect the product by wiping all generally accessible surfaces with a large area disinfectant. If necessary after disassembly of the product! All detergents and disinfectants used must be effective, mutually compatible and gentle to the materials used.

- Clean the product by hand.
- Do not use highly aggressive (pH value less than 4.5 or greater than 8.5) or abrasive agents.
- Cleaning temperature max. 85 °C for 3 minutes.

When maintaining the frame, the following must always be observed:

- Cleaners containing hydrochloric acid, formic acid, sodium hypochlorite, phosphoric acid or acetic acid may not be used.
- Do not use abrasive cleaning aids and equipment such as unsuitable abrasive cleaners, pad sponges and microfibre cloths.
- The instructions for use of the cleaning agent manufacturers must be strictly followed.
- The build-up of calcification must be prevented by regular cleaning. To do this, also remove the seat surfaces.
- After cleaning, rinse sufficiently with clear water to completely remove any remaining product adhesions (cleaner).
- Personal care product residues such as liquid soaps, shampoos and shower gels can also cause damage. After use, rinse residues carefully with clear and warm water.

When caring for the seat and backrest surface made of EPP (expanded poly-propylene), it is essential to observe the following:

- EPP is flame-retardant, odourless and insoluble in water.
- It is easy to clean, sterilise and dishwasher safe at a maximum water temperature of 85°C.
- Clean the EPP parts every 2-4 months.

Maintenance and periodic inspection:

If the above cleaning instructions are followed, the product is maintenance-free.

- Check the Spa shower stool regularly for damage and safe installation.

Product Lifetime:

If the product is used in accordance with the safety regulations and the cleaning and care instructions are followed, the expected product life is five years. However, the effective life may vary depending on the frequency and intensity of use.

Warning! Flammability

The Spa shower stool is made of fire resistant materials (EPP seats and frame) as well as non-fire resistant materials such as the wheel cover.

With the EPP seat plates of 60g/l density used by us, we can make the following statements regarding fire resistance. According to ISO 3795 (FMVSS 302) the product has a burning rate of 40mm/min. According to the UL94 equipment regulation, the product achieves class HBF. According to EN 13501-1, the product achieves fire resistance grade E.

8. Mounting the backrest

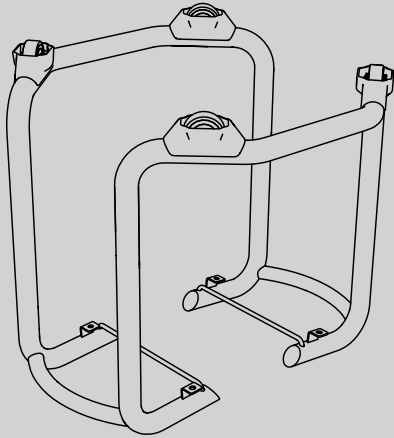
Figure 1

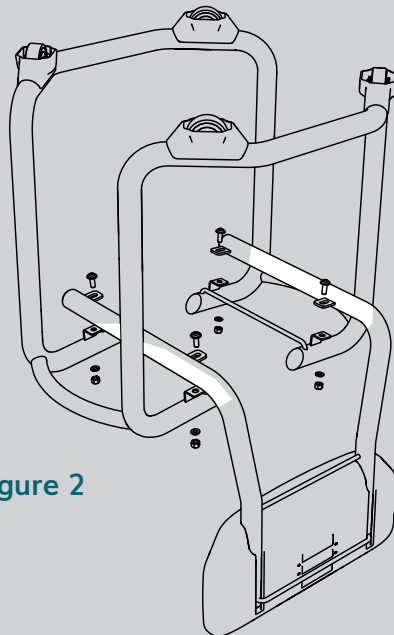
Figure 2

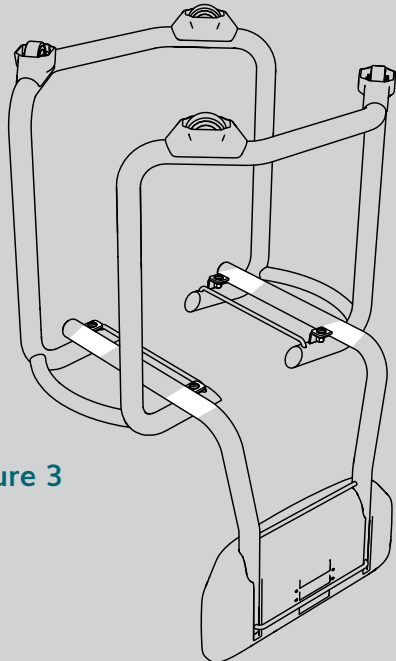
Figure 3

WARNING

Crash hazard!

- Do not stand on the product.
- Do not use as a climbing aid.

Max. Operating temperature:
10 - 40 °C

Max. User weight: 150 kg

Backrest assembly

Remove the two EPP seat plates from the frame and place it upside down on the edge of a table.
(see figure 1)

Screw the backrest to the bottom side of the frame using the four screws.
(see figure 2)

Note: Make sure that the unit is mounted correctly. (see figure 3)

Warning Risk of injury!

Make sure that the backrest is firmly screwed together with all four fixing points.

Note for people with impaired vision.

We would be pleased to send you a user manual in larger written form as PDF or in paper form. Please call us at this number **+49 8141 317 740** or send us an e-mail: **hallo@saljol.de**

9. Warranty

Saljol gives a 2-year guarantee against production or material defects.

Register the shower stool Spa by Saljol in your name and we extend your personal warranty period to 5 years. www.saljol.de/mysaljol

Damage caused by improper handling is not covered by our warranty.

Warranty conditions

All defects of the Spa shower stool are subject to warranty, provided that these defects are demonstrably due to material or manufacturing defects.

i Important!

If you have any complaints, inform your specialist dealer immediately! Technically qualified fitters or instructed operating personnel find the batch number on the metal frame.

1. The warranty is not transferable
2. If your complaint is justified, you will receive the spare parts for your repairs free of charge within a period of 24 months from delivery.
Supplier is your specialist dealer.
3. The warranty for the Spa or individual parts expires if:
 - If the Spa is serviced and maintained by anyone other than the person we specify.
 - if parts are mounted on the spa that are not original parts or those which have not been released by the manufacturer
 - someone uses the spa improperly or intentionally damages it

Excluded from the warranty

The warranty and any general product liability expires if

- the operating instructions are not followed
- repairs and maintenance are carried out improperly
- Technical changes and additions are made to the Spa

Also excluded from the warranty are

- Damage caused by misuse, overloading or overstressing
- Damage caused by cleaners
- Impact damage, as typically caused by rubbing against the wall or tiles
- Damage or injury caused by cigarettes, hair dryer or open fire
- The EPP seat plates of the shower stool Spa

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Saljol GmbH | Frauenstraße 32