



American Hospital Supply Pressure Infusion Bag

Product Information

Item	Description	Volume	Quantity Per Box	Quantity per Master Carton
AHSPIB500	Pressure Infusion Bag with Hook	500cc	10 each	10 boxes
AHSPIB1000	Pressure Infusion Bag with Hook	1000cc	10 each	8 boxes
AHSPIB3000	Pressure Infusion Bag with Hook	3000cc	10 each	8 boxes

Overview

AHS pressure infusion bag provide a trusted means of delivering intravenous, intra-arterial fluid therapy, and pressure monitoring diagnostics. Each infusion bag is equipped with a hook for easy attachment to an IV pole, a 3-way stopcock for pressure control, an oval inflation bladder, and a color-coded gauge to ensure accurate monitoring of patient pressure.

Instructions for Use

1. Insert intravenous bag between mesh/panel and AHS pressure infusion bag.
2. Hang intravenous bag on mounted hook or slide hanger loop through top of IV bag if applicable.
3. Inflate: Close valve on inflation bulb. Pump inflation bulb until gauge indicates the proper pressure as indicated by your pressure monitoring or infusion therapy.
4. Rotate stopcock to “Maintain Pressure” position.
5. Deflate: Open valve on inflation bulb. Caution: For best transfer of pressure to the IV

Statements

Caution: To ensure best results make sure the bag is placed within the pressure infuser panel.

Warning: Pressure of the fluid bag will decrease as fluid is evacuated from it, monitor the gauge pressure continually and inflate to ensure consistent pressure.

Intended for single patient use. Product includes space marked "patient info" for identifying information.



DEHP-free



American Hospital Supply, LLC
 2219 Main Street #464
 Santa Monica, CA 90405
info@american-hospitalsupply.com
 (888) 621-5011