

## When to replace your sleeve?

Replace your sleeve as needed, when it becomes discolored from use, or every 4-6 months.

## Instructions:

1. Hold your LEFT “old” Earaser...marked with a blue line, between your thumb and forefinger (see Fig. A).
2. Insert the filter removal tool into the large opening of the Earaser (follow the string) until you feel the small extended tip engage with the filter. (Fig. B)
3. **DO NOT PULL THE STRING TO REMOVE THE FILTER FROM THE SLEEVE!** Push the filter out of the tip of sleeve, pushing one way, towards the small tip of the Earaser (note the alignment of the color stripe on the filter). Remove the filter from the tool.
4. Remove the tool from the “old” sleeve and place the filter back onto the tool, keeping the same alignment as noted in step #3.
5. Take a Left new sleeve, and place the filter (already oriented on the tool) into the large end of the sleeve. Color stripe on filter faces the back of your head (Fig. C)
6. Push the filter into the sleeve until flush with the opposite end of the sleeve, and **NOT STICKING OUT!**
7. Repeat all instructions for other side.



Fig. A

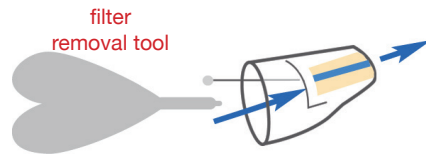


Fig. B

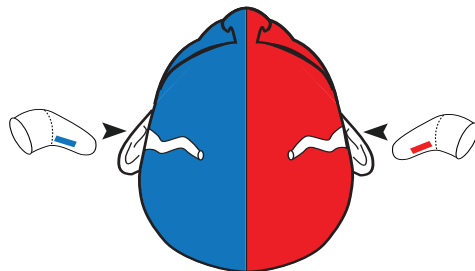


Fig. C