

YOUR HEARING **CLEARLY** MATTERS™




HEARING PROTECTION THAT
FOCUSES ON HEARING™



- World's most comfortable earplug
- Smart Seal™ ensures a perfect fit
- Wear comfortably for hours
- No plugged up feeling
- Replaces custom earplugs
- Will not sweat, tickle or itch
- Use your cell phone while wearing
- Helps prevent ringing in the ears
- Replaceable sleeves save money!

TYPICAL 95dB SPL MUSIC CONCERT
(dB SPL at Eardrum)

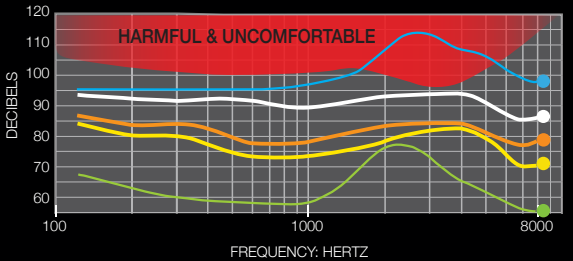

- NATURAL EAR RESONANCE
- EARASERS STANDARD | -19dB Peak
- EARASERS EURO | -26dB Peak
- EARASERS MAX | -31dB Peak
- FOAM EARPLUG

Made In The USA

170 N. Cypress Way
Casselberry, FL USA 32707
info@earasers.com
tel. 1 (407) 339-2422

US Patent #8,800,712
International Patented
ib_earm_1_23
All rights reserved.

PERSONA
MEDICAL



EARASERS™
BE EAR-RESPONSIBLE



CLEARLY **HEARING** MATTERS™



Earasers don't plug up your ears and muffle sound like conventional earplugs.



Inserting Easersers:

1. Make sure your ears and Easersers are clean.
2. Blue is for the Left ear and red is for the Right. The filter inside has a colored indicator (Fig. A).
3. Hold string with index finger and thumb. (Fig. B). Raise earplug up so the line guide is parallel to the floor at eye level. Bring the earplug straight back and guide into your ear canal. (Fig. C)
4. Let go of the string. With your pointer finger, gently push into your ear canal while wiggling your finger in an up/down motion. Push on the silicone until you feel a firm seal. **Talk out loud using a strong voice. Your own voice must sound louder or muffled to you, especially the low frequencies.** If it feels loose or tight or there is little difference in the sound of your voice, contact us and we can help solve the fit issue. **A deep seal will offer the best results.**

Tip: Moistening the ear tip may ease insertion. Opening and closing your jaw may also help.


Removing Easersers from ear:

Simply hold the string with your thumb and index finger and pull outward.





LEFT: BLUE

RED: RIGHT

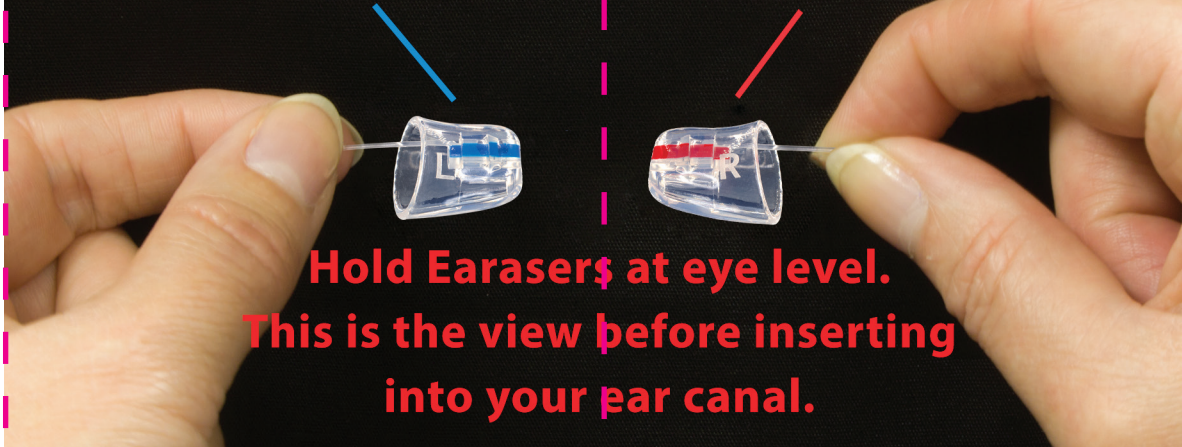

**Hold Easersers at eye level.
This is the view before inserting
into your ear canal.**

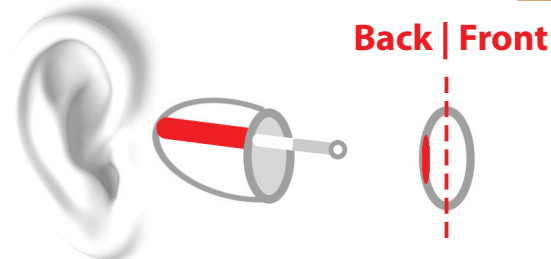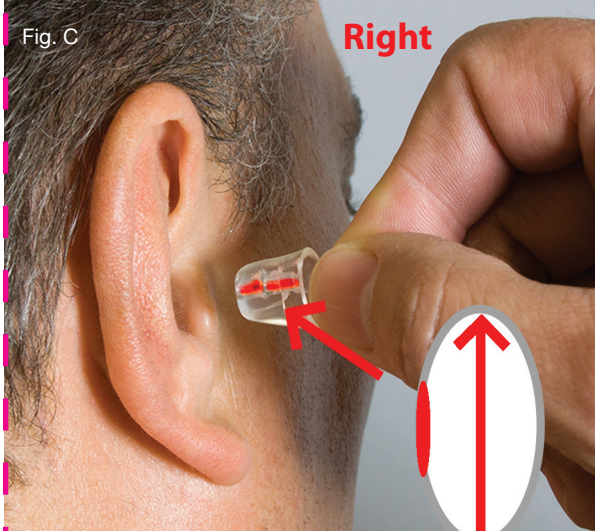

Side of Easersers with color line always faces towards the back of your head. The ear canal is shaped like an oval, standing upright and vertical. Easersers are also shaped like an oval. Insert the Easersers so the long axis remains upright.

Cleaning and Maintenance:

We recommend regular cleaning of your Easersers earplugs to ensure they work as effectively as possible. To remove ear wax from the earplug, use a soft bristle brush such as a toothbrush. Cleaning tools, wax guards and germicidal cleaning wipes are available on our website or at participating stores. A soft, damp cloth may also be used to clean around the outside of your earplugs.

Do not use any chemicals as these can have a detrimental effect on the product. Do not put any items into filters as this may cause damage.

WARNINGS

- Keep out of reach of young children or pets. Product and packaging contain small parts, which can be swallowed.
- If intended use is for any OSHA noise abatement program or Industrial, Military, or commercial exposure, close coordination with a company approved safety officer should be maintained to ensure compatibility with your environment.
- If you experience ANY ringing or muffled feeling in your ears after use, please choose an Easersers earplug with a higher reduction rating.
- Get annual hearing checkups from an Audiologist or Hearing Specialist if you use earplugs.
- Not to be used with any firearms.