

CALIBRE GRAB RAIL WITH TOWEL

Mod 32mm Diameter Rail with Towel Holder

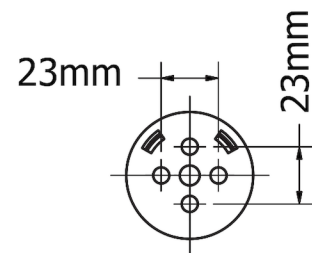


STOCK CODES


32mm Mod 300mm - NRRC3212B
32mm Mod 600mm - NRRC3224B
32mm Mod 900mm - NRRC3230B




TOP



WALL BRACKET

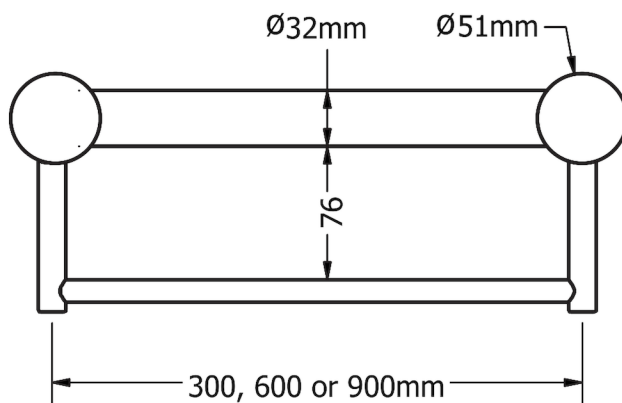

FRONT

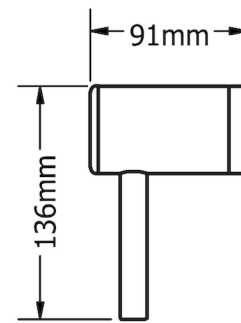

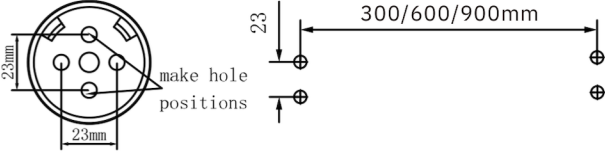
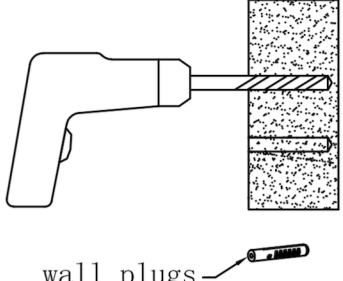
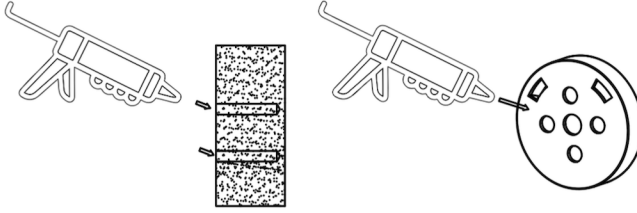
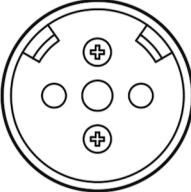
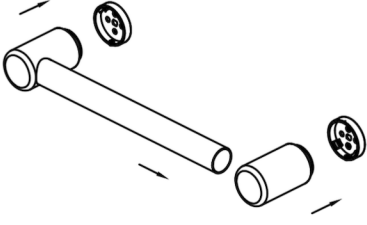
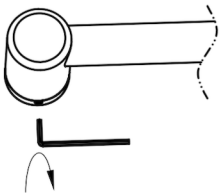

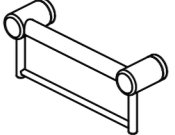
SIDE

AVAIL QUALITY DESIGNS

- Australian Owned and Designed
- Sizes: 300 - 1200mm Lengths as standard, longer available by request
- Available in over 7 finishes including Living Brass, Chrome, Matt Black, Gunmetal, Brushed Nickel, Bronze, Gold
- Modular design, the rail to be mounted horizontally, vertically or at any angle and cut down to suit your bathroom
- 150 kg capacity for all 25mm and 32mm diameter calibre grab rails
- Mod 32mm diameter rails are approved for standard AS1428.1 requirements in DDA bathrooms
- Ergo 25mm diameter rails are suitable for heavy duty towel rails and smaller hands, in non-DDA bathrooms



Installation Instruction

	<p>1. Locate wall studs, install into at least one stud. Place wall seal/mounting bracket on wall at correct height over centre of stud and mark hole positions on wall (minimum 2 supplied screws per bracket).</p>
	<p>2. Drill pilot holes into wall stud approximately 60mm deep to suit mounting screws (and wall plugs if required).</p> <p>Tip: 3mm hole for screws into timber or 8mm hole for wall plugs.</p>
	<p>3. Apply small amount of silicon to mounting bracket and wall in and around the screw holes.</p>
	<p>4. Install mounting brackets by screwing onto wall.</p> <p>Tip: Use appropriate mounting screws for the wall stud material, supplied screws are for timber wall studs.</p>
	<p>5. Slide rail tube into end bracket and push onto wall brackets.</p> <p>Tip: Rail length can be cut down to a smaller size, simply use an angle grinder, hacksaw or pipe cutter then deburr the end with a file or grinder and reinsert into the end bracket.</p>
	<p>6. Tighten grub screws with allen key.</p> <p>Tip: If rail rattles, loosen end bracket, twist slightly and tighten.</p>
	<p>7. Using supplied allen key whilst holding the rail in place, screw the M4*90mm and M4*45mm into the grab rail till tight (according to left picture).</p>
	<p>8. Finished.</p>