

General information

- The HaloPocket is designed for personal use. The salt particles are ground up to 0.3-0.5 microns
- Do not clean your HaloPocket with any liquids. The external part may be wiped clean with a damp cloth.
- Only Pharmaceutical grade salt to be used
- Your HaloPocket comes with a rechargeable Lithium Iron Battery

Warranty – Full labour and parts warranty for 2 years from purchase date.

If you are not 100% satisfied with your product, we offer a full 30 day refund. Conditions apply. Damaged items will not be accepted.

haloPocket

Your haloPocket comes charged, sanitised and ready to use!



Congratulations on purchasing the HaloPocket and taking a positive step towards respiratory health and hygiene.

This package includes:


- 1 No. HaloPocket personal Halogenerator
- Pharmaceutical grade salt
- Measuring spoon
- USB charger
- Cleaning brush.

If you have any issues, please contact us. We're here to help!

Getting to know your haloPocket

Getting started and using your haloPocket

1. Open the cap and add ¼ measuring spoon of salt to the mill (up to the blades)
2. Push cap back in place
3. Press on button for 10 seconds (the mill will activate and grind the salt)
4. Remove the cork
5. Now hold the top of the haloPocket 2-3 inches away from your mouth or nose.

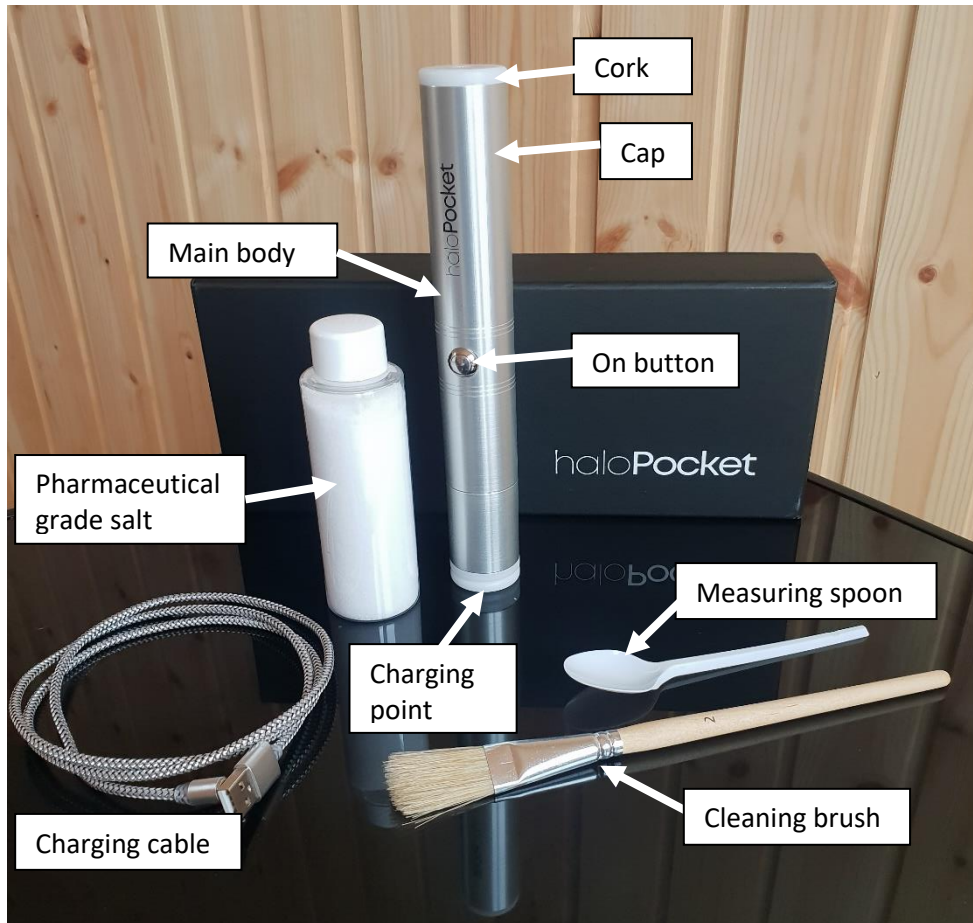

Keep the haloPocket slightly tilted and when breathing in, press the on button. When breathing out, release the button. You will hear the mill start and stop and should see a fine salty mist coming from the top of the haloPocket.

Continue to do this for 3-5 minutes, inhaling deeply through your mouth and nose, up to 2 times a day.



Looking after your haloPocket

Once you have completed your session, if humidity below 50%, close the cap with the cork and store your haloPocket ready for its next use.

If humidity is above 50% then remove the salt from the cap and clean out any remaining salt / particles with the brush.

Ensure your haloPocket does not come in to contact with water.

Periodic cleaning of the blades should be carried out with the supplied brush.

Do not use your haloPocket near computers or electronics.

Charging

Connect charging cable to USB charger and magnetic connector to the base of the haloPocket. When the charging cable is connected, the haloPocket is not operational. A full charge will take approximately 90 minutes and should last 2-3 days. Only the supplied charging cable should be used.