

Thanks for purchasing Lupin! We have also included straps in your delivery that will keep Lupin secure when it is folded and being transported.

Please see the instructions below on how to use it.

If you need any further assistance please call us on 0800 999 6606 and one of our customer service operators will be more than happy to help.



1. Feed the strap through the handle at the top of the backrest and through the bar across the footrest. Make sure the grippy side of the strap is on the inside and against the scooter.



2. Loop the end through the hook and fold it back on itself, securing it to the strap.

Head to our YouTube to see the demonstration on how to use the strap to secure your Lupin - <https://www.youtube.com/watch?v=YN4crQeezvg&t=2s>

Kind Regards,

Betty & Bertie Team

Registered Office: Unit G, The Courtyard Business Centre, Lonesome Lane, Reigate, Surrey, RH27QT.
Registered in England No. 5495581. VAT Reg. No. 862 3170 36.