

# LUPIN QUICKSTART GUIDE



# **WELCOME - LET'S GET STARTED**

Welcome to our Quickstart Guide for setting up your Lupin Mobility Scooter. Here we have instructions on how to open and fold your scooter, how to use the different features and how to charge the battery!

The battery for your scooter should be partially charged and this will be enough for you to give it a quick test before needing to charge it again fully!

Please ensure there are two adults lifting the scooter out of the box and remember to flat pack the box and store away.





### Step 1

When you've taken your scooter out of the box, it should look similar to this position.

### Step 2

Lift up the handlebar stem and push into place. You will need to push the clip down to secure this into place.


### Step 3

Unfold the seat and lift up the clatch found on the back of the seat and lift up the seat. When the seat is at its maximum height you can let go of the clatch and the seat will lock in place.

## <u>ADJUSTING THE FRONT DRAW BAR</u>

- 1. Bring scooter up onto its back wheels
- 2. Locate the locking pin
- 3. Hold the pin out and pull out the metal bar
- 4. Release the locking pin and bring out the front draw bar
- 5. The bar will click into place with the 2nd locking pin **OR**

You can locate this locking pin and pull it out to click in the bar manually

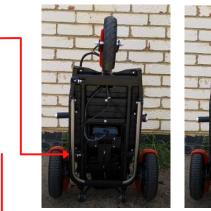



# **ADJUSTING THE REAR DRAW BAR**

Always extend the rear safety bar when driving the scooter. This provides extra stability and prevents tipping of the scooter when reversing or braking on an incline.

### Stage 1

Pull out locater pin

### Stage 2

Extend safety bar to be the first or second hole and allow the locater pin to locate in the desired hole.





### **OPERATING INSTRUCTIONS – GETTING STARTED**

### **ON/OFF - POWER SWITCH**

- 1. To power-on your scooter simply press the on button on the control panel.
- 2. Hold down the button to turn off the scooter

### BATTERY INDICATOR

When the scooter is on the battery light will display up to 4 red lights. The number of lights will reduce as the battery level decreases.

# (NOD) COURT

### **SPEED SETTINGS**

- The MODE button starts with the default gear at 0 (no gear indicator light).
- The corresponding indicator light will be on when the 1st (LOW), 2nd (MED) and 3rd (HIGH) gear is on.
- When pressing the MODE button, the gear shifts in the following pattern: 0-1-2-3-2-1-0.
- The speed range of the 'Slow' setting is from 0.2 6.0 km/h.



The slow speed setting is the only setting that can legally be used when riding you scooter in public. If you plan on using the faster speed settings, this can only be done on private property.

### FORWARD/REVERSE

Set the directional switch to:

D: Forward drive

R: Reverse

NOTICE: Lupin does not emit a warning sound when reversing.



### HORN BUTTON

We advise that you only use it when needed to prevent an accident or injury.

NOTICE: The Horn does not disable when turned off. Ensure nothing is touching the horn when folded.



### **BATTERY HOLDER**

- The built-in 36V 8AH lithium-ion battery is to supply strong power for the running of the scooter.
- It rests in a tray with a 'track' down the middle, that slots into the battery.
- Make sure these are aligned when inserting the battery, or you will be unable to lock the battery in place, and the connection between the battery and scooter may be inconsistent, leading to power failures.



## SETTING THE BRAKE

When setting the handbrake lock, you may need to squeeze the brake lever tighter to apply to the lock.

**Handbrake Lock** 

**Handbrake Lever** 

TIP: You may find it easier to squeeze with one hand and use the other hand to set the lock



