



# HOW TO:

## ASSEMBLE THE MOBILITY SCOOTER FRONT BASKET



### **STEP 1**

Locate the clamp fastening at the back of the basket. Unfold the clamp and start unscrewing this to allow the fastening to open.



### **STEP 2**

When the fastening is open, you can remove the screw and open the fastener out.

**STEP 3**  
Align the basket to the front of the frame and close the fastener around the pole. Make sure you hold this firmly to ensure it doesn't move while tightening.



### **STEP 4**

Align the screw with the thread in the hole and tighten this up securely with your hand. You do not need any tools for this part.

### **STEP 5**

Close the clamp when fully tightened and your basket will now be fully fitted on your mobility scooter.



### **STEP 6**

To open, unclip the clasp at the bottom of the basket to allow the basket to open fully.



### **STEP 7**

Extend the basket outwards to its fully opened position. You will need to then put the bottom of the basket down to allow it to stay open.



### **STEP 8**

When you hear the bottom of the basket click into place, the basket is now fully extended and ready to use.

### STEP 9

To fold away, locate the clip underneath the basket and fold the sides inwards which will allow the basket to fold in on itself naturally.



### STEP 10

When folded in on itself, the basket will click back into its locked position. When folded the basket will not interfere with the folding process.

Upgrade your mobility scooter experience with our innovative Front Basket, meticulously designed to enhance your convenience and make your journeys smoother than ever. Engineered to fit most mobility scooters, including our beloved Zinnia model, this basket effortlessly combines functionality and ease of use.



If you would like to **watch a video** on how to assemble the basket then please enter the following URL in your web browser:

<https://youtu.be/9HhTVPM5PcE>

