



BETTY &
BERTIE
MOBILITY

ZINNIA X

QUICKSTART GUIDE



OPERATING INSTRUCTIONS – GETTING STARTED

BEFORE OPERATING


1. Make sure the battery is fully charged.
2. Make sure you are familiar with the conditions of your planned route, including crowds, animals and potential obstacles.
3. Remember to make sure to keep away from uneven or slippery terrain.
4. Check to see if the key is fully inserted, and the brake lever switch is in the correct position (ON if folding / unfolding the scooter manually)
5. Remember to steer the scooter with both hands on the handlebar
6. Sound the horn to check it is working.

DRIVING

1. Once the key has been inserted and the power switch turned to ON, unfold the scooter fully.
2. Check to make sure the seat is in the correct position and adjust the height of the steering column to a comfortable position.
3. Adjust the speed dial to the desired speed. (The speed dial sets the maximum speed the scooter will move at. The speed from standing still up to that maximum speed is controlled by increasing the pressure on the Control Lever).
4. Turn the key switch to ON, and place hands on the handlebar
5. Gently push the control lever with the thumb of your right hand
6. The brake will release automatically, and the scooter will move forward. Do not push the control lever too far or suddenly, as the speed will rapidly increase (up to the maximum speed set by you using the speed dial)
7. Turn the scooter right or left by turning the handlebars right or left accordingly.
8. The brake will be applied automatically when the directional control lever is in the centre position (or released) and the scooter will stop automatically.

AFTER OPERATING

1. Make sure the scooter has come to a complete stop.
2. Turn off the power by turning the key switch to OFF
3. Carefully get off the scooter
4. Fold the scooter using one of the 2 choices for folding (by remote control or electric switch).
5. Then turn the power switch to OFF and remove the key.

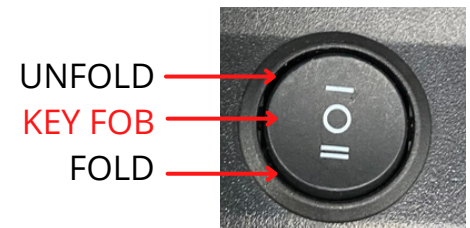

NOTE: When folding or unfolding, the scooter may tilt backwards, and the front wheels may rise off the floor. This is normal, and the scooter won't topple over – simply push the front wheels down to restore the normal position.

UNFOLDING AND FOLDING

The scooter is designed to fold and unfold in two separate ways, which may be more or less suitable depending on the circumstances. If your scooter is fitted with the optional armrests, on refolding the scooter using the remote, the front wheels may lift off the ground due to the additional weight. To rectify this, push down on the front wheels once they are a few inches off the ground.

The ways of folding and unfolding your scooter are:

1. Automatic
2. Electric Switch

ELECTRIC SWITCH

This mode can be used if the remote key fob has been misplaced, is malfunctioning, or is out of battery. Use the switch located below the seat of the scooter, in front of the batteries. Make sure the key is inserted and the power switch turned to ON before using the electric switch.