



## Setting Up and Using a Lancing Device


- 1 Unscrew the cap.

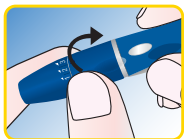


- 2 Insert a **new** lancet into the lancing device.

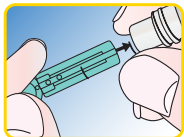


- 3 Twist off the cap on the lancet and save it for later use. Replace the lancing device cap.

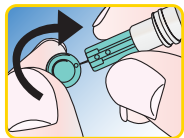


- 4 Set the lancing level.  
Level 5: deepest penetration  
Level 1: least penetration

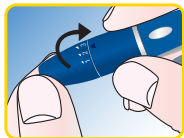

- 5 Cock the lancing device.

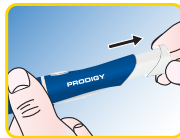

- 6 Press the release button to puncture the skin.

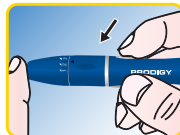

- 7 Unscrew the lancing device cap then replace the lancet cap onto the lancet.



- 8 Slide the eject button to eject the lancet. Discard the lancet according to local safety regulations.  
**\*Do not reuse lancets.**



### WARNING:

- Never share a lancet or lancing device.
- Always use a new, sterile lancet.
- Lancets are for single use only.

**PRODIGY**  
**VOICE**<sup>®</sup>  
Blood Glucose Monitoring System



Quick Reference Guide


Thank you for choosing the Prodigy Voice® blood glucose monitoring system. This quick reference offers easy instructions on how to use your Prodigy Voice® meter effectively. A Voice meter audio manual CD is also included in the meter box and an instructional video is available at [www.prodigymeter.com](http://www.prodigymeter.com).

## Blood Glucose Testing

Wash your hands and the intended sample site before and after using the system or obtaining blood samples. Use only fresh capillary blood samples for testing. Always use a new sterile lancet and keep the lancing device clean.


## Applying Blood Correctly

With the test strip inserted, bring the meter close to your finger or other test site and place blood beside the blood drop. The blood will automatically be drawn into the absorbent channel once the test strip is close enough to the blood drop. Do NOT smear blood onto the test strip or place the blood on top of the test strip.

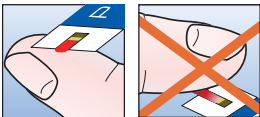

## Confirmation Window

If the absorbent channel is not full, add more blood. You will hear a confirmation beep once the channel is full.

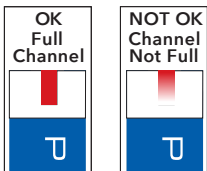

## Performing a Blood Glucose Test

### 1 Insert Test Strip

Insert test strip, the meter powers on automatically.

No Coding required.



*"Thank you for using Prodigy Voice.*


*Today's date...*

*The time...*

*Please apply blood to the test strip."*



### 2 Apply the Blood Sample

"" appears, apply blood.

The meter **beeps**.



*"Now Testing."*

The meter counts down from 6 to 1.



### 3 Hear and See Results

Hear and see accurate results.



*"Your blood glucose 103 milligrams per deciliter."*



## Recalling Individual Test Results

Your meter stores 450 of the most recent blood glucose test results with date and time in its memory. With the meter powered off, press the "M" button to turn the meter on. Press the "M" button again to enter memory mode. Your most recent test result will appear with date and time. Use the Up and Down buttons to navigate through all individual test results.



## Recalling Test Result Averages

With the meter powered off, press the "M" button to turn the meter on. Press and hold the "M" button for 3 seconds to enter Memory Average Mode. Your 7 Day Average will appear. Use the Up and Down buttons to scroll through the 7, 14, 21, 30, 60 and 90 day averages.