

LUXEMED

TAKING CARE OF YOUR FURNITURE



CLEANING AND DISINFECTION

Before cleaning the chair/table, unplug it from the mains (if applicable).

Clean the chair/table after each use.

To clean the upholstery, apply a solution of warm water and a mild detergent (for example, neutral soap) with a well wrung soft cloth. Wipe over once cleaned.

Disinfect upholstery and other surfaces with a suitable bactericide for this purpose.

Apply the disinfectant on a soft cloth, never directly on the chair/table.

Never use a mixture of disinfectants or a mixture of disinfectant and soap.

Wipe the upholstery over once cleaned & disinfected with a cloth and water only. This will remove any residue from the upholstery.

Avoid the accumulation of moisture on the parts of the metal structure, as they could rust.

PLEASE READ

Since we do not know which disinfectant will be used, we cannot guarantee that it will not cause damage to the upholstery. Read the instructions and warnings for use of the disinfectant product carefully before its purchase or use.

IMPORTANT INFORMATION:

ALWAYS TEST prior to use by applying a small amount of the disinfectant to an inconspicuous area, if available, and then leave for 30 minutes.

DO NOT USE

- hot or boiling water
- hydrocarbons including alcohol
- acetone
- furniture polishes
- bleach
- concentrated detergents
- abrasive powder or paste cleaners
- steel wool
- silicon or petroleum-based cleaners
- solvent based or abrasive cleaning agents

MAINTENANCE AND WARRANTY

Carry out a visual inspection at least once a month to check the general conditions of the chair/table. Moreover, you must do the following:

- Check the correct adjustment of the different sections
- Check the proper functioning of the motors and their movements
- Check and clean the gas struts
- Check and tighten screws (if necessary)
- Grease the parts with friction
- Check the conditions of the electrical wiring and the socket
- Check the wheel immobilization system

LuxeMED will repair or replace parts found upon examination to be defective in material and/or workmanship

LuxeMED will not warrant the safety and or correct functioning of products where any original components have been changed or modified. Furthermore, safety is not guaranteed where components have been replaced with non-original approved parts. If any faults are detected upon receipt of this product please phone LuxeMED. Any faults that are detected during normal use should be reported to LuxeMED immediately to determine if warranty conditions apply and if so, the necessary repair or replacement work to be completed.


IMPORTANT INFORMATION:

Improper and/or incorrect usage of your product will void product warranty

CONTACT US

 www.luxemed.com.au

 info@luxemed.com.au

 1800 954 331