

Changing Bubble Tube Water

All bubble tubes require that the water in them be changed every 6-12 weeks to slow the build up of any bacteria. To change the water in an Experia LED Bubble Tube:

1. Remove the lid off the column.
2. Use a length of hose (4 yards minimum for a 80" column) to siphon the water out of the column into a bucket. Empty the bucket and repeat.

Note: We recommend wrapping a towel around the base of the clear column before removing the water. In the event of a spill, this offers a little extra protection against water entering the base.

Troubleshooting

Q. The unit doesn't work.

A1. Is it switched on? If so,

A2. Has the unit scrolled through its colors? If so,

A3. Have you pressed a button on the platform? If so,

A4. Please contact us for further assistance.

Contact Details

For USA please email support@experia-usa.com

For UK and rest of the world email support@experia-innovations.co.uk

Telephone: (800) 882-4045

Please note: email is our support team's preferred communication and this takes precedence over telephone calls.

Proudly
factured

experia **USA**
multisensory for everyone

manu-
in the

U.S.A. by



WARNING: CHOKING HAZARD


Small Parts: Not suitable for children under 3 years.

©

Copyright 2010-2013
All rights reserved.
Experia is a registered



Experia
trademark.

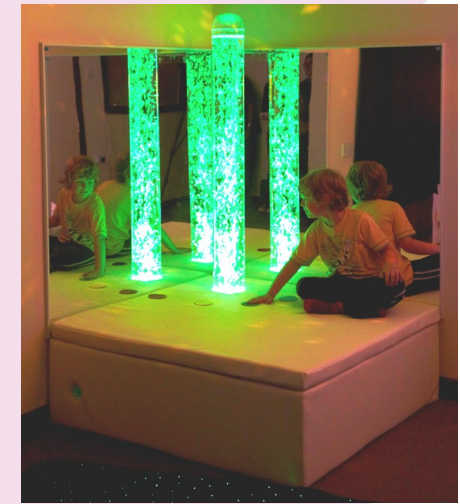

Interactive LED Bubble Tube Corner

480093

Product Manual

For optimal results, please read and follow instructions carefully.

Please keep for future reference.

Experia USA, Inc.
2880 McEver Road
Bldg. 1 - D
Buford, GA 30518
(800) 882-4045
info@experia-usa.com

experia
multisensory for everyone

Features

- Ergonomic design
- Latest LED technology.
No changing bulbs.....EVER!
- Clear domed Lid
- Domed base
- Low voltage transformer – supplied
- Easy maintenance
- Durable & easy to use
- 1 Year Manufacturers Guarantee

Pack contents

- 1 - Interactive LED Bubble Tube—60” with domed lid
- 1 - Interactive LED Bubble Tube Platform with built in switches
- 1 - Low voltage transformer
- 1 - 36” W x 48” H Acrylic Mirror Pair
- 1 - Bubble Tube Bracket
- 1 - BCB Bubble Tube Water Treatment Fluid

Please check with your maintenance personnel to determine the proper fasteners and anchors for attaching the mirrors to your wall. Any and all installations must be carried out by a competent professional.

Mounting the Mirrors

Please refer to the Acrylic Mirror instructions.

Fitting the Platform and Bubble Tube

1. Refer to the Bubble Tube Platform Product Manual for complete instructions on the installation of the platform.
2. Start with the platform near the corner where it will be installed. Lay the bubble tube platform on its side. You will notice two connection points on the base of the bubble tube. One is for the transformer (larger bottom connector), and the other is for the control box (smaller top connector).
3. You will need to START putting the bubble tube up and through the platform. Once you have started doing this.....

Tools required

- Drill
- 7mm Drill Bit
- Screwdriver
- Level
- Pencil
- 2 people

4. Connect the cable from the transformer to the socket on the base of the tube.
5. You will notice a black box on the underside of the bubble tube platform. Connect the cable from this to the control box socket on the base of the tube.
6. With one person holding the tube and the other holding the platform, slowly and carefully turn the bubble tube and the platform so that are both upright.
7. Move both units carefully to the final location. Once in place, try moving the bubble tube to ensure it is sitting flat to the ground. Using a level, ensure that the bubble tube is vertical.
8. Run the electrical cord out from under the platform in the corner. (Note: In the far corner of the platform there is a soft part of the underside for this cable to run where it will not be pinched. Refer to your platform manual.)
9. Install the bubble tube bracket approximately 3” from the top of the tube.
10. Now fill the bubble tube with water. **(We recommend only using distilled water)**. If you spill any water, please ensure this is cleaned immediately. We recommend wrapping a towel around the tube when filling. Do not fill the tube to the top; stop filling when 3” from the top of the tube. Place the domed cap on top of the tube.
11. Plug the power cord into a nearby outlet. The bubble tube will now illuminate and begin to scroll through its many colors.

Please note: Use only the transformer supplied. Use of a different transformer may result in damage to the product and void the warranty.

Controlling the Colors

1. Simply press either the red, yellow, green or blue circle on the soft platform to change the tube to that color. Pressing the black circle will stop and start the bubbles.
2. If no switch is activated for 5 minutes the unit will revert to passive mode and the tube will begin changing colors automatically.

Cleaning and Maintenance

- The unit should be cleaned with a damp soft cloth. A non abrasive cleaning agent may be used for stubborn stains. Be sure no liquid enters the base of the unit.
- The unit has been designed for use by people with special needs, however this does NOT mean it is unbreakable. You must be sure the unit is not hit with anything and that no one pulls or pushes the unit.

