

# VARYFLEX<sup>®</sup>

## THE SECRET TO ENDODONTIC SUCCESS

Precision Built. Created in the UK.

[ENDOPERFECTION.COM](http://ENDOPERFECTION.COM)

# VARYFLEX<sup>®</sup> TAPER FILES

put you firmly in control of  
endodontic treatment

## EVIDENCE-BASED DESIGN


brings genuine value into  
endodontics for you and  
your patients.

## A GENTLE SYSTEM

that can work in any dentist's  
hands for predictable,  
reproducible results.

## SIMPLY DOES IT

by taking each file to the  
working length and the ideal  
shape is created.

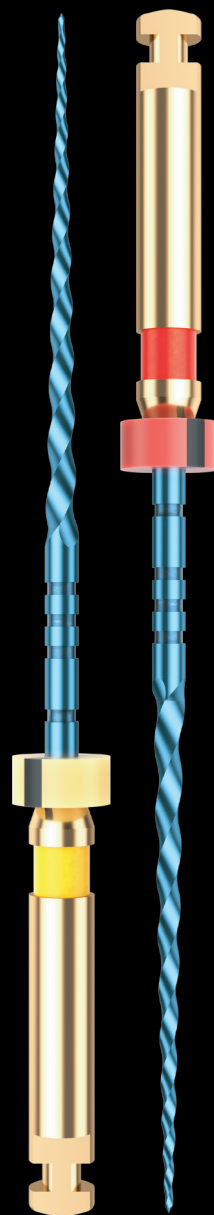

## TACTILE CONTROL

due to the trademark 11mm  
shank, making treatment  
much easier to carry out.

## ACHIEVE GREAT RESULTS








in both simple and complex  
cases with confidence.

## A FRACTION OF THE COST

of other multi-file systems  
yet there is absolutely no  
reduction in quality.



Our own UK designed file system with a state-of-the-art, flexible alloy. You will love the feel of these beautifully made files.

		PACK SIZE	FILE LENGTH
<b>S1</b> 0.18/0.02		x6	25mm
		x6	31mm
<b>S2</b> 0.20/0.04		x6	25mm
		x6	31mm
<b>F1</b> 0.20/0.07		x6	25mm
		x6	31mm
<b>F2</b> 0.25/0.08		x6	25mm
		x6	31mm
<b>F3</b> 0.30/0.09		x6	25mm
		x6	31mm
<b>F4</b> 0.40/0.06		x6	25mm
		x6	31mm
<b>F5</b> 0.50/0.05		x6	25mm
		x6	31mm


# STEP BY STEP GUIDE

Create a guide path by taking a size 15 hand file or our guide path rotary file, to the WL. Run in an appropriate speed and torque controlled motor at 250 - 300 rpm and 3Ncm.


Take S1 to working length.





Take S2 to working length.

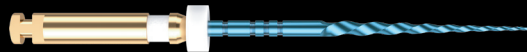

Take F1 'pecking' to working length.  
Gauge with 20 hand file. If constriction larger than size 20, take F2 to length and then F3, F4 or F5 if needed.

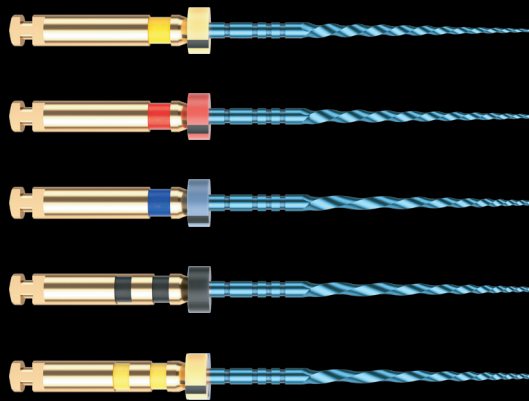

## Matching Gutta Percha & Paper Points Available

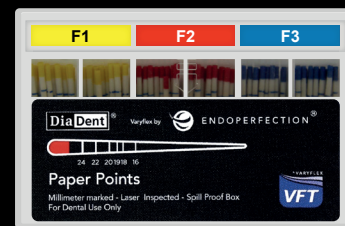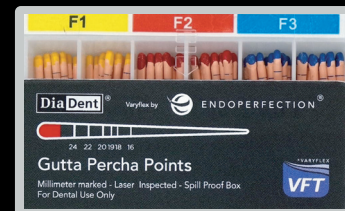

[ENDOPERFECTION.COM](http://ENDOPERFECTION.COM)