



VARYFLEX®

THE SECRET TO ENDODONTIC SUCCESS

Precision Built. Created in the UK.

ENDOPERFECTION.COM

VARYFLEX[®]

RECIPROCATING FILES

simplifying endodontic treatment
with just one file!

UK DESIGNED

range of endodontic files that offers amazing value without compromising on quality.

ONE FILE

does it all, creating the appropriate shape to the working length.

INCREDIBLE FLEXIBILITY

enabling negotiation of canals at the same time as being rigid enough to flare canals predictably.



SINGLE-USE STERILE FILES

at a fraction of the price of other blister packs.

COMFORTABLE AND EASY TO USE

as a result of the heat-treated nickel titanium and 11mm shank.

SIMPLE CONVERSION

for use in the same way and same motor settings as other reciprocating files.




The same great alloy as our VaryFlex Taper files. Available in two sizes and with our trademark 11mm shank. You choose which files work the best for you and your patients.

		PACK SIZE	FILE LENGTH
25 25/08		x6	25mm
		x6	31mm
40 40/06		x6	25mm
		x6	31mm

STEP BY STEP GUIDE

Create a guide path by taking a size 15 hand file or our guide path rotary file, to the WL. Using a dedicated endo motor, run the VFR files at 150-180° CCW and 30-50° CW and run at 300 RPM.

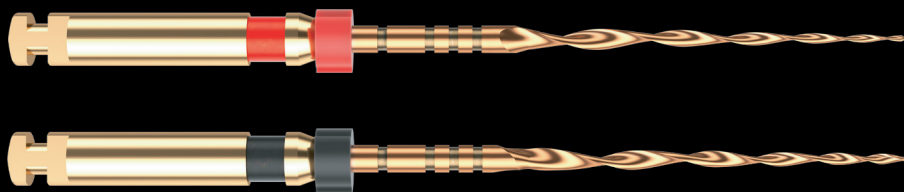

Using a combination of pecking and brushing, slowly advance the file apically. If the WL is not reached after 4-5 "pecks", remove the file, irrigate, check patency and then re-introduce the file. Repeat until the WL is reached.

Matching Gutta Percha & Paper Points Available



ENDOPERFECTION.COM