Lansinoh.

EN

POST BIRTH WASH BOTTLE

For gentle washing of the perineal area to soothe post-birth discomfort.

How to use:

- Fill bottle with warm water. Screw cap on securely and extend nozzle firmly.

- While on the toilet, invert bottle between legs aim the tip towards the perineal area.

- Squeeze bottle gently and repeat as needed.

Cleaning instructions

- After each use, disassemble and clean with mild soap and warm water only (do not boil).

- Leave to dry before storing.

Distributed by: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | lansinoh.co.uk

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Distribuit de: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.ro

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Jakelija: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.fi

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Distribueret af: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinohdanmark.com

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Distribuerad av: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinohdanmark.com

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Dreifit af: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK

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Дистрибьютор: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.ru

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Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinohme.com

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Distribútor: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.sk

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Distributor: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.cz

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Разпространител: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.bg

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• Deixar secar antes de guardar

Distribuído por: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.pt

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• Ostavite da se osuši prije spremanja

Distribuirai: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK | www.lansinoh.hr

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• Enne hoiule panemist laske kuivada.

Turustaja: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK

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Platintojas: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK

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• Pred shranjevanjem naj se steklenička osuši.

Dobavljai: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK

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• La tørke før lagring

Distribuert av: Lansinoh Laboratories UK Ltd, Ground Floor, West Mayesbrook House, Lawnswood Business Park, Leeds, LS16 6QY, UK

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Lansinoh® is a registered trademark of Lansinoh Laboratories, Inc. © 2021 Lansinoh Laboratories, All Rights Reserved

PE68420IF1220