

 **CVS**Health.

3-IN-1

# COMFORT COMMUNE

by

**MICHAEL  
GRAVES  
DESIGN**

superior design  
+ function for the  
best daily care

**INSTRUCTION MANUAL**



## CVS Health® 3-in-1 Comfort Commode



by

**MICHAEL  
GRAVES  
DESIGN**

At CVS Health®, we are passionate advocates for transforming health care into a more personalized experience through new technologies and innovations that can help improve the quality of your everyday life. Architect and designer Michael Graves shared these values and our partnership with Michael Graves Design brings you a collection of thoughtfully created adaptive products that deliver superior comfort, style, and functionality. It is through partnerships like this and many others that we continue to bring you innovations in care.

### **ERGONOMIC**

- Innovative, wider seat design improves user experience
- Swing away armrests improve transfers and user access


### **CUSTOMIZABLE**

- Refined furniture look - great for bathroom or bedroom
- Removable backrest improves use over toilet
- Height adjustable from 16 to 21 inches

### **SIMPLE**

- Works with most round and elongated toilets
- Tool free assembly

## Parts for your new 3-in-1 Comfort Commode

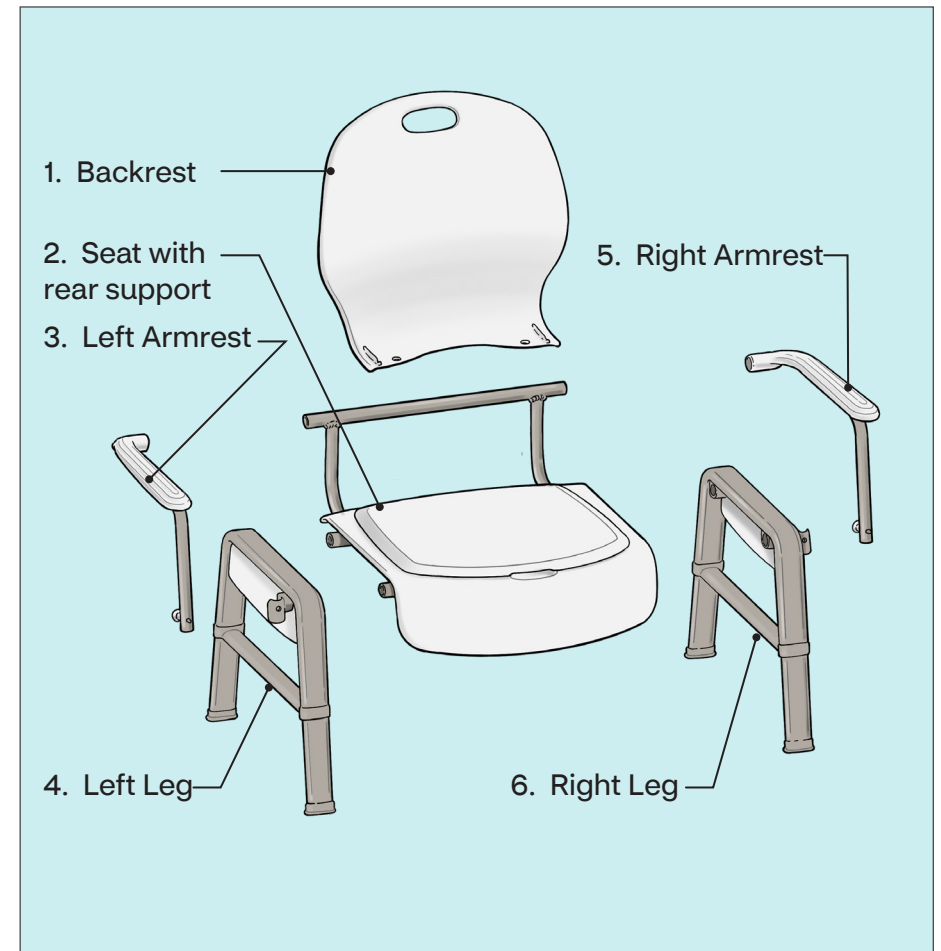


## Accessories and Hardware




## How to Assemble your new Comfort Commode


1. Lay the Seat on a covered flat surface like a table or counter with the seat top facing down. Line the back edge up with the edge of your surface.

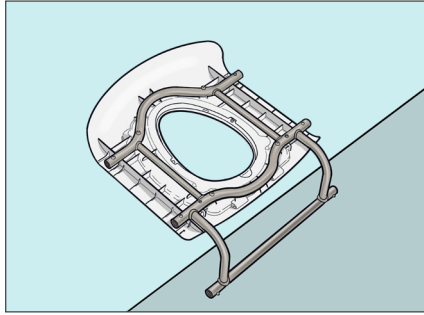


2. The Right Leg attaches to the tubes on the underside of the Seat. The Legs height adjustment holes face the back of the seat. Press the spring pins on the Seats tubes and slide the Leg on. **The pins will click in place.**

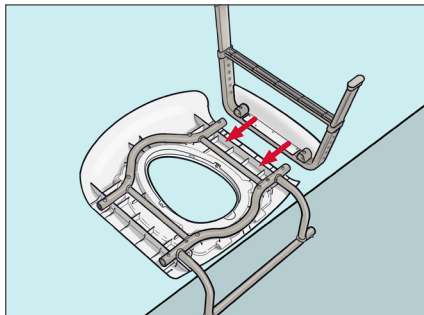


3. Repeat step 2 with the left leg. **The spring pins should click securely into the holes on the legs.**

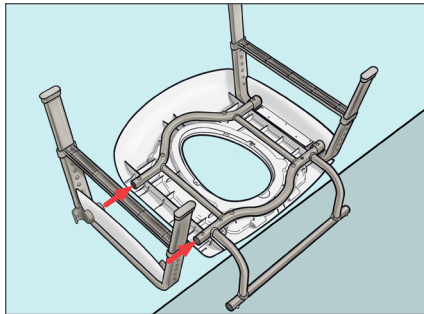


4. Double check to ensure the legs are on the correct side. The height adjustment holes face the back of the chair. The Armrests will not attach properly if the legs are on the wrong side.

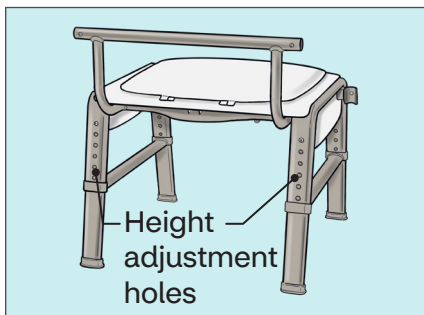


5. Turn chair upright and place it on a flat surface or the ground.

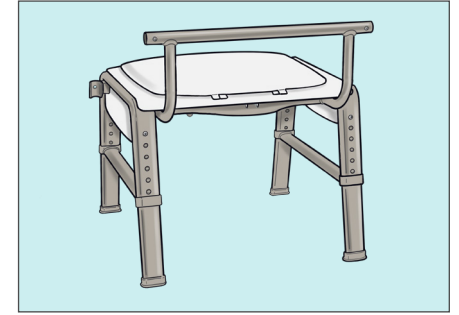


6. To attach Armrests, press the spring pin down on the rear support, then slide the Armrest on. The Armrest should be angled downward, so **the spring pin locks into the armrest with an audible click.**

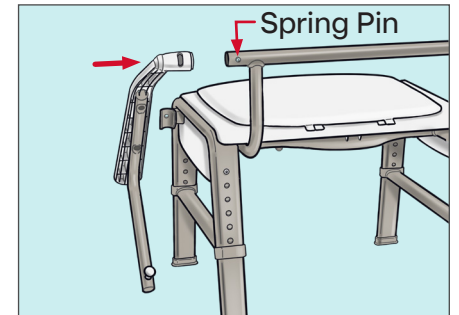

7. Pull the armrest up from the front until it locks into place. **The spring-loaded locking pin will secure itself in the bracket on the side of the commode with an audible click.**

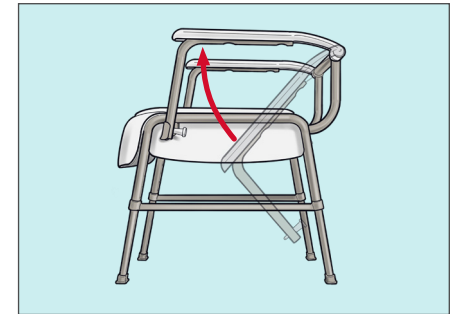


8. Lift the lid and seat. If you will be placing the commode over a toilet, insert the splash guard. If you will be using the commode away from a toilet, insert the pail.

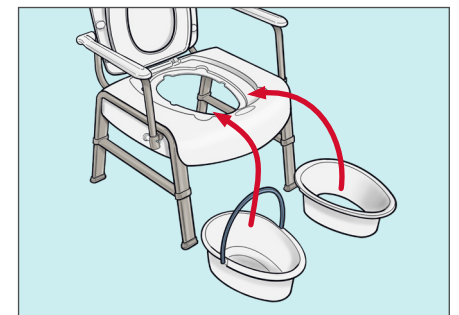


## Optional Backrest Attachment

Your commode can be used over a toilet with the splash guard installed. Consider leaving the Backrest off when using the commode over a toilet.


1. For use away from the toilet, replace the splash guard with the pail and attach the Backrest for a furniture look and feel. Slide lip of Backrest between rear support and seat. The Backrest lip will be underneath the seat.

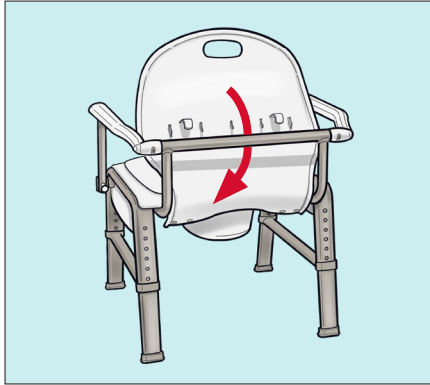

2. Securely clip the Backrest to the rear support bar.

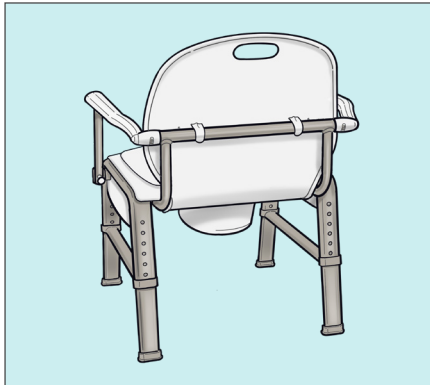



3. Tilt the commode forward and attach the bottom lip of the Backrest to the underside of the Seat using included thumb screws.

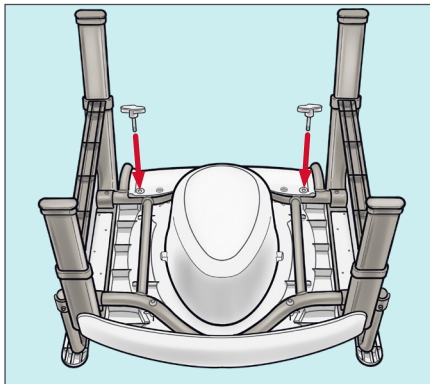

 **Do not over-tighten.**



## How to Adjust the Height

1. Press the spring pin on one leg and pull the lower portion of the leg to the desired height.
2. Repeat for the other three legs ensuring they are all set to the same height.
3. Before using your new commode, **confirm all spring pins are securely locked in place.** The sound of the click will be audible.

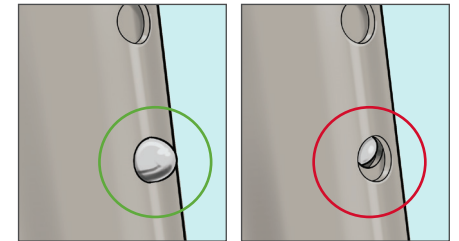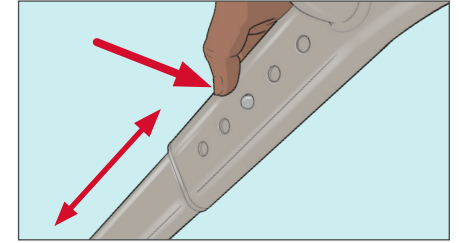

## Use and Tips

- Arm rests can be lowered to provide more space or facilitate side transfers. Pull locking pin knob to release and lower each arm.
- To return, pull Armrests up to engage locking pins into brackets on each side of the commode.

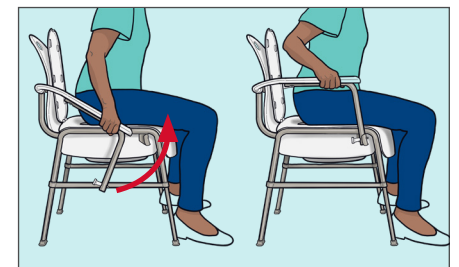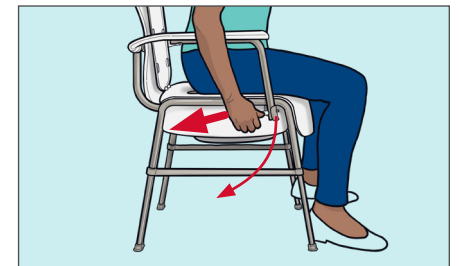


## Important

---

1. Do not exceed weight capacity of 350 lbs.
2. To ensure an even and stable height, make sure that the height-adjustment buttons are securely through and visibly protruding through the same respective height adjustment hole on each leg.
3. Check height adjustment points and all joints and fittings before each use to ensure product is secure and stable.
4. If commode is being used over the toilet, the toilet seat must be down to ensure a stable fit over the toilet. The toilet lid must be lifted or removed before using the commode.
5. Ensure the arms are fully locked and secured into the side of the chair before putting any weight on them.
6. When unlocking and stowing the arms, carefully slide each arm back to avoid any swing that could harm the user or surroundings.
7. Do not use if the rubber leg tips look cracked or worn.
8. Wipe clean with a nonabrasive cleaner and warm water.
9. Do not stand on commode.

**Distributed by: CVS Pharmacy, Inc.**  
One CVS Drive, Woonsocket, RI 02895  
© 2021 CVS/pharmacy  
CVS.com® 1-800-SHOP CVS  
Made in China  
V-37417

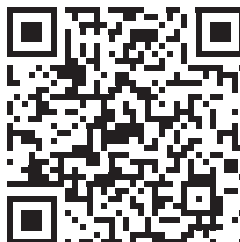

Scan here to learn more about our partnership with Michael Graves Design and view additional products.