


MICHAEL GRAVES DESIGN

Quick Fold Cane

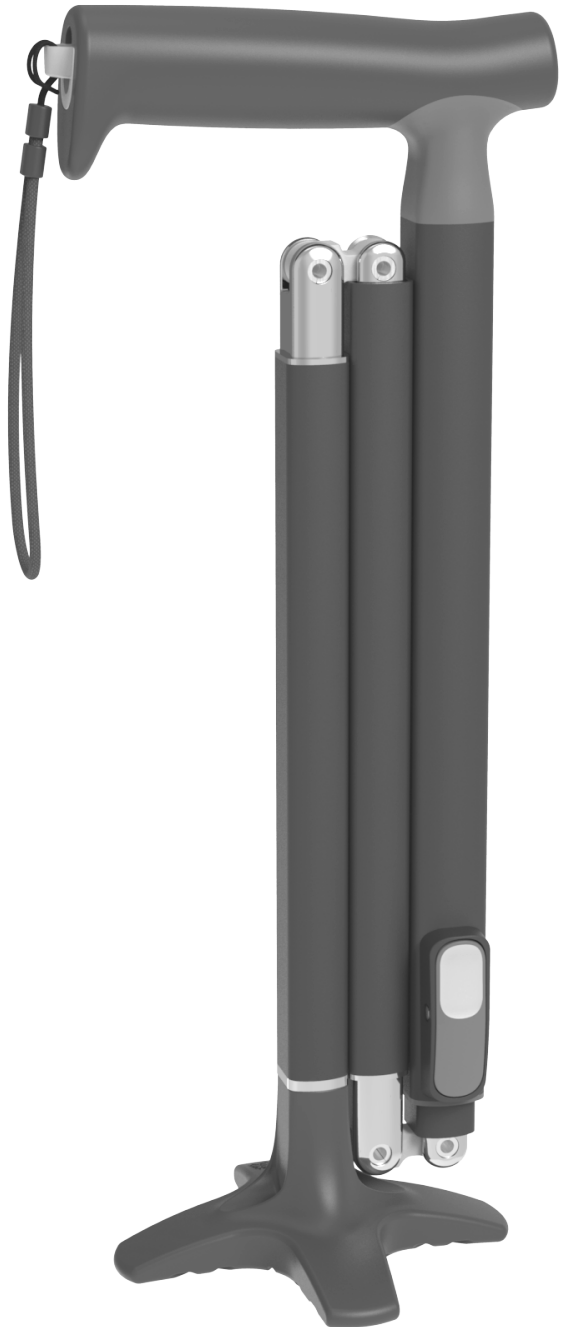

USER MANUAL

Maximum weight capacity - 300lbs.

MICHAEL GRAVES DESIGN


Quick Fold Cane

Michael Graves Design set America's expectation that great design should be affordable and accessible. With Michael Graves paralysis, the brand focused its design power on improving people's healthcare experiences too. This cane was designed to help you live your complete life full of moments of joy.


To learn more or to say hi visit us at:
www.MichaelGravesDesign.com

SETTING THE HEIGHT


1. Press the yellow area on the locking lever to unlock the height.

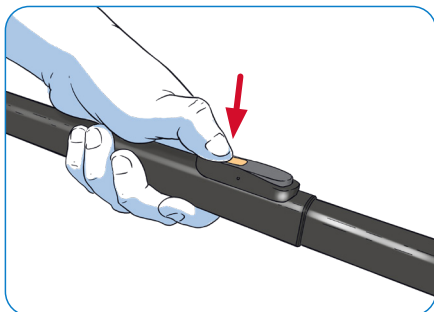


2. While pressing the locking lever, slide the lower shaft in or out of the upper shaft to adjust the height. Release locking lever when the height is set right. Ensure the pin has reset securely.

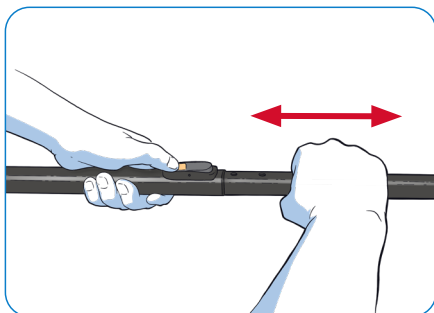


3. Set the height correctly by aligning the top of the grip with the crease in your wrist while standing straight and looking forward. Note: right height may vary based on your shoes or when barefoot.

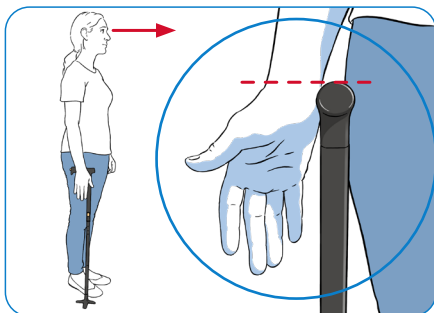


UNFOLDING THE CANE

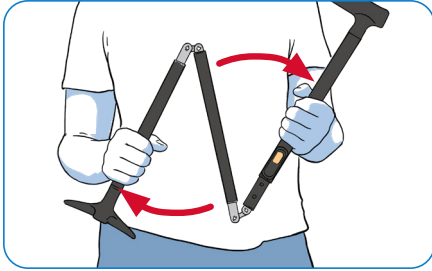

1. Pull the cane sections apart to break the magnetic bond holding it closed.

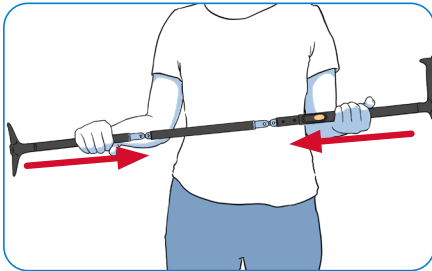


2. Straighten the cane out and let the hinges retract into the shafts.

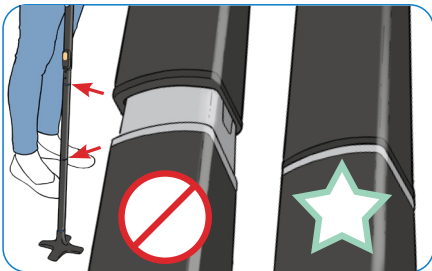


3. Be sure that each shaft is fitting tightly against the next shaft and no hinges are showing. If the cane was stored at its lowest height setting, be sure to reset the cane to the appropriate height before use.


FOLDING THE CANE

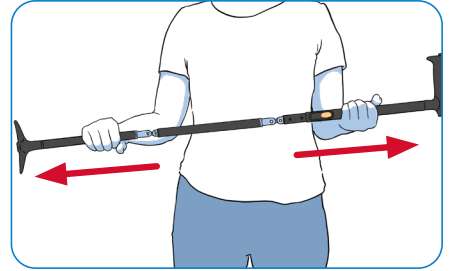

1. Grasp the bottom shaft section and top shaft section. Pull away from each other, exposing the hinges.

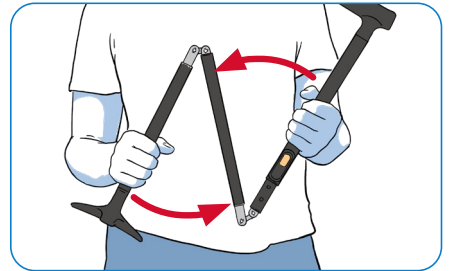


2. Once the internal hinges are exposed, fold the sections together.

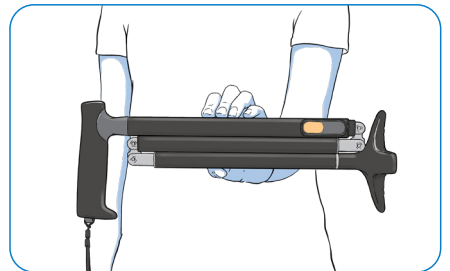


3. Internal magnets will help hold the sections together. Honestly, it feels magical! For the most compact storage, set the cane height to its lowest position.

SWAPPING THE TIPS

1. The tips are designed to be removed and attached with relative ease and without the need for tools. To remove, grasp the tip with your hand, or between your feet, and pull.

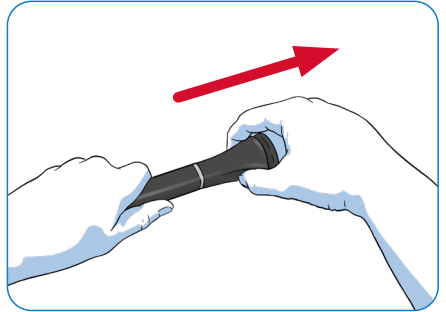

2. To attach a tip, align it to the cane's bottom post so the grooves on the post line up with the internal ridges inside the tip. Press the tip on until it is fully covering the bottom post.

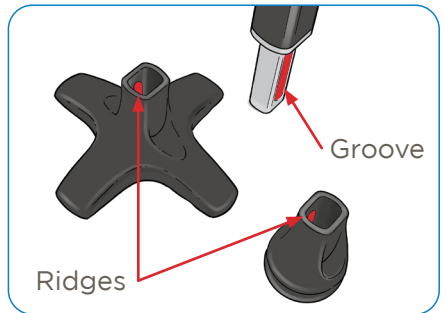

IMPORTANT

- This product contains magnets which may interfere with pacemakers, defibrillators, or other medical devices. Maintain a safe distance of separation between your medical device and this product. If you suspect this product is interfering with your medical device, stop using it and contact a medical professional.
- Do not exceed weight capacity of 300 lbs. Canes are a balance aid and do not support a users full weight.
- Use ONLY Michael Graves Design cane tips.
- Wipe clean with a non-abrasive cleaner and warm water.
- Do not submerge in water.

www.MichaelGravesDesign.com

Designed in Princeton, NJ - Made in China

© 2021 Michael Graves Design Group, Inc.