

 **CVS**Health.

EASY FOLD
**TRAVEL
WALKER**

by

**MICHAEL
GRAVES
DESIGN**

superior design
+ function for the
best daily care

INSTRUCTION MANUAL



CVS Health® Easy Fold Travel Walker



by

**MICHAEL
GRAVES
DESIGN**

At CVS Health®, we are passionate advocates for transforming health care into a more personalized experience through new technologies and innovations that can help improve the quality of your everyday life. Architect and designer Michael Graves shared these values and our partnership with Michael Graves Design brings you a collection of thoughtfully created adaptive products that deliver superior comfort, style, and functionality. It is through partnerships like this and many others that we continue to bring you innovations in care.

ERGONOMIC

- Innovative grip angle helps improve user experience
- Unique center column helps fold, store and creates greater clearance


CUSTOMIZABLE

- Easier & more efficient to adjust the height
- Integrated brakes help with stability when using wheels

SIMPLE


- Easy to use trigger helps make folding easier
- Tool Free Assembly

Parts for your new Easy Fold Travel Walker

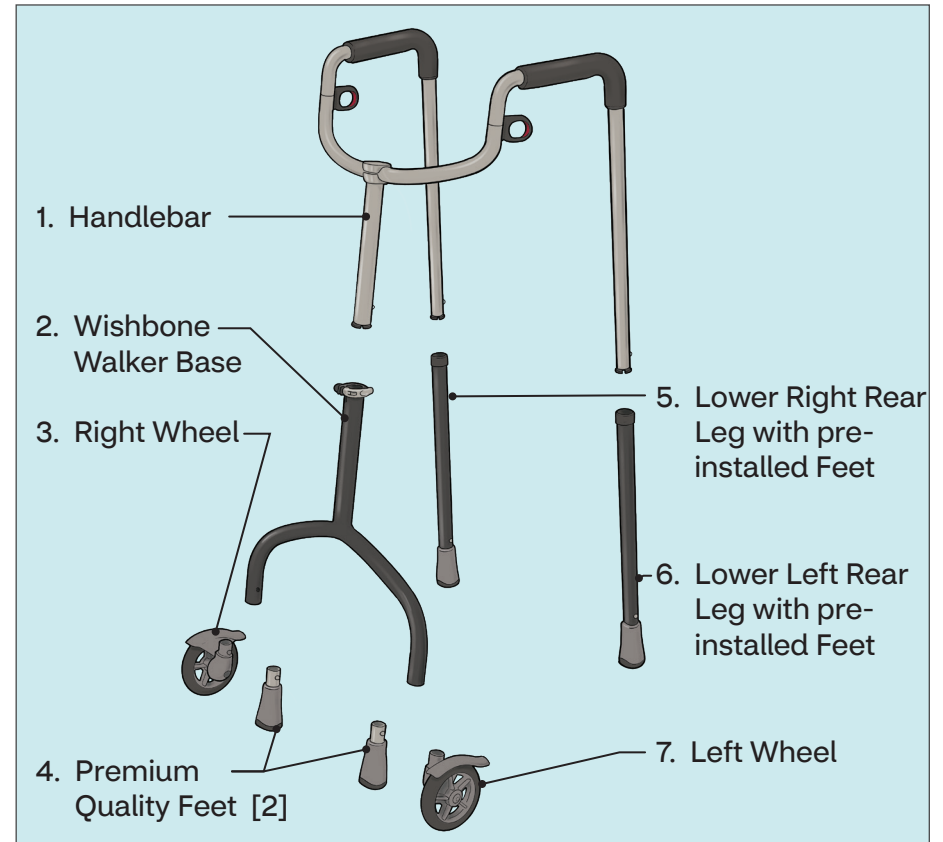


The Active Angle

Most walker grips are parallel to the ground. These walker grips angle down towards the front to help improve your posture and wrist comfort.

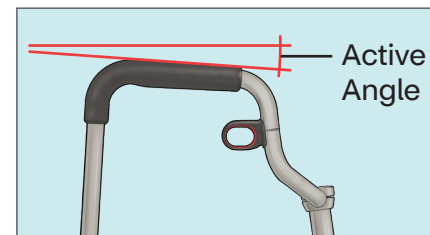


Premium Quality Feet

Ditch the yellow tennis balls because these are contoured to glide across surfaces. When you lean on the walker a rubber foot emerges and securely grips the ground.

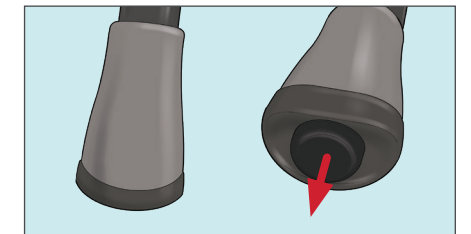

Front Wheels or Four Premium Quality Feet?

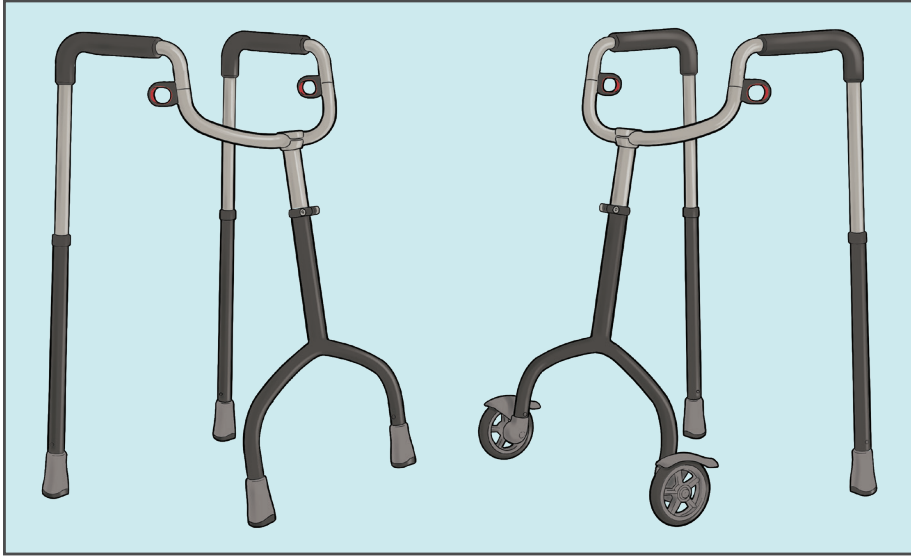

First, determine whether you need Wheels or Feet on the front of your walker.

If you are unsure, check with your healthcare provider.


This walker is not designed for use with wheels on all four legs.

How to Attach the Premium Quality Feet

1. Align one of the Feet to the Wishbone Base, and insert it into the Base.


The two spring pins on the Premium Quality Foot will click into place on the leg when properly attached.

2. Repeat for the other Premium Quality Foot.

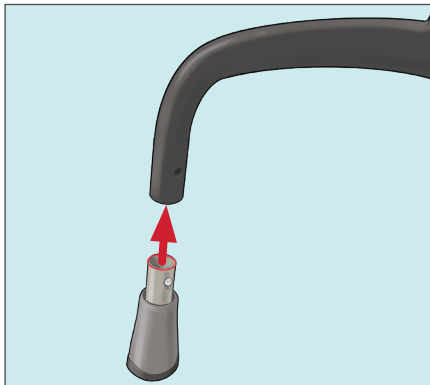


How to Attach the Wheels


1. The Wheels are Left/Right side specific. Each Wheel should be attached so that they are on the outside of the Wishbone Base, with the fender tab facing the same direction as the height adjustment holes on the Wishbone Base.

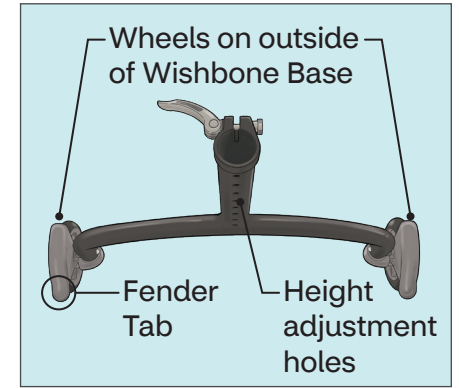

2. Insert the Left Wheel into the Wishbone Base. **The spring pin on the Wheel will click into place** on the Wishbone Base when properly attached.
3. Repeat for the Right Wheel. **The spring pin on the Wheel will click into place.**

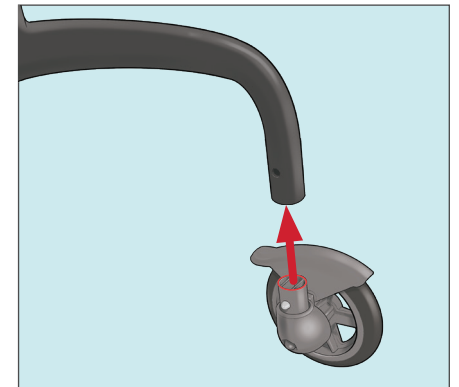


4. Engage the brakes for the rest of the assembly process by pressing the fender tab down. The Wheels should not be able to roll. If they still roll, do not use and contact customer service.

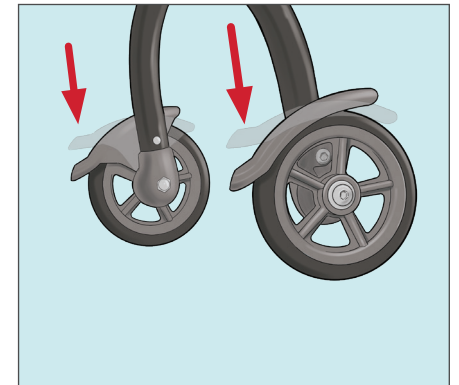


How to Assemble your Easy Fold Travel Walker


1. Open the Cam-Lock Lever on the Cam-Lock at the top of the Wishbone Base.

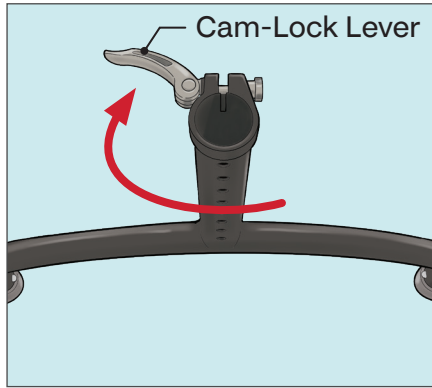


2. Press the spring-pin on the front tube of the Handlebar and insert it into the Wishbone Base until the spring pin pops into one of the height adjustment holes on the Base with an **audible “click”** (all height adjustment holes face the back of the walker).

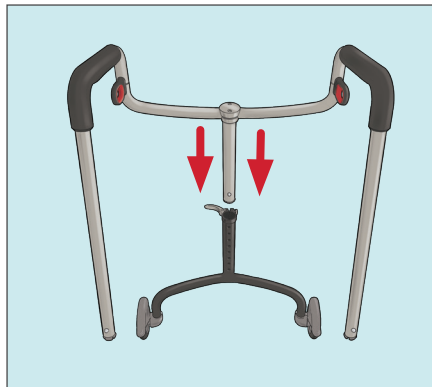


3. Press the spring pin on the back leg of the Handlebar and insert it into a Lower Rear Leg until the spring pin pops into one of the height adjustment holes on the Lower Rear Leg with an **audible “click”** (All height adjustment holes face the back of the walker).

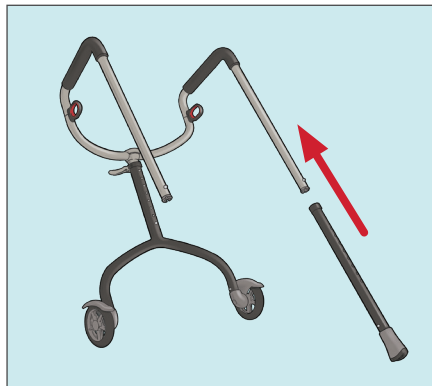

4. Repeat step 3 with the other Lower Rear Leg. Ensure spring pin pops into one of the height adjustment holes on the Lower Rear Leg with an **audible “click.”**

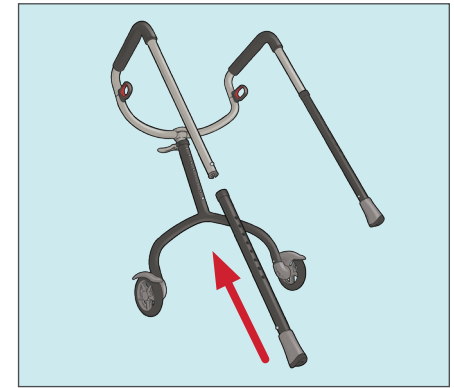


5. **Make sure all spring pins are locked securely in place.**

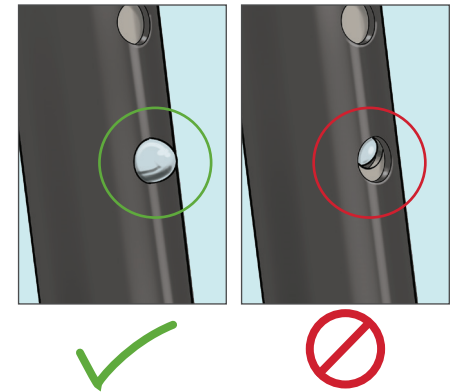


6. Close the Cam-Lock Lever on the front column.

If the Cam-Lock Lever does not feel tight or is able to move, do not use. You must turn the adjustment knob opposite the lever to adjust it so the lever closes tightly.

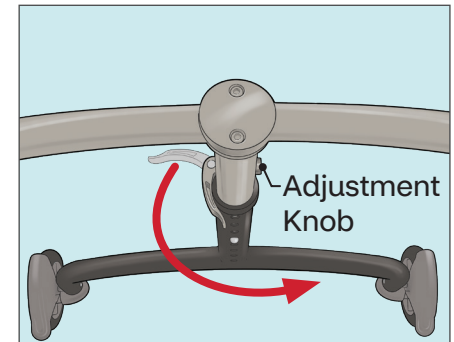

* See instructions on next page for height adjustment.


How to Adjust the Height of your new Walker

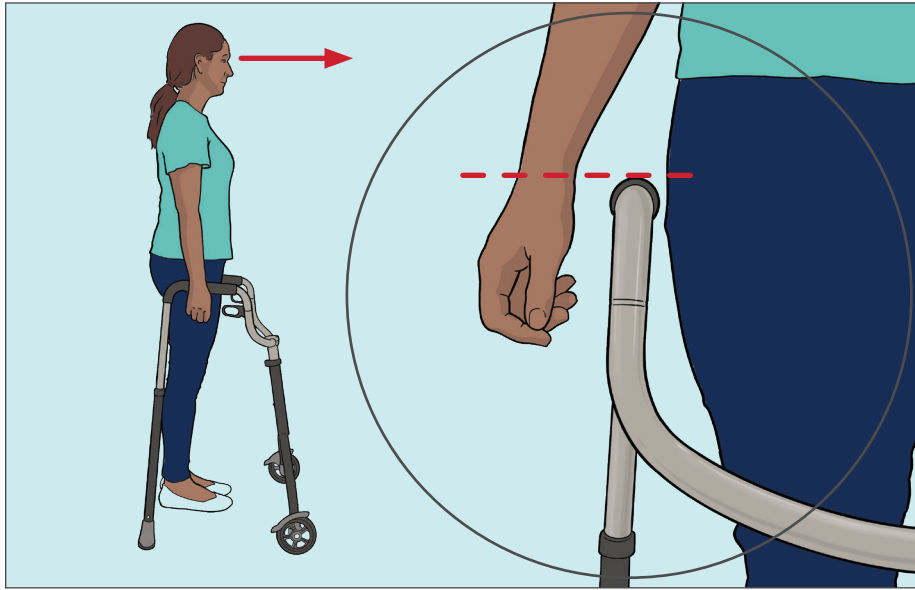


1. Typically, the walker height should be set so the grips align with the inner crease of your wrist when you are standing up straight and looking forward. Confirm what is right for you with your healthcare provider.

This is the same whether you use 4 feet, or 2 wheels and 2 feet.

2. Press the spring pin on a Lower Rear Leg and set the height to the appropriate hole as indicated in step 1 above. Repeat for the other Lower Rear Leg, **ensuring the legs are set at the same height.**

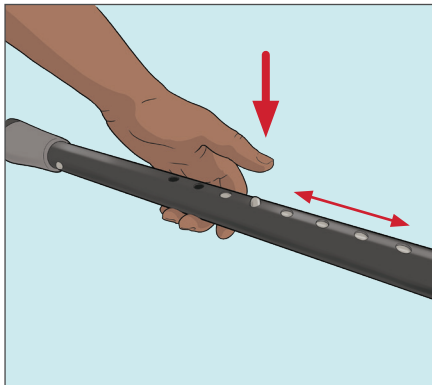


3. Open the Cam-Lock Lever on the Front Column.

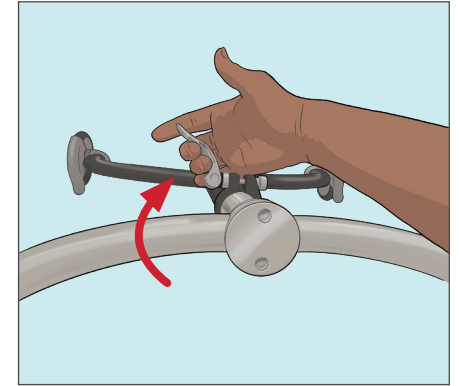


4. Press the spring pin on the Front Column and adjust the height to the **same hole** as the Lower Rear Legs.

Never set the Front Column to a different height from the Rear Legs.

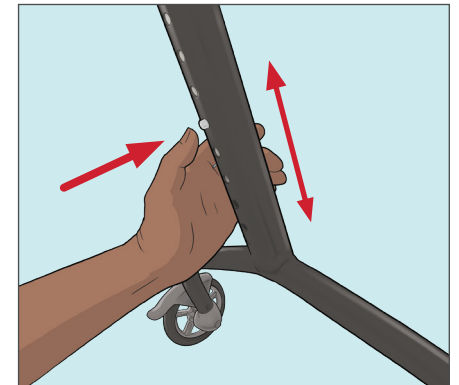


5. Close the Cam-Lock Lever on the front column.

If the Cam-Lock Lever does not feel tight or is able to move, do not use. You must turn the knob opposite the Cam-Lock Lever to adjust it so the Lever closes tightly.

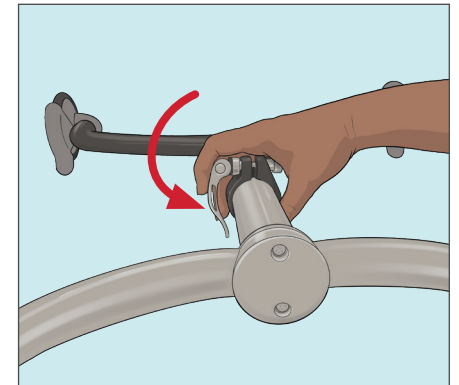


Use and Tips


To fold, pull each trigger upward and rotate each grip inward until trigger drops back down into place. It does not matter which side you fold inward first.

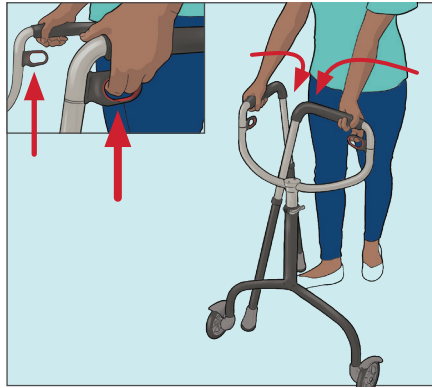

To lock each wheel, step down on the fender tab to set the brake. If they still roll, do not use and contact customer service.

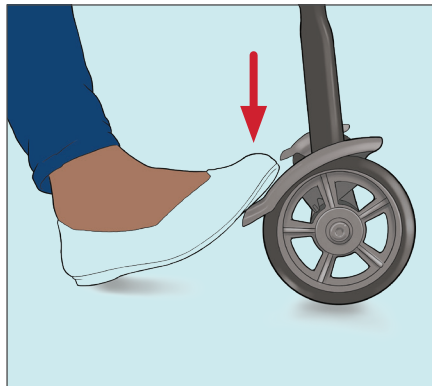


Make sure the brakes are locked when leaning folded walker against a wall or other object.

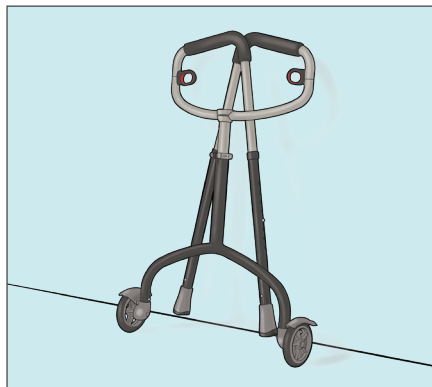


Use and Tips

To unfold, pull each trigger upward and rotate each grip outward until trigger drops down into place.

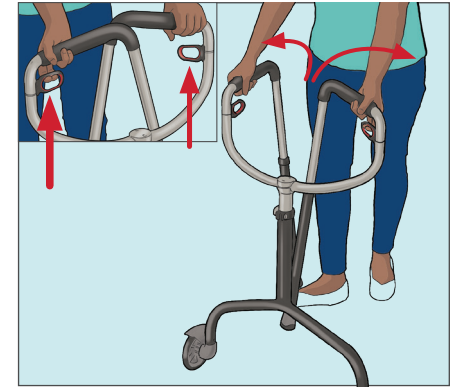


To unlock each wheel, push up on the fender tab to release the brake.

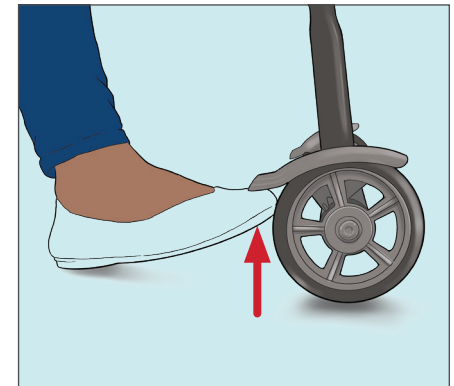

Inspect wheels and feet regularly for wear/damage. To remove wheels or feet, press the spring pins above the wheel or foot, then pull the wheel or foot to remove.


Replacement walker wheels are available on **CVS.com**, item 243577.

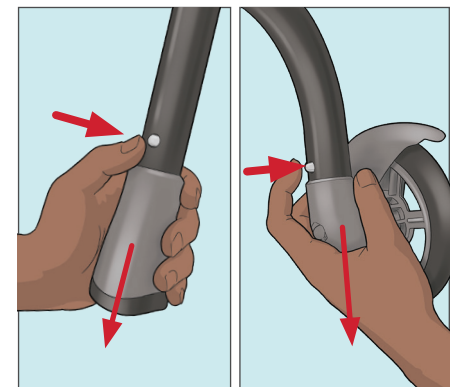

Important

1. Do not exceed weight capacity of 300 lbs.
2. To ensure an even and stable height, make sure that the height-adjustment buttons are securely through and visibly protruding through the same respective height adjustment hole on each leg.
3. Check height adjustment points and all joints and fittings before each use to ensure product is secure and stable.
4. Before each use, ensure that the frame is fully unfolded and stable, and wheels and feet are securely affixed to the walker. Cam-lock lever should be in a secure, closed position.
5. Do not leave your Walker in moist places; dry it immediately if it exposed to moisture.
6. All feet and wheels must be in contact with the ground during use to prevent injury or fall.
7. Do not use for support on stairs or ramps.
8. Do not use to move backwards.
9. Do not use the frame of the walker to hang or store any personal belongings.
10. Do not use the walker as an aid stand up as wheels could still roll and loss of balance or injury could occur.
11. When engaging the brake on or off, ensure that your other foot is firmly grounded and you are well balanced before using your foot or bending down.
12. Check walker wheels and feet periodically for wear and replace as needed. Replacement wheels are available on **CVS.com**, part number 243577.

Distributed by: CVS Pharmacy, Inc.
One CVS Drive, Woonsocket, RI 02895
© 2021 CVS/pharmacy
CVS.com® 1-800-SHOP CVS
Made in China
V-37417

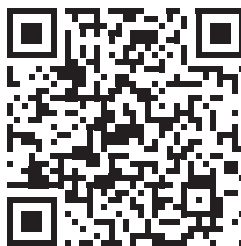

Scan here to learn more about our partnership with Michael Graves Design and view additional products.