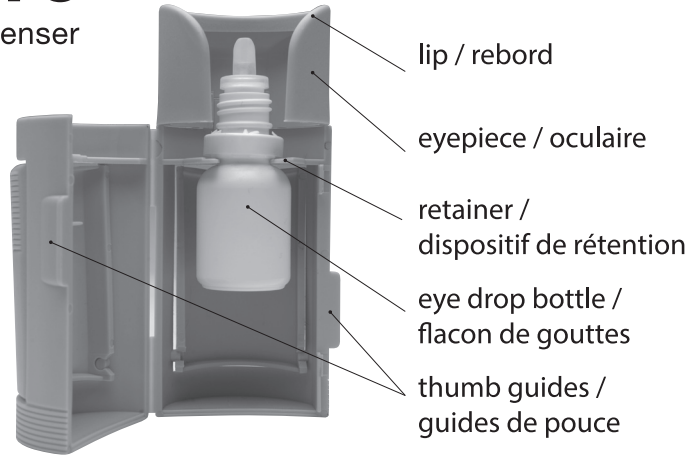
Opticare®

eye drop dispenser

REF CG101

Opticare eye drop dispenser targets the eye and squeezes the bottle

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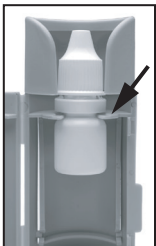
2. Place the bottle inside Opticare so that the retainer fits firmly around the base of the bottle neck.

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2a. Some bottles require a bottle collar to ensure a secure fit. Slide collar (supplied) around the base of the bottle neck.

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2b. Place bottle inside Opticare as per step 2. The optional collar sits on top of the retainer and the base of the bottle neck sits below the retainer.

3. Close Opticare by clicking sides together securely. Open eyepiece to remove bottle cap before use.

4. Use finger to pull down lower eye lid. Tilt head back and rest lip of eyepiece gently against upper eye socket. Gently squeeze the side panels to deliver drop(s) into the eye. Replace bottle cap after use.

To ensure proper hygienic care, always wash Opticare in hot soapy water before installing a new eye drop bottle.

Opticare is designed for single person use and is compatible with a number of eye drop bottles. For a list, visit www.opticare.uk

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Email: info@cameron-graham.co.uk
www.opticare.uk

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Canada
McArthur Medical Sales Inc.
1846 5th Concession West
Rockton, Ontario LOR 1X0
www.mcarthurmedical.com

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Cameron Graham Ltd