English

AH! YES® COCO is the same product as YES® OB in Applicators – rebranded for the USA. AH! YES® offers the same excellent customer service and great products for our customers in the USA. Please read this leaflet carefully before use

AH! YES® COCO – plant-oil based personal lubricant

AH! YES® COCO is a luxurious plant-oil based natural lubricant, with organic coconut and jojoba in boxes of 6 or 30 convenient pre-filled single-use applicators. AH! YES® COCO nourishes and conditions skin, giving temporary relief from vaginal dryness, lasting lubrication and comfort.

Description

- Natural, long-lasting plant-oil based intimate lubricant.
- All ingredients chosen for their beneficial skin properties.
- Enables discreet and hygienic delivery of the right amount to the right place.
- Enhances pleasure and ease in love making.
- The natural plant-based alternative to silicone lubricants.
- Not made with concerning ingredients (no glycerine, no hormones, no gluten, no petrochemicals, no parabens, no synthetic Silicone oils).
- Recommended by Health Care Professionals and trusted by women to ease and enhance comfort and provide lasting lubrication during sexual intercourse.
- Safe for use with silicone toys but not latex compatible.
- Not condom compatible
- Plant oils condition skin, no need to wash off.
- Can be used in showers, bathtubs and hot tubs (its long lasting)
- Certified Organic by the Soil Association (Bristol, UK), with 94% Certified Organic ingredients

Directions for use:

AH! YES® COCO assists your love making by enabling penetration, increasing pleasurable glide and sensation. When you want to reach higher in the vagina, this product offers you a discreet and convenient way to deliver lubrication in the right amount to the right place.

- To enhance pleasure and ease in love-making a 5ml applicator is typically sufficient, but more may be used if required.
- Applicators are single use only, dispose of applicator after use. DO NOT REUSE APPLICATOR. If additional lubrication is needed, use a new applicator or apply lubricant by hand

See diagram for instructions

Indications for use

AH! YES® COCO is a personal lubricant for vaginal and penile application, intended to moisturize and lubricate to enhance the ease and comfort of intimate sexual activity and supplement the body's natural lubrication AH! YES® COCO is NOT compatible with natural rubber latex, polyurethane or polyisoprene condoms.

Ingredients

AH! YES® COCO: Sunflower seed oil*, Coconut oil*, Jojoba seed oil*, Hydrogenated vegetable oil, Grape seed oil*, Tocopherol (Vitamin E).

* Organically grown ingredients.

Certified 94% organic by the Soil Association, Bristol, UK.

Storage: Store in a cool dry place between 41-86°F (5-30°C). Use before expiry date. Once opened use applicators within 48 hours. Keep out of direct sunlight.

Caution: AH! YES® COCO is not a contraceptive or a spermicide.

Warning: If the twist off cap is damaged or opened on any of the applicators enclosed DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use. If symptoms persist, seek medical advice.

NB. AH! YES® COCO is not suitable for use with the ThinPrep Pap smear test (see website)

Why certified organic?

When you buy something that is certified organic, not only are you making the best choice for your own wellbeing, your purchase is also supporting a system that respects human health, animals, wildlife and the environment. Certified organic products meet stringent standards and organic certification is your assurance that our ingredients and products have been grown and processed according to strict procedures. Only certified organic products have their production, processing, distribution and sales assessed to ensure the finest organic and ecological integrity.

Packaging and recycling

AH! YES® minimizes the environmental impacts of all our packaging. So please do help us achieve that aim. All our bottles are made from polyethylene which is the most widely accepted plastic by recycling centers. The cartons, boxes and package inserts can all be recycled.

100% Guarantee: AH! YES® products are 100% satisfaction guaranteed. Please contact us by telephone, live chat on the website or email if you have any concerns or questions about our products.

Certified Organic by: The Soil Association, Bristol, UK.

Approved by: The Vegetarian Society.

Manufactured in Great Britain for

Bio-Tech Lubricants Ltd, Contact: The Timber Barn, Selborne Road, Greatham, Liss, Hampshire, GU33 6HG, UK.

AH! YES® logos are registered trademarks of YES Syzygy Ltd.

Please contact us by telephone or email if you have any concerns or questions.

Contact us

USA Toll Free: 1-888-777-0125

Website: www.ahyes.org

Email: help@ahyes.org

Our love for Planet Earth is reflected in everything we do – creating glorious products that are amazing to use, kind to the environment, respectful of all life and natural.

Truly, madly, deeply natural.

Sarah and Susi

AH! YES® Founders

See our full range overleaf

XAMIFUC_V1_04-22

MADE WITH SAFE INGREDIENTS

MADE SAFE

MADE SAFE SCREENS
INGREDIENTS FOR KNOWN:

- ✓ BEHAVIORAL TOXINS
- ✓ CARCINOGENS
- ✓ DEVELOPMENTAL TOXINS
- ✓ ENDOCRINE DISRUPTORS
- ✓ FIRE RETARDANTS
- ✓ HEAVY METALS
- ✓ NEUROTOXINS
- ✓ HIGH RISK PESTICIDES
- ✓ REPRODUCTIVE TOXINS
- ✓ TOXIC SOLVENTS
- ✓ HARMFUL VOCs

It is imperative to follow manufacturer guidelines for use of certified products.

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Fabricante

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AH! YES® COCO

Plant-oil based personal lubricant

PRE-FILLED APPLICATORS

AH! YES

Natural Intimate Health

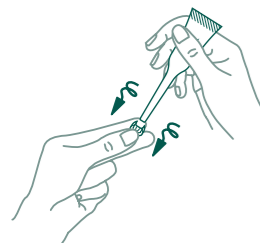
XAMIFUC_V1_04-22

Directions for use – AH! YES® COCO in pre-filled Applicators

When you want to reach higher in the vagina, this product offers you a discreet and convenient way to deliver lubrication and moisture in the right amount to the source of discomfort.

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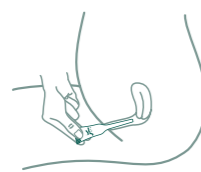
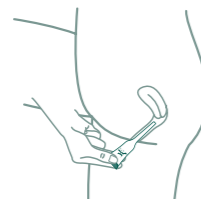
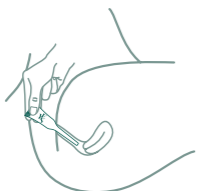
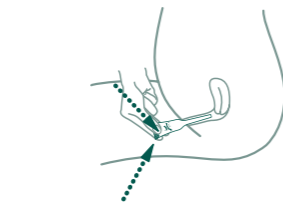
The AH! YES® range

Whether you use a little or a lot, for pleasure or for health, discover the full AH! YES® collection of natural and organic intimate products, in a range of sizes and variants to meet your differing needs.

AH! YES® WB

Is an intimate lubricant formulated to hydrate and revitalize dry vaginal tissues and alleviate discomfort. AH! YES® WB is pH matched to the typical vaginal environment. Maintaining typical vaginal pH may help prevent Thrush and bacterial infections.

AH! YES® OB

Is a plant-oil based, longer-lasting lubricant and intense moisturizer, which is soothing and richly emollient. Nourishes delicate intimate tissues. Not condom compatible.

AH! YES® VM (Vaginal Moisturizer)

Is a long-lasting vaginal moisturizer. Designed to match the typical vaginal environment with none of the skin damaging or concerning chemicals typically found in vaginal moisturizers.

AH! YES® CLEANSE

Is a sensitively formulated intimate wash, designed to respect the delicate vaginal ecology, with a refreshing and effective cleansing action. Available in Unfragranced and Rose.

AH! YES!

Natural Intimate Health