## English

AH!YES is the USA version of YES products, offering the same great product and excellent customer service, but with a new name especially for the USA!

Please read this leaflet carefully before use

### AH!YES OB – plant-oil based personal lubricant

AH!YES OB is a luxurious plant-oil based natural lubricant, whose rich oils and butters nourish and condition skin, giving lasting lubricity and comfort with relief from vaginal dryness.

### Description

- Natural plant-based intimate lubricant.
- For lasting lubricity and glide.
- Dual function: outstanding lubrication and great body massage.
- With a velvety sensuous feel.
- The natural plant-based alternative to silicone lubricants.
- Not made with any known skin irritants (no glycerine, no hormones, no gluten, no petrochemicals, no parabens).
- Recommended by Health Care Professionals and trusted by women to soothe and relieve discomfort during sexual intercourse.
- No known side effects.
- Safe for use with silicone toys but not latex compatible.
- Not condom compatible
- Plant oils and butters condition skin, no need to wash off.
- Can be applied freely and generously, according to the body's needs.
- Can be used in showers, bathtubs and hot tubs (its long lasting)
- Certified Organic by the Soil Association (Bristol, UK), with 99% Certified Organic ingredients.

AH!YES OB is available in a range of tube sizes.

### Directions for use:

AH!YES OB assists your love making by enabling penetration, increasing pleasurable glide and sensation. A rich, nourishing cream that liquefies into a smooth oil on skin, it is easy to dispense and distribute, and is designed for finger application, which allows you to decide how much you want to put where. Simply squeeze the cream on to your hand, warm it by rubbing between your fingers to

ensure a smooth velvety quality, and use your finger tips to apply the desired amount wherever you want it. (No applicator included). AH!YES OB is also perfect for massage.

**Care note:** AH!YES OB may occasionally present with a slightly grainy texture, which is easily remedied. Simply place tube in a sealed bag (don't forget to squeeze the air out of the bag before sealing) and immerse it in a mug of hot water, until it is completely liquid, shake to mix, then place in the refrigerator to cool quickly. This will restore the perfect smoothness of the AH!YES OB texture. This is a function of using exclusively natural ingredients which are temperature sensitive. The product is perfectly safe to use.

**Pleasure tip:** Experience the Double Glide effect – apply a layer of WB on top of OB for a lasting and new dimension in love making.

### Indications for use

AH!YES OB is a personal lubricant for vaginal and penile application, intended to moisturize and lubricate to enhance the ease and comfort of intimate sexual activity and supplement the body's natural lubrication AH!YES OB is NOT compatible with natural rubber latex polyurethane or polyisoprene condoms.

### Ingredients

**AH!YES OB:** Helianthus annuus (sunflower) seed oil\*, Butyrospermum parkii (shea butter)\*, Prunus dulcis (sweet almond oil)\*, Cera alba (bees wax)\*, Theobroma cacao (cocoa) seed butter\*, Tocopherol (vitamin E).

\* Organically grown ingredients.

Certified 99% organic by the Soil Association, Bristol, UK

**Storage:** Replace cap when not in use. Store in a cool dry place between 5+30°C. Use before expiry date. Keep out of direct sunlight.

**Caution:** AH!YES OB is not a contraceptive or a spermicide.

**Warning:** If the seal on any tube enclosed is damaged or opened DO NOT use, return to place of purchase. If irritation or discomfort occurs, discontinue use. If symptoms persist, seek medical advice.

### Why certified organic?

When you buy something that is certified organic, not only are you making the best choice for your own wellbeing, your purchase is also supporting a system that respects human health, animals,

wildlife and the environment. Certified organic products meet stringent standards and organic certification is your assurance that our ingredients and products have been grown and processed according to strict procedures. Production, processing, distribution and sales are monitored to maintain the finest organic and ecological integrity.

### Packaging and recycling

AH!YES aims to minimize the environmental impacts of all our packaging. So please do help us achieve that aim. All our bottles are MDPE. Polyethylene (MDPE) is the most widely accepted plastic by recycling centers. The cartons, boxes and package inserts can all be recycled.

**100% Guarantee:** AH!YES products are 100% satisfaction guaranteed. Please contact us by telephone or email if you have any concerns or questions about this guarantee.

**Certified Organic by:** The Soil Association, Bristol, UK.

**Approved by:** The Vegetarian Society.

### Manufactured in Great Britain for

Bio-Tech Lubricants Ltd, Contact: The Timber Barn, Selborne Road, Greatham, Liss, Hampshire, GU33 6HG, UK.

AH!YES logos are registered trademarks of YES Syzygy Ltd.

Please contact us by telephone or email if you have any concerns or questions.

### Contact us

**USA Toll Free:** 1-888-777-0125

**Website:** www.ahyes.org

**Email:** help@ahyes.org

Our love for Planet Earth is reflected in everything we do – creating glorious products that are amazing to use, kind to the environment, respectful of all life and natural.

Truly, madly, deeply natural.

Sarah and Susi  
AH!YES Founders

See our full range overleaf

## MADE WITH SAFE INGREDIENTS

# MADE SAFE

MADE SAFE SCREENS  
INGREDIENTS FOR KNOWN:

- ✓ BEHAVIORAL TOXINS
- ✓ CARCINOGENS
- ✓ DEVELOPMENTAL TOXINS
- ✓ ENDOCRINE DISRUPTORS
- ✓ FIRE RETARDANTS
- ✓ HEAVY METALS
- ✓ NEUROTOXINS
- ✓ HIGH RISK PESTICIDES
- ✓ REPRODUCTIVE TOXINS
- ✓ TOXIC SOLVENTS
- ✓ HARMFUL VOCs

It is imperative to follow manufacturer guidelines for use of certified products.

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**Aprobado por:** The Vegetarian Society.

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Bio-Tech Lubricants Ltd, Contacto: The Timber Barn, Selborne Road, Greatham, Liss, Hampshire, GU33 6HG, UK.

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AH!  
YES  
Natural Intimate Health

# The AH! YES range

Whether you use a little or a lot, for pleasure or for health, discover the full AH! YES collection of natural and organic intimate products, in a range of sizes and variants to meet your differing needs.

**AH! YES WB**  
Is an intimate lubricant formulated to hydrate and revitalize dry vaginal tissues and alleviates discomfort. AH! YES WB is pH matched to the typical vaginal environment. Maintaining typical vaginal pH may help prevent Thrush and bacterial infections.

**AH! YES OB**  
Is a plant-oil based, longer-lasting lubricant and intense moisturizer, which is soothing and richly emollient. Nourishes delicate intimate tissues. Not condom compatible.

**AH! YES VM (Vaginal Moisturiser)**  
Is a long-lasting vaginal moisturizer. Designed to match the typical vaginal environment with none of the skin damaging or concerning chemicals typically found in vaginal moisturizers.

**AH! YES CLEANSE**  
Is a sensitively formulated intimate wash, designed to respect the delicate vaginal ecology, with a refreshing and effective cleansing action. Available in Unfragranced and Rose.