



emulait™

BREAST PUMP ADAPTER 3-PACK
USER INSTRUCTION MANUAL



What's Included



Narrow Neck Adapter x 1



Tall Wide Neck Adapter x 1



Short Wide Neck Adapter x 1

Before First Use

Clean hands and surfaces thoroughly before handling. Sterilize the Emulait breast pump adapters in boiling water for about five minutes. Let them cool until they are no longer warm to the touch. To ensure hygiene, rinse and dry thoroughly.

How to Assemble



Remove the Emulait Nipple from the bottle. Select compatible adapter and hold it with the wider end facing towards the bottle. Gently twist adapter onto the bottle. Connect your breast pump.

How to Disassemble



Disconnect your breast pump from the adapter. Gently untwist the adapter from the bottle.

Cleaning and Sterilizing

Fully remove the Emulait breast pump adapter from your Emulait bottle. Wash adapter in hot water and soap. Rinse and dry thoroughly.

The Emulait breast pump adapters are dishwasher safe (top rack only), and safe for use in an electric sterilizer, microwave sterilizer or boiling water. Use sterilizing solutions or sterilizers strictly according to the manufacturer's instructions. Steam can cause severe skin burns, use caution when boiling on the stove or using any sterilizing equipment. Avoid unnecessary handling of sterilized items and equipment.

For Your Child's Safety and Health

⚠ WARNING!

Inspect carefully before every use.

Always check the Emulait breast pump adapters for any cracks or signs of damage.

Dispose of the Emulait breast pump adapters at the first signs of damage or weakness.

When not in use, keep the Emulait breast pump adapters out of the reach of children.

Always use this product with adult supervision.

The Emulait breast pump adapters are not toys and should not be given to children to play with or suck on.

Yours Truly