

MyPerioPath® Collection Recommendations & Oral Rinse Collection Instructions

When following:

Use of over-the-counter oral care products

Use of prescription strength antimicrobial or antibiotic rinses such as Chlorhexidine or stannous fluoride products

Dental procedures that disrupt biofilm such as scaling and root planing or prophylaxis

Administration of systemic antibiotics for pre-procedural medication or chronic medical conditions

Administration of systemic antibiotics for acute conditions

Application of local antibiotics

Wait before sample collection:

30 minutes

24 hours


2 weeks

No wait necessary

6 weeks

4 weeks

Oral Rinse Collection Instructions:

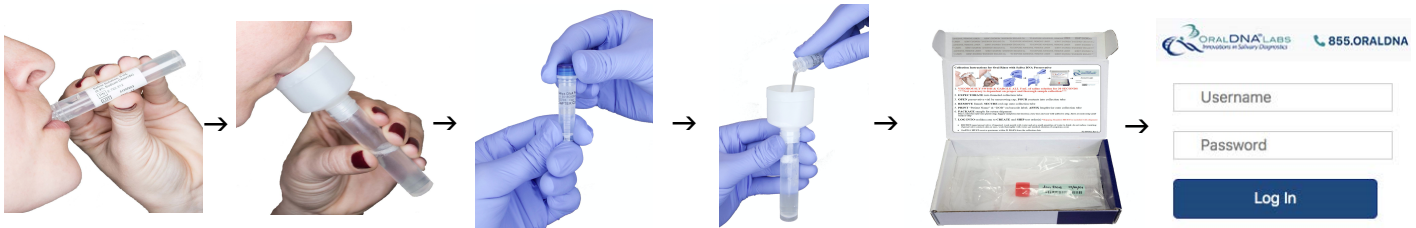

1. **VIGOROUSLY SWISH & GARGLE ALL 5 mL** of saline solution for **AT LEAST 30 SECONDS**
2. **SPIT** saline into funneled collection tube; **POUR** preservative (blue cap) into collection tube
3. **DISCARD** funnel; **SEAL** collection tube with red cap
4. **PRINT** "Patient Name" & "DOB" on barcode label; **AFFIX** lengthwise onto collection tube
5. **PACK** box with bagged collection tube; **CLOSE & SEAL** box
6. **LOGIN** to oraldna.com to **CREATE** and **SHIP** test order(s)

OralDNA® Labs **MUST** receive specimens within **21 DAYS** from the collection date

DO NOT ingest preservative; if ingested, wash mouth with water and give small quantities of water to drink; do not induce vomiting; if preservative contacts skin or eyes, wash thoroughly with water; get medical attention if symptoms occur