zenbed

coolmask

User guide

EN TABLE OF CONTENTS

| | |
|----------------------|---|
| instructions | 4 |
| Care and maintenance | 4 |
| Security warning | 4 |

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How to use your Cool Mask

Place the mask in the freezer for 60-120 minutes in the provided freezer bag or use your own. If the mask is too cold when you remove it from the freezer, you can let it sit at room temperature until it reaches the desired temperature. Once the mask has reached the desired temperature, place it on your head. If your headache, swelling, or pain worsens or persists, contact your own doctor and/or seek medical assistance.

CARE AND MAINTENANCE

Washing Instructions - Wipe the mask with a damp towel or cloth. Do not submerge the mask or cover it with water. If the product gets wet, simply allow it to air dry.

Drying instructions - If the product gets wet, let it air dry in the shade.

SECURITY WARNING

Always thoroughly check the interior lining and cover for ice crystals, tears, and holes before use. Do not use the mask if ice crystals or gel are visible.

Use the mask only as directed.

Do not ingest.

Apply only to dry skin.

Do not apply the product if it is too cold to touch or wet.

Do not apply to infected areas or open wounds without consulting a doctor.

Do not put in the dryer or washing machine.

Do not submerge in water.

Always freeze in a freezer bag.

If the gel comes into contact with the skin or eyes for any reason, immediately wash the area with water and soap. If an allergic reaction occurs, thoroughly cleanse the area with water and soap and contact your doctor immediately.

If you need assistance or have questions about your order, please contact service@zenbed.eu

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