

OP1 / 2-4C4TCF
OP3 / 4-4C4TCF

Offset Plates

Product Information

OP1/2-4C4TCF



OP3/4-4C4TCF

Description:

The offset plate displaces a bottom-loaded connection component $\frac{3}{4}$ inch (19 mm) for OP3/4-4C4TCF or $\frac{1}{2}$ inch (12 mm) for the OP1/2-4C4TCF. The offset plate comes with bolts to connect it to a pyramid or receiver and is manufactured in carbon fiber.

Instructions

Simply attach the offset plate to the outside distal end of a socket by bolting to a Prosthetic Design lock. Then bolt the connecting component to the offset plate. Use Loctite 242 and torque bolts to 12 Nm (9 ft lb).

Specifications

OP1/2-4C4TCF

Added Height: 6.5mm (.25")
Weight: 36g (1.26 oz)
Load Rating:
Activity Levels K1-K2 325 lbs. (147 Kg)
Activity Levels K3-K4 265 lbs. (120 Kg)

OP3/4-4C4TCF

Added Height: 6.5mm (.25")
Weight: 40g (1.41 oz)
Load Rating:
Activity Levels K1-K2 325 lbs. (147 Kg)
Activity Levels K3-K4 265 lbs. (120 Kg)

Suggested L-Code: L5910 – BK / L5920 – AK

Warranty Information: PDI warrants its components to purchasers of the prosthetic components only. No warranties are made to consumers. All components are warranted for 30 days from original purchase date. This warranty applies only to manufacturing defects and does not cover: (1) any damage due to misuse, altering of the component, improper service or accidents; (2) any secondary costs that might be incurred in the exchange of components; (3) wear; normal, or due to incorrect selection of components. During the warranty period, defective parts will be replaced or repaired at no charge. Missing bolts will be replaced at retail prices. This warranty is in lieu of all other warranties, expressed or implied, including any implied warranty of merchantability or fitness for particular purpose. The remedies provided above are exclusive; in no event shall Prosthetic Design, Inc. be liable for any special, indirect, incidental, or consequential damages.

Returns: A restocking fee totaling 2% of retail price will be charged to customer should the customer wish to return any PDI product for credit. The part must be unused, and in its original, unopened packaging. In addition, the return must be initiated within 30 days of its purchase in order to qualify for the credit. If returned after 30 days of purchase date a 15% restocking fee will be charged.



Prosthetic Design, Inc., 700 Harco Drive, Clayton, OH 45315 USA www.prostheticdesign.com
Telephone 800-459-0177 International Telephone 937-836-1464 Fax 937-832-5361

