



miniTREAD

powered by onthemuv®

*Congratulations on purchasing your new Onthemuv® miniTREAD®!!
Please be sure to read this entire Owner's Manual as it contains pertinent
information about service, care, and safety instructions about the product.*



Onthemuv.com
support@Onthemuv.com
310.873.3133

Owner's Manual

TABLE OF CONTENTS

- Before You Begin**4
- Safety Instructions**..... 4-5
- Equipment Warning/Caution Labels**6
- Equipment Specifications**6
- Equipment and Safety Warnings**6
 - Emergency Stop.....6
 - Supervised Use.....6
- About Your Onthemuv® miniTREAD®**7
- Key Component List.....7
- Assembly Instructions7
- Getting Started**.....8
- Operating your miniTREAD®8
- How to sit while using the miniTREAD®8
- Adjusting the Seat Height.....8
- Incorrect and Correct Posture while using the miniTREAD®9
- Using the Chair Wheel Stoppers9
- How to use your Remote Control.....10
- Setting the Remote Control.....10
- How to Adjust your Speed10
 - Start.....10
 - Speed Control10
 - Stop.....10
- Incline Adjustment.....11
- How to use the Incline Mechanism11
- Emergency Stop/Reset Button.....11
- Care and Storage**11
 - miniTREAD® Maintenance.....11
 - Carrying your miniTREAD®11
- MUV® Towards Fitness miniTREAD® Program**12
 - Current Exercisers.....12
 - New Exercisers.....12
- Parts List**13
- Ordering Replacement Parts**13
- Customer Service**13
- Warranty & Disclaimer** 14-15

BEFORE YOU BEGIN

Welcome to Onthemuv®! Purchasing your miniTREAD® is a step towards health!

Before you begin, it's important that you familiarize yourself with this important information on safety and use of the miniTREAD® so you and your loved ones do not get hurt.

For an audio version of the Owner's Manual, please refer to our Onthemuv YouTube Channel. For a large print version of the Owner's Manual, please go to www.Onthemuv.com.

SAFETY INSTRUCTIONS

Please adhere to the following safety guidelines and cautions:

- 1. WARNING:** Before beginning any exercise program, you should consult with your doctor. It is recommended that you undergo a complete physical examination.
- 2. DANGER: NEVER STAND ON THE miniTREAD®! Use ONLY while safely seated.** Failure to follow all warnings may result in serious injury or death.
- 3. WARNING: Read this Owner's Manual in its entirety prior to use** to reduce the risk of serious injury.
- 4. WARNING:** Connect this product to a properly grounded outlet only. See Grounding Instructions.
- 5. GROUNDING INSTRUCTIONS:** This product must be grounded. If it should malfunction or breakdown, grounding provides a path of least resistance for an electric current to reduce the risk of electric shock. This product is equipped with a cord having an equipment-grounding conductor and grounding plug. The plug must be plugged into an appropriate outlet that is properly installed and grounded in accordance with all local codes and ordinances.
- 6. DANGER:** Improper connection of the equipment-grounding conductor can result in a risk of electric shock. Check with a qualified electrician or serviceman if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product — if it will not fit the outlet, have a proper outlet installed by a qualified electrician. Make sure the product is connected to an outlet having the same configuration as the plug. No adapter should ever be used with this product.
- 7.** Operate the miniTREAD® on a solid, level surface.
- 8.** Use the miniTREAD® ONLY for the intended use as described in this Owner's Manual. *Do not* modify the equipment or use attachments not recommended by the manufacturer. *Do not* use any accessories that aren't specifically recommended by the manufacturer, these might cause injuries or cause the unit to fail.
- 9.** When using the miniTREAD® always sit with the knees below the hips. Please refer to the posture guide on pages 8 and 9. If you experience any pain or discomfort, please consult your physician prior to continuing use.
- 10.** Work within your recommended exercise level, do NOT work to exhaustion. If you feel any pain or abnormal sensation, STOP YOUR WORKOUT and consult your physician immediately. Begin exercising carefully to avoid injury. If you feel any pain, STOP! Pain is NOT gain!

4 Walk while seated!

- 11.** Warm-Up and Cool Down walk: If you're exercising for more than 5-minute increments at a time, start slowly for the first minute or two before you increase the speed. Then repeat the same process when you've finished. Get familiar with the miniTREAD®. Start out slowly and refer to our MUV® TOWARDS FITNESS miniTREAD® program for recommended guidelines.
- 12.** The miniTREAD® equipment is not intended for use by young children or pets. Please keep children under the age of 13 and pets away from the miniTREAD® when it is in use. Close supervision is necessary when this product is used near children or persons lacking understanding or capacity to use it safely and independently.
- 13.** Children should be supervised to ensure that they do not play with the product.
- 14.** Indoor use only. Do not use outdoors.
- 15.** Make sure there is adequate space available around the miniTREAD®.
- 16.** Do not operate where aerosol (spray) products are being used or where oxygen is being administered.
- 17.** STOP the miniTREAD® before raising or lowering the incline.
- 18.** Do not carry the miniTREAD® by the supply cord or use cord as a handle.
- 19.** Make sure the cords are safely and securely placed away from the feet to avoid tripping.
- 20.** Do not place any object on the moving tread and keep clothing free and clear of all moving parts. Never drop or insert any object into any opening.
- 21.** The miniTREAD® is designed for use wearing shoes or slippers. Do not wear loose pants over ankles, ankle jewelry or high heels while using the miniTREAD®.
- 22.** Excessive heating can occur and cause fire, electric shock, or injury to persons. Do not operate under a blanket or pillow.
- 23.** CONSUMER USE ONLY: The Onthemuv® miniTREAD® though compact and portable, is built for consumer use only and can withstand up to 80 pounds of leg load. Maximum user weight: 300 lbs.
- 24.** To reduce the risk of burns, fire, electric shock, or injury to persons: The miniTREAD® should never be left unattended when plugged in.
- 25.** To disconnect, turn all controls to the off position, then remove plug from outlet.
- 26.** Always unplug the Electrical Cord when the miniTREAD® is not in use, before cleaning and before taking any parts off or putting any part on the miniTREAD®.
- 27.** Never operate the miniTREAD® if it has a damaged cord or plug, if it is not working properly or if it has been dropped or damaged. Return the miniTREAD® to a service center for examination and repair.
- 28.** To prevent shock, keep all electric components, such as the motor, cord, and switch away from water.
- 29.** Call Customer Service at 310-873-3133 for replacement label, manual, video or questions.

EQUIPMENT WARNING/CAUTION LABELS

See below for placement of the following Warning/Caution labels on your unit.

WARNING LABEL 1 1 location





WARNING LABEL 2 1 location



CAUTION LABEL 1 4 locations



WARNING LABEL 3 1 location

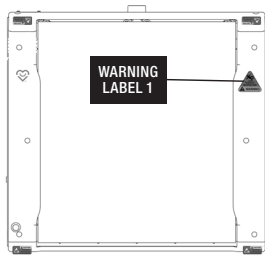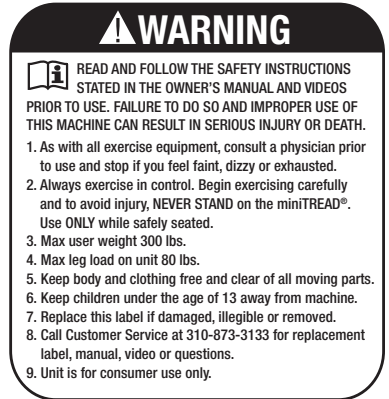

TOP VIEW



FRONT VIEW

BACK VIEW

EQUIPMENT SPECIFICATIONS

APPROXIMATE

Length ... 18.82" (47.8 cm)

Width 19.84" (50.4 cm)

Height5.51" (14 cm)

Weight 23.6 lbs. (10.7 kg.)

Maximum Weight Capacity of user: 300 lbs. (136 kg.)

Maximum Leg Load Weight on unit: 80 lbs. (36 kg.)

EQUIPMENT AND SAFETY WARNINGS

Emergency Stop

The **RED** Emergency Stop Button on the bottom left of the equipment will immediately halt the miniTREAD® equipment if pressed. To reset after use, wait 30 seconds and press the **GREEN** Start Button twice to restart the unit. Alternatively, turn the Power Switch OFF for a few seconds and then back on.

Supervised Use

If your medical provider has not cleared you for independent use of the miniTREAD® equipment, have balance issues or muscle weakness, please make sure you are supervised by a caregiver while using the miniTREAD®. This is for your own safety. Again, **NEVER STAND on the miniTREAD®**. Use while safely seated in a properly positioned chair.

6 *Walk while seated!*

ABOUT YOUR Onthemuv® miniTREAD®

The Onthemuv® miniTREAD® is used from a seated position and the motion can best be described as a hybrid of a miniature Treadmill and a Bike, so your calves, hamstrings, quads and core are all engaged. The motorized movement helps keep you motivated for continued use and movement becomes automatic — almost like being on autopilot.

The miniTREAD® calculates Steps, Distance, Calories, Speed and Time Spent. The Onthemuv® miniTREAD® weighs just 23.6 lbs. and can be easily transported to your workplace or in your home environment.

Key Component List - Standard Parts Included:

Pre-assembled Onthemuv® miniTREAD®



Power Cord, which plugs into any standard electrical outlet.



Remote Control



Remote Control Cord



#5 Allen Wrench (for adjusting the Belt when/if necessary).



Leveling Feet (located on the back of unit on both sides)

Assembly Instructions

We've made it easy for you! Minimal assembly is required.

Motorized miniTREAD®:

1. Remove the miniTREAD®, Power Cord, Remote Control, Remote Control Cord and #5 Allen Wrench from the box.
2. Place the miniTREAD®, with the belt side up and the on/off switch on the left side, on a level, horizontal and firm surface, for example, in front of your chair in your TV room or under your desk or kitchen table.
3. It is important that the miniTREAD® is kept on a solid, level surface. Please use the Leveling Feet and adjust the miniTREAD® to accommodate for minor variations. If the unit is not on a solid, level surface, user may need to adjust the Belt periodically as it may lean to one side or the other. Once balanced, periodically check and continue to use the Leveling Feet to ensure balance is maintained. If you find the Belt moves to one side, please refer to our: [Onthemuv YouTube Channel](#), for video instructions on Belt Adjustment.
4. Plug in the Power Cord to the rear of the unit and to the wall into any standard outlet. The Power Cord will work on both 110V/220V.
5. Plug in the Remote Control Cord to both the Remote and the miniTREAD®. You may plug in the Remote Cord to the Remote and the miniTREAD® using either side of the Cord. The Remote Plug is located on the left side panel of the unit.

GETTING STARTED

There are several adjustments you will need to make prior to using the miniTREAD®:

1. Adjust the height of your Seat for optimal leg movement and comfort.
2. Adjust the Distance of the miniTREAD® from your chair to be comfortable.
3. Increase or decrease the Speed of the miniTREAD® Belt using the Remote Control.

Operating your miniTREAD®

1. Turn ON the Power Switch (located at the back left side) by pushing the Button on the left side of the miniTREAD®. The Power Switch will light up when on.
2. Press the **GREEN** 'O' Button to turn on the Remote.
3. Use the **BLUE** Up and Down Arrows to increase or decrease the Speed of the miniTREAD®.
4. Use the **RED** 'X' Button to stop the miniTREAD® from operating.
5. Turn off the Power Switch when done with your session.



How to sit while using the miniTREAD®

Find your correct sitting height, then adjust the distance of the miniTREAD® from your seat.

Adjusting the Seat Height


Step 1: Sit in a chair, with the miniTREAD® on the floor in front of you, your feet in the center of the miniTREAD® Belt. You can sit forward and upright on the chair or, with your back supported against the chair back.

Step 2: If you are sitting at the correct height, your knees will be slightly **BELOW** your hip flexors.



If your hips are lower than your knees, your chair is too low. It may be difficult to extend your legs and/or may feel unstable. If you have an adjustable chair, raise the chair seat to create the correct hip flexor to knee alignment as shown in Step 2. If your chair is not adjustable, use a seat cushion to adjust your height.

One goal of the miniTREAD® equipment is to engage the Core (abdomen) muscles. This can be accomplished by sitting up straight. If you need a seat cushion to help support your back, please refer to our website.

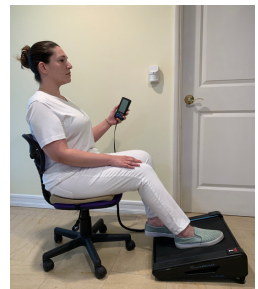

Incorrect and Correct Posture when using the miniTREAD®



**INCORRECT
POSTURE**



**CORRECT
POSTURE**

Using the Chair Wheel Stoppers

If you are using the miniTREAD® on a chair with wheels, your chair may move. We offer Wheel Stoppers as an accessory, available on our website.


Place the Chair Wheel Stoppers under the front most wheels on your chair. Position the Wheels so they are to the right and left of the miniTREAD® — a wheel should not be centered, or your feet may hit it during use.



Chair Wheel Stoppers
(optional accessory)



How to use the Remote Control

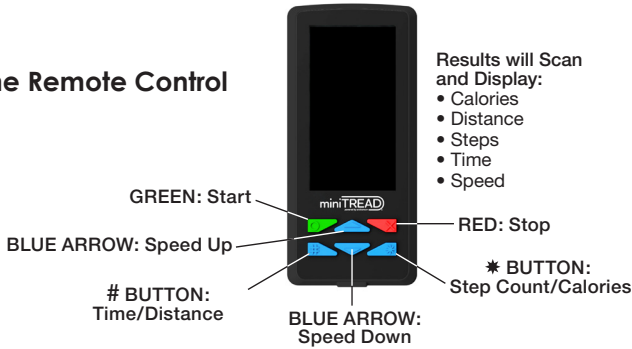

Setting the Remote Control

The Remote can be set for mph/miles or kph/km, (with mph/miles being the default setting). User may switch to kph/km by pressing the **GREEN and RED Button** simultaneously BEFORE pressing the **GREEN GO Button**. This preference may only be set when the miniTREAD® is stationary, prior to starting the Remote.

How to Adjust your Speed

Start

Once you have your seat height adjusted, use the Remote, **press the GREEN Start Button** turned on the miniTREAD® Belt to the first increment of .3 mph.

Speed Control

Increase or decrease the Speed of the miniTREAD® using the Remote. Begin to 'walk' placing one foot in front of the other. **Press the BLUE 'up arrow'** to incrementally increase the Speed and the **BLUE 'down arrow'** to decrease the Speed. Make sure you can comfortably 'walk' with your feet out in front of you. The miniTREAD® should be a minimum of 6" in front of the chair. Using the miniTREAD® should be comfortable.

- Feet should not kick under the chair or behind your knees while in use.
- When legs are outstretched the knees should not be fully extended.

The Belt Speed ranges from .3 mph, which is very slow, up to 3 mph. It's important to slowly increase Speed and wait a minimum of 10 seconds between each increase until you have found your comfortable Speed. If you find the miniTREAD® is moving too quickly, **press the BLUE 'down arrow'** immediately to reduce the Speed or the **RED arrow to STOP**. If needed, you can place your feet on either side of the miniTREAD® or on the floor until it is at your desired Speed.

The Remote function enables you to personalize your walk every time you use the miniTREAD®. As you get stronger and fitter, you can continue to use the Remote to adjust the Speed with an easy click of a Button!

Stop

Press the RED STOP Button on the Remote to STOP the unit and make any appropriate adjustments. The **GREEN START Button** will restart the unit, and your results will be saved. To clear the results and reset, press the **RED STOP Button** twice.

What should your miniTREAD® walk feel like?

- The walking pattern should be comfortable.
- Using the miniTREAD® should feel like you are walking comfortably...while seated.
- You should **NOT** feel any pain in your hips, knees, ankles or your back. If you feel any discomfort, adjust the placement of the miniTREAD®, the chair or the Posture Cushion(s), or discontinue use.

Incline Adjustment


⚠ WARNING

STOP the miniTREAD® BEFORE OPENING OR CLOSING THE INCLINE.

The natural incline of the miniTREAD® equipment is at approximately 8 degrees. A user may increase this incline to 14 degrees by releasing the **BLUE Incline Latch** on the front of the unit. Once released, gently pull up on the unit to snap it into place and then operate the miniTREAD® as indicated. **To CLOSE the Incline, pull up on the BLUE Incline Latch** once again, place one hand on each side of the unit and gently **press down** until it snaps into place. Occasionally, this may be a little more difficult on carpet. Should you experience difficulty when closing the **BLUE Incline Latch** on carpet, simply place a ruler or a book under the sides of the unit and try again. Make sure both sides of the unit have snapped closed prior to starting it back up again.

How to use the Incline Mechanism

- To Open:** Release Incline Latch to lift the miniTREAD® until it snaps into place.
- To Close:** Release Incline Latch and push down on the miniTREAD® until it snaps into place.
- miniTREAD® must be on a hard surface to close.

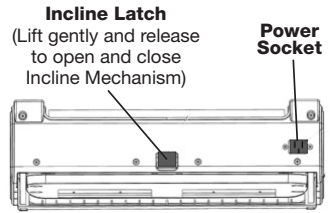

miniTREAD® Front View

Emergency Stop/Reset Button

The **RED Emergency Stop/Reset Button** on your miniTREAD® will be triggered if an excessive amount of weight is sensed on the unit or if a user presses it as an Emergency Stop. The Remote will display an error message “EO”. To reset the unit, press the **RED Emergency Stop/Reset Button** once again, wait 30 seconds and **press the GREEN Start Button** on the Remote twice. Alternatively, the user may turn OFF the Power Switch, wait a few seconds, turn the miniTREAD® back ON and begin using.

CARE AND STORAGE


Your Onthemuv® miniTREAD® has been designed to require minimum maintenance. To ensure optimal function, we recommend the following:

miniTREAD® Maintenance

- Keep the miniTREAD® clean!** For optimum performance, it's important to avoid dust and dirt from accumulating between the miniTREAD® and the Belt. If dirt accumulates, the miniTREAD® Belt may not operate as smoothly.
- Please keep the unit covered (a small towel should do the trick) and stored in a dry area.** Store after use. It's a good idea, though not necessary, to store the miniTREAD® unplugged and away from children and high traffic areas.
- Safely store your miniTREAD® with Cords out of reach.**
- Store away from humidity. Do not keep near water.**
- Always unplug your miniTREAD® when not in use.**

Carrying your miniTREAD®

The miniTREAD® is equipped with a Recessed Carrying Handle on both sides of the unit for portability.

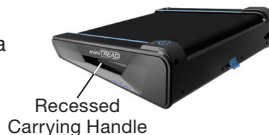

MUV® TOWARDS FITNESS miniTREAD® PROGRAM

Current Exercisers

If you have been regularly exercising, you can use the miniTREAD® any time you want. We suggest to get used to how it feels and functions, that you start with 5-15 minute blocks (or less) and increase your time by 5 minutes until you work up to the time you desire.

New Exercisers

If you're just starting out, or re-starting to move again after 6 months or more, you want to ease into using the miniTREAD® the first couple of weeks. There is a slight learning curve to walking while seated, which challenges your muscles differently than walking while standing. We suggest the following, adjusting Speed as needed each session:

Day 1: Walk for 1 minute at a comfortable pace; Start slowly depending upon your doctor's recommendation and your comfort level.

If you've been inactive for six months or more, you may want to start with our "Welcome Back to Fitness" sample program:


Day	Walking Time	# Times per day	Total walking time per day
1	1 minute	Once	1 minute
2	1 minute	Twice	2 minutes
3	1 minute	Three	3 minutes
4	1 minute	Four	4 minutes
5	1 minute	Five	5 minutes

Once you can walk 5 minutes in a day, begin to add time to your daily walk in 1- minute increments until you are walking for 10-15 minutes consecutively; the pace you set is up to you, and always adjustable, using the Remote to increase or decrease your Speed. You can also break it up as it works best for your schedule. Remember, you can use the miniTREAD® when you watch TV, working while seated at your desk or during a designated exercise time.

Regardless of your fitness acumen, be sure to go through the steps of adjusting your seat height and distance of the miniTREAD® from your chair each time you use it.

PARTS LIST

ITEM #	PART#	DESCRIPTION	QTY
1	20-MMT1	Onthemuv® miniTREAD®	1
2	20-MMTPC	miniTREAD® Power Cord	1
3	20-MMTRC	Remote Control	1
4	20-MMT-RCC-6	Remote Control Cord	1
5	5-Allen Wrench	#5 Allen Wrench	1
6	20-MMT-BELT	miniTREAD® Belt	1
7	20-MMT1-OM	Onthemuv® miniTREAD® Owner's Manual	1

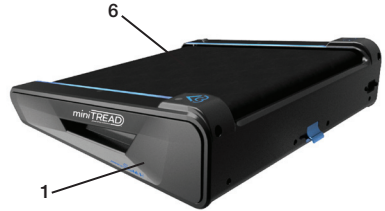

ORDERING REPLACEMENT PARTS

To order replacement parts, email us at support@Onthemuv.com.

When ordering replacement parts, please contact our Customer Service Department, at 310.873.3133, Monday through Friday - 10:00 am to 4:00 pm, PT.

IMPORTANT: You must have your Serial Number and this Owner's Manual ready when calling for parts.

Serial #: _____

PLEASE ALSO PROVIDE THE FOLLOWING INFORMATION:

1. Name, Mailing Address and Telephone Number
2. Date of Purchase
3. Where Product was Purchased
4. Model Number
5. Part Order Number and Description

CUSTOMER SERVICE

How may we help you? If you have any questions regarding your Onthemuv® miniTREAD®, please call our Customer Service Specialists at 310.873.3133 or email us at: support@Onthemuv.com.

We pride ourselves in excellent customer service. We are committed to offering top quality merchandise with a 100% money back guarantee if you are not completely satisfied within 30 days of date of purchase.

WARRANTY & DISCLAIMER

Warranty is good and available to the original purchaser only as noted by Onthemuv® invoice and is not transferable nor assumable. Onthemuv® cannot warrant products that have been abused, neglected and or poorly maintained. All equipment should be reviewed at the time of delivery for damage, breakage, loosening of nuts and bolts, components and or any other moving parts. Any and all claims for warranty must be received in writing within 60 days of defect. Do not alter, modify or redesign any Onthemuv® products or use any replacement parts or materials other than those components original to Onthemuv® as it will limit the warranty and liability of Onthemuv®. Onthemuv® reserves the right to change, replace or modify products, parts, accessories and or design without notice. Deposits on cancelled orders will be subject to handling and restocking charges.

WHAT IS COVERED

Onthemuv® Inc. warrants that under normal, personal, residential use, the Onthemuv® miniTREAD® is free from defects in material or workmanship. Parts are warranted for one (1) year from the date of purchase.

WHO IS COVERED

The original purchaser or any person receiving the Product as a gift from the original purchaser.

WHAT WE WILL DO TO CORRECT COVERED DEFECTS

Onthemuv® Inc. obligation under this warranty is limited to replacing the defective part(s); or at its option, repairing or replacing (or refund in its sole discretion) the product.

We will ship to you any new or rebuilt replacement part or component, or, at our option, replace the Product. Such replacement parts are warranted for the remaining portion of the original warranty period.

WHAT IS NOT COVERED

Any failures or damage caused by unauthorized service, misuse, negligence, accidents, improper or unauthorized alterations, installation or modification, debris resulting from any construction activities in the Product's environment, rust or corrosion as a result of the Product(s) location, failure to follow manufacturer's direction for use or storage, improper maintenance, vandalism, usage for commercial or rental purposes, or "ordinary wear and tear", or by failure on your part to use, operate and maintain the Product as set out in your Owner's Manual. This warranty does not cover any separately sold items, parts, or accessories.

OWNER'S MANUAL

It is VERY IMPORTANT THAT YOU READ THIS OWNER'S MANUAL before operating the Product. Remember to perform the periodic maintenance requirements as specified in the Owner's Manual to assure proper operation and your continued satisfaction.

HOW TO GET PARTS & SERVICE

Customer Support Services: Support@Onthemuv.com.

For service call 310.873.3133. Monday through Friday from 10:00 am to 4:00 pm, Pacific Standard Time, and provide your name, address and the model number of your Product. Customer Support will tell you how to get a replacement part, or, should you have the Extended Warranty advise you on how and where to ship the Product for service.

CLAIM PROCEDURES

Claims for defective merchandise must be made within ONE year from invoice date. Claims for missing parts must be made within 60 calendar days after the merchandise is received. Any claim for defective merchandise returns must be packed in original packaging. We reserve the right to specify that items be returned to the original warehouse for inspection or be inspected by our representative in the field. Pictures are required to claim defective merchandise, along with a copy of the original invoice. If the claim is justified, the item(s) or part(s) will be repaired or replaced or a credit will be issued. It is our policy to replace parts whenever possible.

Before shipping:

1. Obtain a serial number from the product serial label.
2. Securely pack your Product (use the original shipping carton, if possible).
3. Write the serial number on the outside of the carton.
4. Insure the Product, and
5. Include a letter explaining the defect or problem and a copy of your proof of purchase if you believe the service is covered by warranty.

There are no other warranties other than expressly set forth herein. **Any implied warranty of merchantability or fitness for a particular purpose is limited to the duration of this warranty.** No agent or retailer of the Onthemuv® miniTREAD® has authority to modify this warranty in any respect.

Onthemuv® SHALL NOT BE HELD LIABLE FOR INDIRECT OR CONSEQUENTIAL DAMAGES ARISING OUT OF USE OR PERFORMANCE OF THIS PRODUCT. These include – but are not limited to – loss of use of the product; loss of time, enjoyment, or revenue; inconvenience; installation or removal costs; or other incidental or consequential damages.

EXCLUSIVE WARRANTY

THIS LIMITED WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES OF ANY KIND EITHER EXPRESSED OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND ALL OTHER OBLIGATIONS OR LIABILITIES ON OUR PART. We neither assume nor authorize any person to assure for us any other obligation or liability concerning the sale of this Product. Under no circumstances shall we be liable under this warranty, or otherwise, of any damage to any person or property, including any lost profits or lost savings, for any special, indirect, secondary, incidental or consequential damages of any nature arising out of the use of or inability to use this Product.

EFFECT OF STATE LAWS

This warranty gives you specific legal rights and you may have other rights, which vary, from state to state. This warranty is valid in the U.S. and Canada only. Some states do not allow the exclusion or limitation of implied warranties or of liability for incidental or consequential damages, so the above limitations or exclusions may not apply to you.

CHANGES IN WARRANTY NOT AUTHORIZED

No one is authorized to change, modify or extend the terms of this limited warranty.

EXTENDED WARRANTY

You may purchase the Extended Warranty for your motorized miniTREAD® or self-powered Sitmill® equipment within 2 weeks of purchase. Refer to our website for additional information, or return to the original point of purchase with the original receipts and they will assist you in purchasing your Extended Warranty.

OUR PLEDGE TO YOU

Our Products are designed and manufactured to the highest standards. Onthemuv® wants you completely satisfied with our Products and will do everything possible under the terms of this warranty to keep you secure in knowing you have bought the best!

Manufactured and Distributed by: Onthemuv® Inc., Los Angeles, California, USA

Warranty service address may differ and will be provided upon approved Return Authorization.

For warranty support, keep the original proof of purchase and record the following information:

Serial Number _____

Date of Purchase _____

Purchase Location _____

To register your Product Warranty, go to www.Onthemuv.com/register.



miniTREAD[®]
powered by onthemuv[®]

Walk while seated!

©2020 Onthemuv[®] Inc. All rights reserved.

Onthemuv[®], MUV[®], Sitmill[®] and miniTREAD[®] are registered trademarks of Onthemuv[®] Inc.

Patents: www.Onthemuv.com/patents

No part of this booklet may be reproduced in any form, by any means,
electronic, mechanical or otherwise, without the express written
consent of the copyright holder.

20-MMT1-OM