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Hallufix  
INNOVATIVE ORTHOTIC SOLUTIONS

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Hallufix® products  
the perfect choice for  
Hallux valgus treatment

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
Tel. +49 (089) 96 05 779 - 0  
Fax +49 (089) 96 05 779 - 66

info@hallufix.de  
www.hallufix.com

(04.21/20.000)  
Hallufix® is a registered trademark.


### Toe Spreader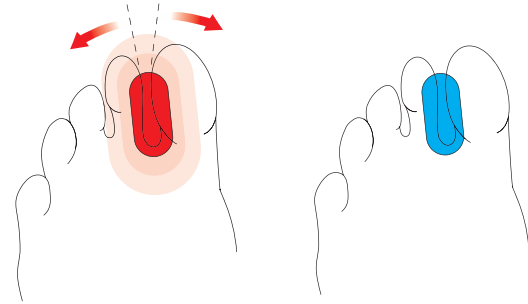

**Indication:**  
Pressure pain and  
chafing caused by toes  
that are close together or  
on top of each other.

**Solution:**  
The Toe Spreader  
separates the big toe  
from the 2nd toe.


**Recommended use:** Spreading of tight and displaced  
toes; Hallux valgus (bunion)  
**Mode of action:** Hallufix® Softies Toe Spreader gently  
spreads toes to prevent pressure marks, chafing and  
skin irritations. The soft gel gently creates space  
between the toes providing maximum comfort while  
walking.  
**Package content:** 2 Toe Spreader (see bottom of  
package for sizing)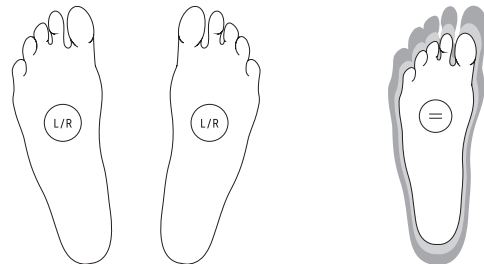

**Toe Spreader**  
can be worn left or right

**Toe Spreader**  
are available in two sizes


### Hammer Toe Crests

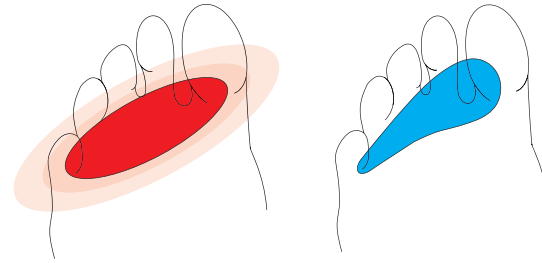**Indication:**  
Pressure points and  
chafing caused by toe  
misalignment.

**Solution:**  
The toes are stretched  
by the cushion.


**Recommended use:** Hammer and claw toes  
**Mode of action:** Hallufix® Softies Hammer Toe Crests  
relieve and ease pressure on toe tips, caused by  
hammer or claw toes. The toe crests bring the toes in a  
stretched position. Wearing in shoes they avoid pres-  
sure marks, corns, calluses, blisters and skin irritation  
on the toe tips. The soft gel is skin friendly and allows  
optimal adjustment to the foot.  
**Package content:** 2 Hammer Toe Crests (see bottom of  
package for sizing)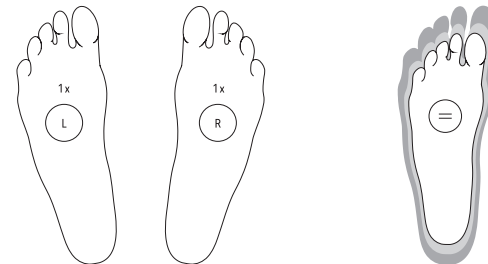

**Hammer Toe Crests**  
(1x left + 1x right)

**Hammer Toe Crests**  
are available in two sizes


### Bunion Protection PLUS

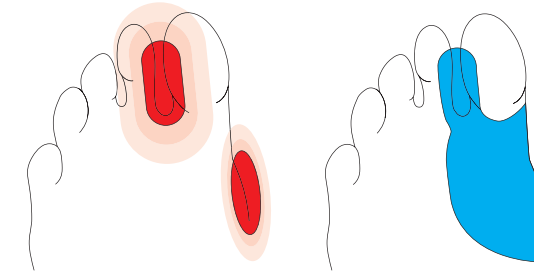**Indication:**  
Bunion pain and chafing  
between toes.

**Solution:**  
The Bunion Protection  
PLUS protects the bunion  
and separates the big toe  
from the second toe.

**Recommended use:** Hallux valgus (bunion)  
**Mode of action:** Hallufix® Softies Bunion Protection  
PLUS protects twice. It separates the big toe from the  
second toe and protects the sensitive bunion while  
wearing shoes. Hallufix® Softies Bunion Protection  
PLUS prevent pressure marks, chafing and skin  
irritations. The soft gel gently creates space between  
the toes and molds the contour of the bunion providing  
maximum comfort while walking.  
**Package content:** 1 Hallufix® Bunion Protection PLUS,  
one size fits all

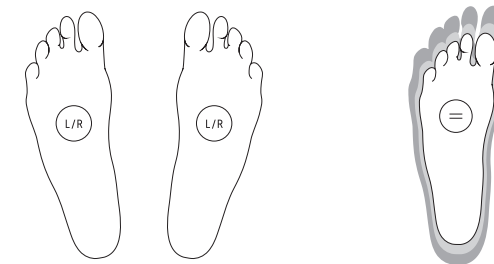**Bunion Protection PLUS**  
can be worn left or right

**Bunion Protection PLUS**  
is suitable for all foot sizes.

## General Information on the Hallufix® Softies Products

**Material:**  
The TPE gel used in medical applications is particularly  
suited to use in the orthopaedic field. Flexibility, optimum  
reset function, slip-resistant, shock absorbing and  
hygienic, are characteristic attributes of the Hallufix®  
Softies Products.

**Cleaning:** Hand clean with water + soap.

### Contraindications:

- There is no known hypersensitivity or allergic reaction  
associated with the products when used correctly.
- Do not use on unprotected wounds or abscesses.

**Disposal:** The product must be disposed of in  
accordance with the nationally applicable regulations.  
**Warranty:** Hallufix Softies are guaranteed for 3  
months from the date of sale when used properly.  
**Reusability:** The product is only intended for use on  
one person and must not be reused on another person.

You want to know more  
about Hallux valgus (bunion) ?  
Visit our website: [www.hallufix.com](http://www.hallufix.com)

# Hallufix

INNOVATIVE ORTHOTIC SOLUTIONS