

Q-Optics



TTL LOUPES - INSTRUCTIONS

Q-Optics

GET STARTED HERE



q-optics.com/welcome

For everything you need to get started with your Q-Optics gear including your headlight, follow the QR code or URL link for PDF manuals and instructional videos.

Thank you for making us your choice in loupes!



(972)298-2669 - local
(800)858-2121 - US / Canada



support@q-optics.com



INCLUDED IN YOUR KIT



INSIDE YOUR LOUPE CASE:

- Your custom TTL Loupes
- Loupe Strap
- Replacement Nosepads
- Cleaning Solution
- Microfiber Cleaning Cloth
- Side Shields (Classic and Panto frames)
- RX Insert (Eclipse frame - if purchased)

Replacement accessories are available online at:
q-optics.com/shop

If your purchase also includes a headlight, you will find additional headlight accessories in your loupe case. For full details regarding your headlight, please review your headlight system manual.

Your loupe case style may vary.

QUICK DISCONNECT ACCESSORIES

HEADLIGHTS



Align the headlight mount channel opening to the Q-Optics logo mount on the frame of the loupes.



Slide the headlight onto the frame until fully seated.

Remove your headlight prior to washing your loupes!

PRESCRIPTION INSERT (OPTIONAL)

The mount for the RX Insert is located on specific Eclipse frames above the fixed nosepiece and will align with the channel mount on the backside of the RX Insert itself.



Align the frame mount to the channel mount on the RX Insert and slide downward onto the screws. Insert should set behind nosepads.

Slide upward and off to remove.



Your RX insert can be removed for easier cleaning.

ADJUSTING YOUR FIT




CORRECT



INCORRECT

Frames should rest high on the nose.

Nosepad Adjustments

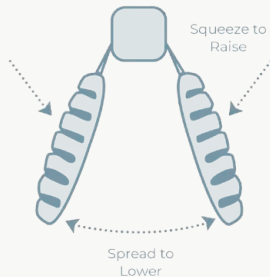

To raise loupes on your face, press nosepads closer together; to lower loupes, spread nosepads further apart.

Temple Arm Adjustments



Bend the temple arm "ear tips" for a more secure, comfortable fit.

You may also visit your local optometrist or eyewear shop for an adjustment.

USING YOUR ACCESSORIES

Nosepads

Both saddle style and individual nosepads are installed and removed in the same manner. Remove nosepad by sliding down the metal prongs on the frame. Mount the new pad(s) by aligning the hole in the nosepad to the metal prong, sliding pad upward over the ridges of the prong until fully seated.





Sport Straps

A strap is included to help keep your loupes in place while wearing. To attach the strap to your loupes, align the hole in the rubber tip of each end of the strap to one of the silicone tips of your loupe temple arms. You may need to wiggle the rubber end to fit over the temple arm.



Right and left side shields will be indicated in print on each shield.

Side Shields

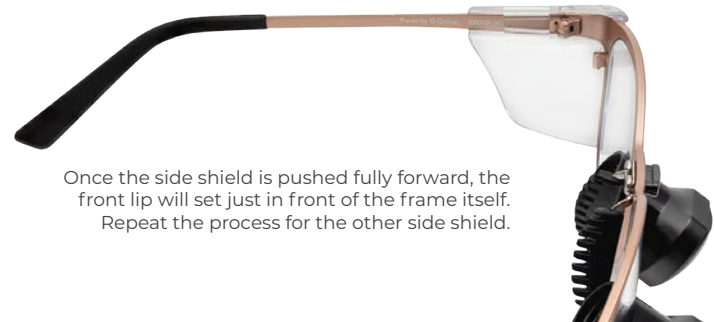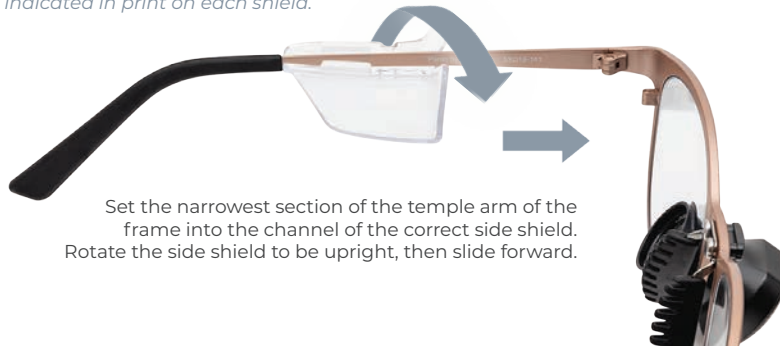

CLEANING YOUR LOUPES



RINSE

Step 1



Remove headlight from your loupes!

Rinse down your loupes with gentle soap and lukewarm running water.

DISINFECT

Step 2



Optional:

Disinfect your loupes using CaviCide and a microfiber cloth or CaviWipes (wipe gently)

WIPE

Step 3



Remove spots and streaks using the including Q-Optics lens cleaner spray and microfiber cloth. Allow to dry thoroughly.



Never soak your loupes or headlight, not even in water



Never place loupes or headlights in an ultrasonic cleaner



Use microfiber or cotton cloth to wipe (no paper products)

PROPER CARE AND MAINTENANCE

Proper storage and care over time will help maintain the optimal performance and fit of your loupes. Keeping them stored in the custom case that is provided and adhering to the guidelines provided in this guide will preserve the durability over many years of use.

WARRANTY

Q-Optics guarantees that your loupes are free from defects in material, construction, and design for the LIFETIME of your practice.

Headlights, as electronic devices, are covered under a separate, limited warranty.

To view the warranty in full, please visit:

www.q-optics.com/support/warranty/

Replacement parts can be purchased online at: www.q-optics.com/shop



Q-Optics / Quality Aspirators

1419 Godwin Lane · Duncanville, TX 75116

P: (972)298-2669 · F: (972)298-6592

www.q-optics.com

You may wish to keep these instructions for reference as well as the original packaging should you need to return your loupes for maintenance.

© 2023 Q-Optics. All rights reserved. Product specifications are subject to change at any time.
UI308-10003-02, ECN 2158