Where do I find the batch number on my lift chair?

You can find the batch number in two locations-one is on the back of this manual and the second is underneath the footrest of the chair. Please note that batch numbers must be provided in order for any replacement parts or service to be carried out.

WARNING

If you are uncertain about your chair and the way it is working, unplug it and contact seller for futher assistance.

WARRANTY

- 1. Electronic components (motor/transformer/controllers)-1 year.
- 2.Cover (fabric/PU/PVC/ combined fabric/leather/etc)-2 years.
- 3. Reclining mechanism-3 years.

Your new chair is covered by above warranty, please note that this does not include the man-made damage.

If man-made damage is found during the gurantee time, our company will answer for the maintenance and will charge maintenance costs generated by this (shipping and handling).


Batch number:



Thank you for choosing our product!

User Manual

ift chair series-9193

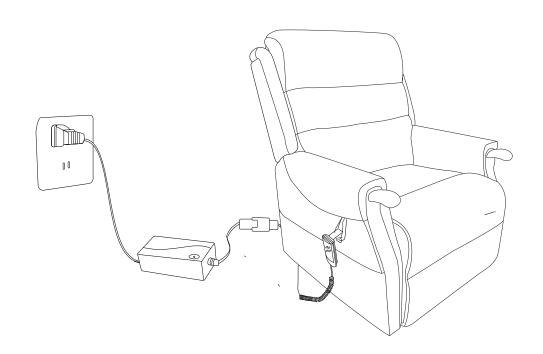

Dear valued customer,

We hope this chair brings convenience and comfort to your daily life. If you have any ideas on how we can improve the chair to provide a better user experience, please send them to us via email. Your suggestions will help us grow and produce more professional products that better serve our customers. Thank you for your help!

Welcome

Thank you for choosing our product!

We hope our product meets or exceeds your expectations. Should you have any questions, feel free to contact the seller including your contact information (name, phone number). Our customer representative will contact you soon.

We suggest that you keep this manual in a safe place. It contains essential information regarding the operation of your chair.

Contents

General Warnings	2
Specific Warnings	3
Parts	4
Assembly Steps	5/6/7
Remote control	7
Cleaning and Maintenance	8
Warning	8
Safe Seating Position	9
General trouble-shooting guide	10/11

General Troubleshooting Guide.

How can I ask for help when something goes wrong with my product?

Click contact link, send us an email, please include:

- 1. Batch Number.
- 2. Brief problem description.
- 3. Contact info. (phone number, email).
- 4. Video and pictures demonstrating the problem.

What if my lift chair does not work at all?

- 1. Make sure the outlet is turn on and has electricty.
- 2. Ensure the chair is plugged.
- 3. Make sure there are no cable damages.

My chair only partically working, what is the reason?


The unworking part may need to be replaced. (All the accessories can be replaced very easily, the seller will provide an online teaching video if needed.)

What if my chair or controller starts to overheat or smell like smoke?

Unplug the chair ,and discontinue use of the chair immediately.Contact seller for further assitance.

Noises

All motors hum and vibrate slightly. This is normal. There are many moving parts and you may hear noises when operating the chair. However, if the noise is excessive, contact a customer service representative.

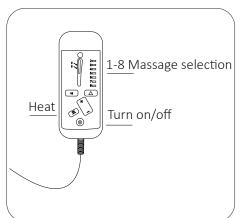



Specific Warning

People with heart disease should be cautious when using this massage function.

Safe Seating Position

The chair is a lifting device and it is important that you are positioned correctly in the chair when it is rising and lowering. If the correct body position is not maintained, there is a risk of injury or damage to the chair. The central position should be used either in the standing or sitting mode.

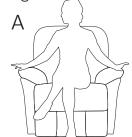


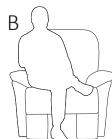




- -A:Correct position:Sit centrally with arms on the chair.
- -B:Incorrect position:The body is unstable and there is a possibility of injury.
- -C:Keep your feet on the floor as the chair is lifting.


Caution!

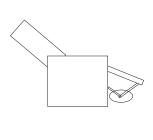

The rise/recline of the footrest is supported by a scissor mechanism. Do not leave children or pets unsupervised or directly around the chair. There is a potential risk of trapping by the mechanism. Take care to avoid trapping clothing, blankets, fingers etc.

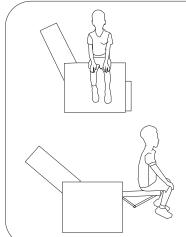

Do not sit on the armrests or footrest, the mechanism may be damaged by putting too much weight on it.

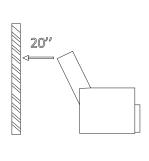


Make sure there is enough distance between the wall and the chair back BEFORE fully reclining. Reclinging the chair too close to the walk will damage the chair. We suggest you keep 20 inches between the wall and your chair.

9 -------

Specific Warnings

- -Maximum loading capacity 300 lbs!
- -Ensure that only one person sits in the chair at a time.
- -Do not route any cables under the chair or through any moving part of the chair. There is a risk of the cable being damaged which could cause an electric shock.
- -Do not use the chair with an electric blanket.
- -Do not dry wet areas with hair dryers, etc.
- -Do not over expose the leather to sunlight.
- -Do not leave the chair in the recline position.
- -Do not move the chair while the power cable is connected to the outlet.
- -Keep all electronics away from moisture and extreme temperatures.
- -If the chair becomes damaged, unplug and contact the seller for service.
- -Unplug the base before cleaning.
- -Avoid getting liquid on to the electrical system.
- -When the chair is in the raised position, do not sit down heavily onto it, but, lower yourself gently. Severe impact could damage the mechanism.
- -Ensure that the footrest does not catch on the carpet.
- -Be careful while using sharp objects such as needles, pins, scissors, etc.
- -Do not drag the chair when moving or lifting as this may cause damage to the lifting mechanism and trap the cables.
- -Do not leave the chair in the UP or DOWN position when not in use.
- -We do advise that the chair checked once every 12 months.
- -Only use authorized accessories for the chair.

Remote control: Before you operate the controller.

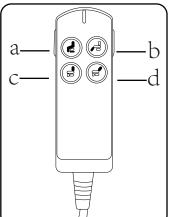

Ensure all the cables are clear of the mechanism. The chair isn't situated on top of any cables.

- a Lowers/Closes footrest and lifts chair.
- b Lifts/Opens footrest and lowers chair.
- c Lowers/Opens backrest.
- d Lifts/Closes backrest.

Cleaning and Maintenance


Cleaning-quick steps to clean the chair:

- 1. Please dust the chair frequently.
- 2. Wipe over using a damp and soft cloth. Let the chair dry thoroughly before use.
- 3. Use a vacuum equipped with a soft brush attachment.
- 4. Use eco-friendly soap(we highly recommend using cleaners and conditioners that are made specifically for leather, never use any type of harsh cleaners, especially alcohol-based products.)

WARNING:


- 1. Greasy spillages can be made worse when encountering water.
- 2. Do not over expose faux leather or true leather to sunlight.
- 3. Always unplug the chair before cleaning it. Do not reconnect it until you are sure it is completely dry. Never allow water to get into electronics.

Step 5. Connect the cords as shown below.

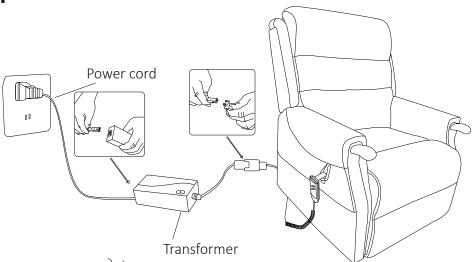

Connect the power cord, and transformer and extension cord as shown above.

(The cords and transformer are in the accessory package which is tied inside the chair base!)

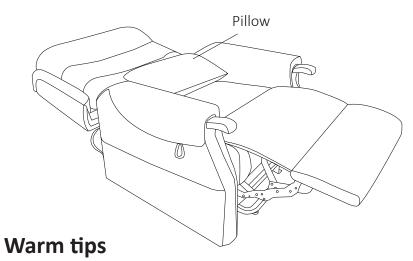

This pillow is recommended when lying down on this chair it will help to support your body and make you more comfortable during sleep.

Parts


The first box contains the following parts:

A: Accessory package




This package is inside the chair base and contains:

1 Motor Power Cord		2 AC Adapter	
3 Wrench	T	4 Two Castors	

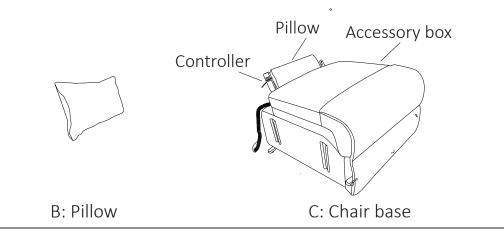


The second box contains the following parts:

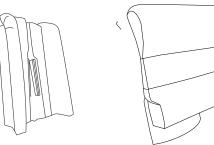






E: Chair back

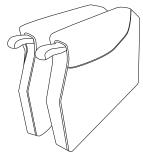

F.Left & Right arm

Assembly Steps Step 1. Fasten the castors on the rear of the chair.

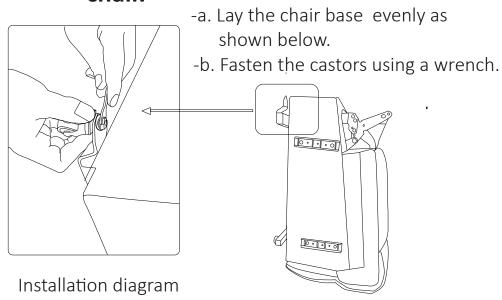


Step 2. Assemble the chair arms.




- -a.Level the chair base.
- -b. Align the locking clip.
- -c. Push the arm down evenly with both hands to the bottom.

Step 3. Assemble the chair back.

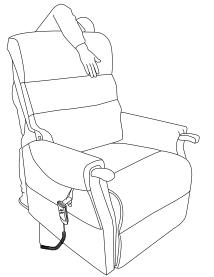


- -a. Align the locking clip.
- -b. Slide the backrest into the chair base.

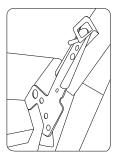

Installation diagram

Step 4. Assemble the chair ears.

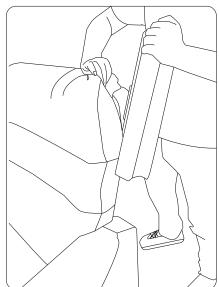

- -a. Align the locking clip.
- -b. Push the ear down evenly with two hands to bottom.

5 ------

6