


# User Manual Lift chair series

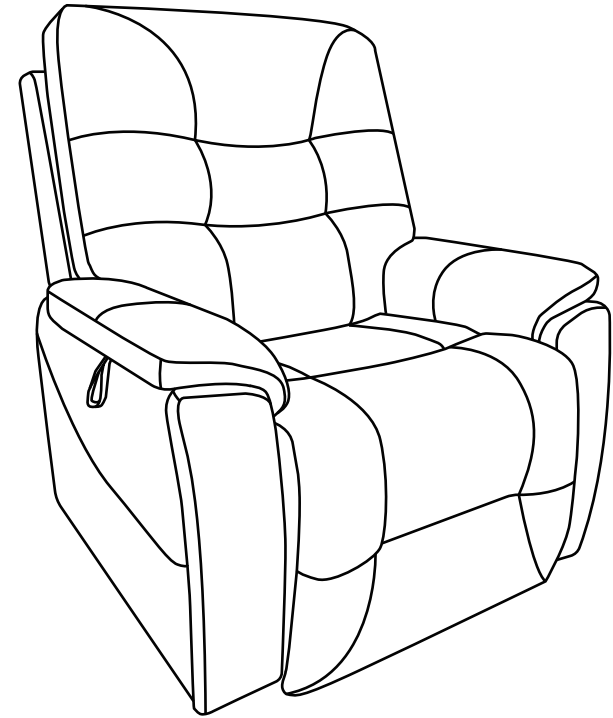

Note:  
Manual picture for reference only. Actual product might be different . However installation steps are the same.

## Welcome

Thank you for choosing our product!

We hope our product meets or exceeds your expectations. Should you have any questions, feel free to contact the seller including your contact information (name, phone number). Our customer representative will contact you soon.

We suggest that you keep this manual in a safe place. It contains essential information regarding the operation of your chair.

## Contents

--General Warnings	2
--Specific Warnings	3
--Parts	4
--Assembly Steps	5/6/7/8/9
--Remote Control	10
--Safe Seating Position	11
--Cleaning and Maintenance/Warm Tip	12
--General Trouble-shooting Guide	13/14
--Warranty	14

## Where do I find the batch number on my lift chair?

You can find the batch number in two locations-one is at the end of this manual and the second is underneath the footrest of the chair. Please note that batch numbers must be provided in order for any replacement parts or service to be carried out.

## WARNING

If you are uncertain about your chair and the way it is working, unplug it and contact seller for further assistance.

## WARRANTY

1. Electronic components (motor/transformer/controllers)-1 year.
2. Cover (fabric/PU/PVC/ combined fabric/leather/etc)-2 years.
3. Reclining mechanism-3 years.

## Noises

Your new chair is covered by above warranty, please note that this does not include the man-made damage.

If man-made damage is found during the guarantee time, our company will answer for the maintenance and will charge maintenance costs generated by this (shipping and handling).

Batch number:

Thank you for choosing our product!

## General Trouble-shooting Guide.

How can I ask for help when something goes wrong with my product?

Click contact link, send us an email, please include:

1. Batch Number.
2. Brief problem description.
3. Contact info. (phone number, email).
4. Video and pictures demonstrating the problem.

What if my lift chair does not work at all?

1. Make sure the outlet is turned on and has electricity.
2. Ensure the chair is plugged.
3. Make sure there are no cable damages.

My chair only partially working, what is the reason?

The unworking part may need to be replaced. (All the accessories can be replaced very easily, the seller will provide an online teaching video if needed.)

What if my chair or controller starts to overheat or smell like smoke?

Unplug the chair, and discontinue use of the chair immediately. Contact seller for further assistance.

### Noise



All motors hum and vibrate slightly. This is normal. There are many moving parts and you may hear noises when operating the chair. However, if the noise is excessive, contact a customer service representative.

## General warnings

- Indoor use only.
- Ensure that only one person uses this chair at a time.
- Do not dry wet areas with hair dryers, etc.
- Do not over expose the leather to sunlight.
- Do not leave the chair in the recline position.
- Take care to avoid sharp objects coming into contact with the chair such as needles, pins, scissors, etc.
- Do not leave the chair in the UP or DOWN position when not in use.
- Do not drag the chair when moving or lifting as this may cause damage to the lifting mechanism and trap the cables.

### Caution!

The rise/recline of the footrest is supported by a scissor mechanism. Do not leave children or pets unsupervised or directly around the chair. There is a potential risk of trapping by the mechanism. Take care to avoid trapping clothing, blankets, fingers etc.

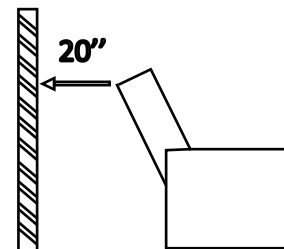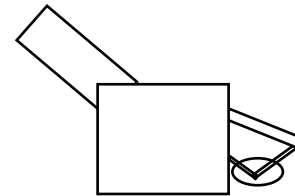

Make sure there is enough distance between the wall and the chair back BEFORE fully reclining. Reclining the chair too close to the wall will damage the chair. We suggest you keep 20 inches between the wall and your chair.


## Specific Warnings

- Maximum loading capacity 300 lbs!
- Ensure that only one person sits in the chair at a time.
- Do not route any cables under the chair or through any moving part of the chair. There is a risk of the cable being damaged which could cause an electric shock.
- Do not use the chair with an electric blanket.
- Do not move the chair while the power cable is connected to the outlet.
- Keep all electronics away from moisture and extreme temperatures.
- If the chair becomes damaged, unplug and contact the seller for service.
- Unplug the base before cleaning.
- Avoid getting liquid on to the electrical system.
- Do not step on the armrests or footrest, the mechanism may be damaged by putting too much weight on it.
- When the chair is in the raised position, do not sit down heavily onto it, but, lower yourself gently. Severe impact could damage the mechanism.
- Ensure that the footrest does not catch on the carpet.
- Be careful while using sharp objects such as needles, pins, scissors, etc.
- Do not drag the chair as this may damage the mechanism or break the cables.
- We do advise that the chair checked once every 12 months.
- Only use authorized accessories for the chair.

## Cleaning and Maintenance

### Cleaning-quick steps to clean the chair:

1. Please dust the chair frequently.
2. Wipe over using a damp and soft cloth. Let the chair dry thoroughly before use.
3. Use a vacuum equipped with a soft brush attachment.
4. Use eco-friendly soap (we highly recommend using cleaners and conditioners that are made specifically for leather, never use any type of harsh cleaners, especially alcohol-based products.)

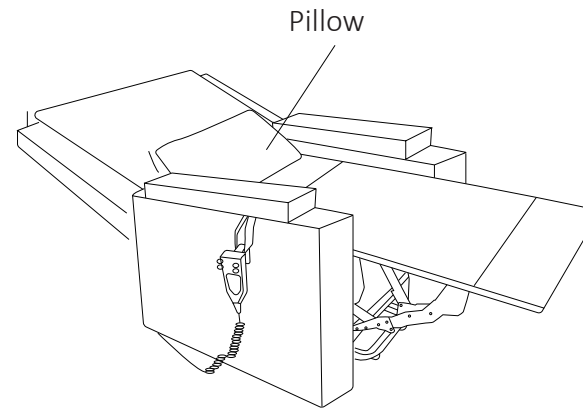

### Warm Tip


This pillow is recommended when lying down on this chair. It will help support your body and make you more comfortable while you sleep.

### WARNING:

1. Greasy spillages can be made worse when encountering water.
2. Do not over expose faux leather or true leather to sunlight.
3. Always unplug the chair before cleaning it.
4. Do not reconnect it until you are sure it is completely dry.
5. Never allow water to get into electronics.

## Safe Seating Position

The chair is a lifting device and it is important that you are positioned correctly in the chair when it is rising and lowering. If the correct body position is not maintained, there is a risk of injury or damage to the chair. The central position should be used either in the standing or sitting mode.

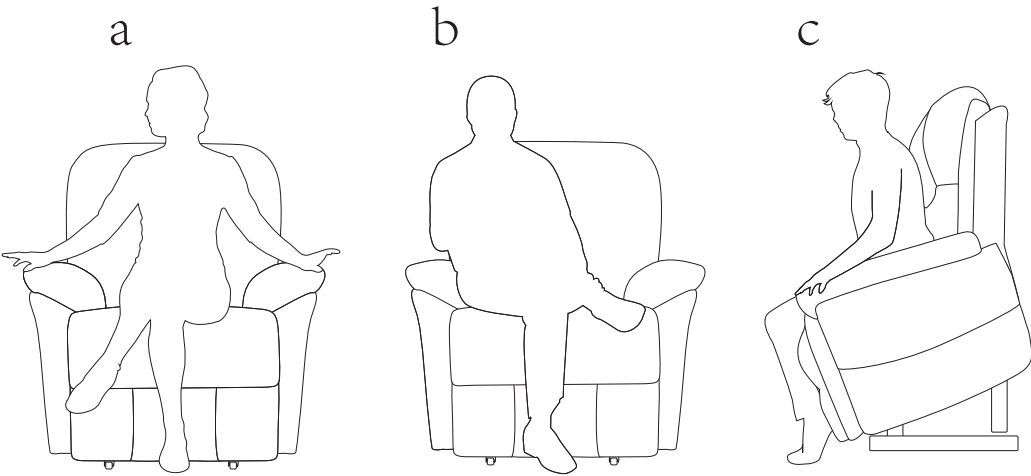

A: Correct position: Sit centrally with arms on the chair.


B: Incorrect position: The body is unstable and there is a possibility of injury.

C: Keep your feet on the floor as the chair is lifting.

## Parts


The first box contains A B C parts.

A: Accessory package

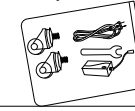


This package is inside the chair base and contains.


1 Motor Power Cord	
2 AC Adapter	
3 A Wrench	
4 Two Castors	

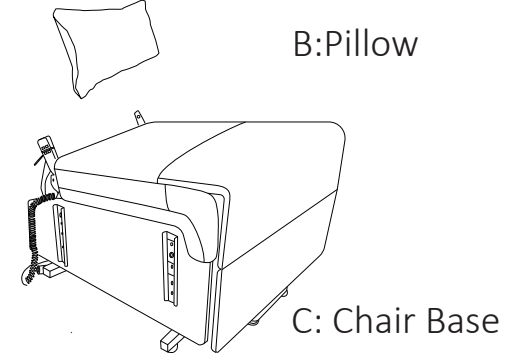

The second box contains D E parts.

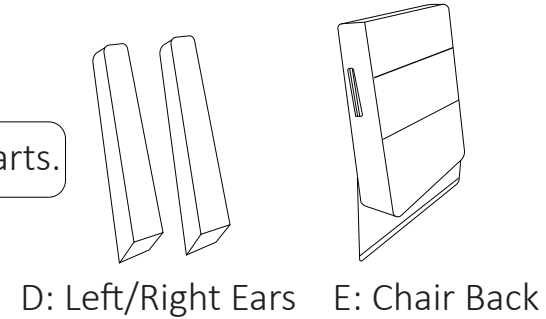


The second box contains F parts.

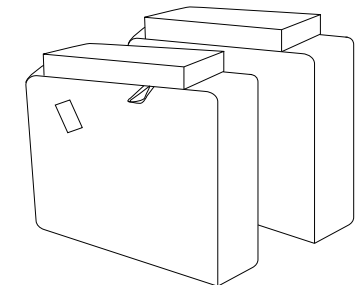


F: Left/Right Arms

# Assembly Steps

## Step 1. Level the chair base.

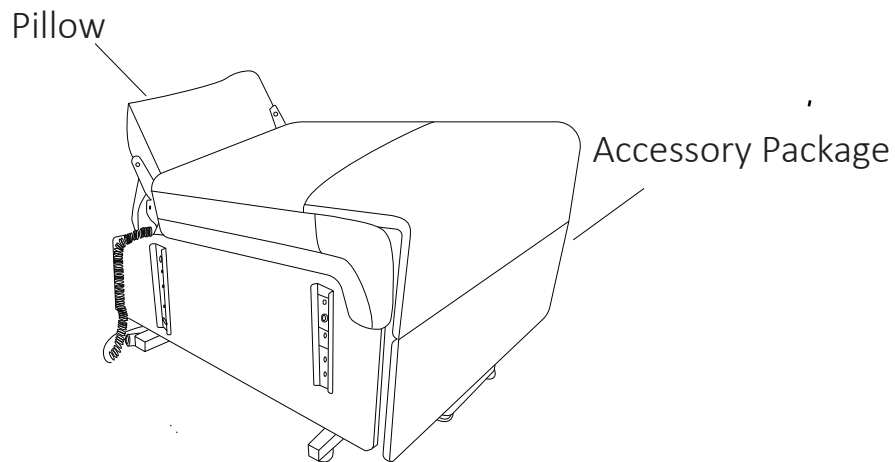


## Step 2. Fasten the castors on the rear of the chair.


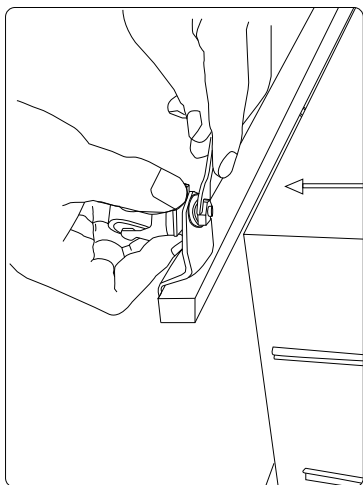


Installation Diagram


- A. Lay the chair base evenly as shown below.
- B. Fasten the castors using a wrench.

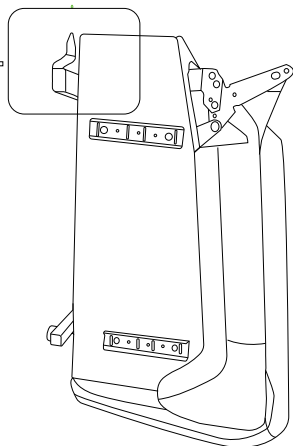

# Cup holder Manual opening and closing

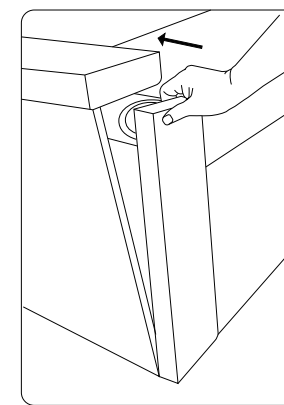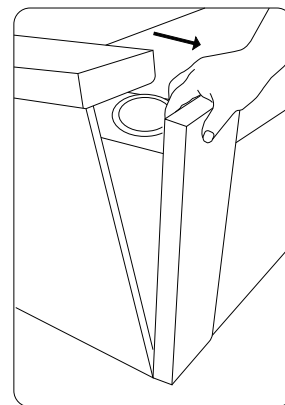


## Remote control

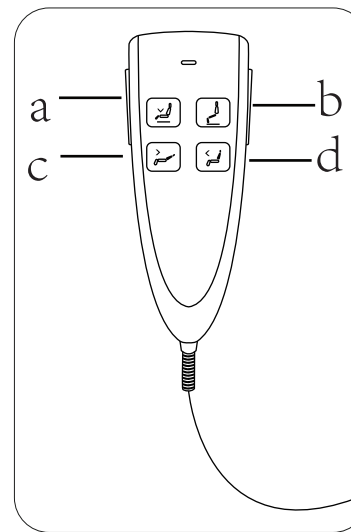

Before you operate the controller. Ensure all the cables are clear of the mechanism. The chair isn't situated on top of any cables.


- a Lifts/Opens footrest and lowers chair.
- b Lowers/Closes footrest and lifts chair.
- c Lowers/Opens backrest.
- d Lifts/Closes backrest.

## Step 8. Assemble the chair ears.

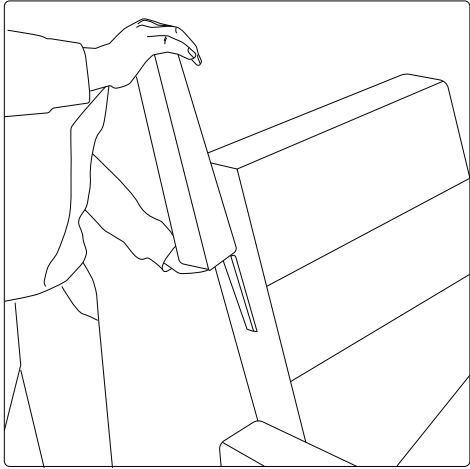


- A. Align the locking clips.
- B. Push the ear down evenly with two hands to bottom.

Step 9. The whole installation process is complete, please enjoy your chair.

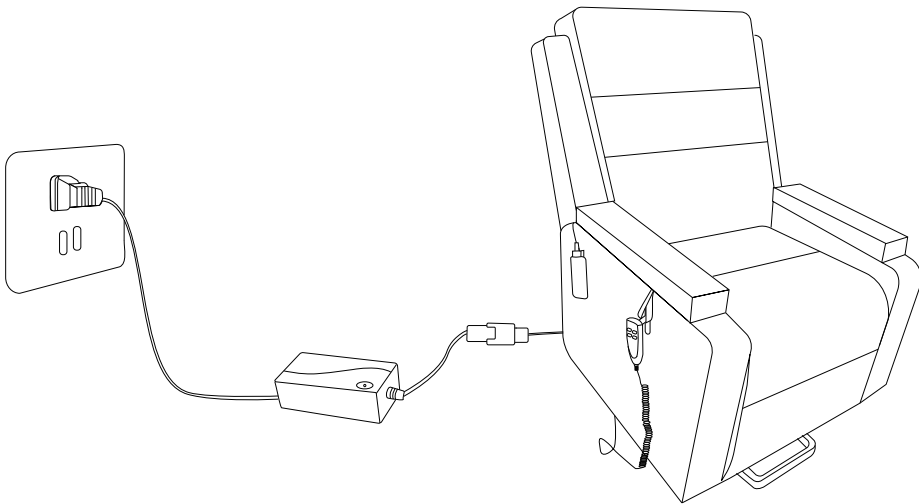


## Step 3. Assemble the chair arms.

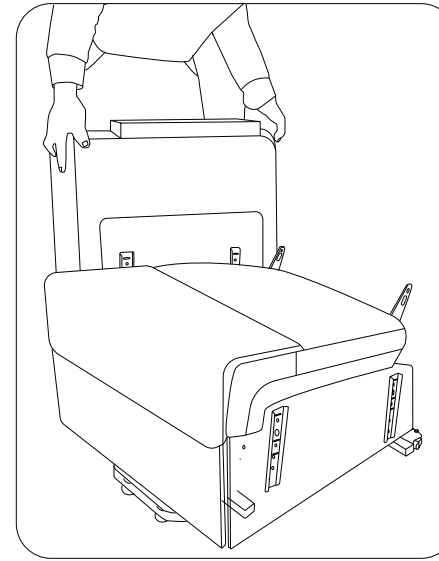


- A. Align the locking clips.
- B. Push the arm down evenly with both hands to the bottom.

## Step 4. Connect the cords.

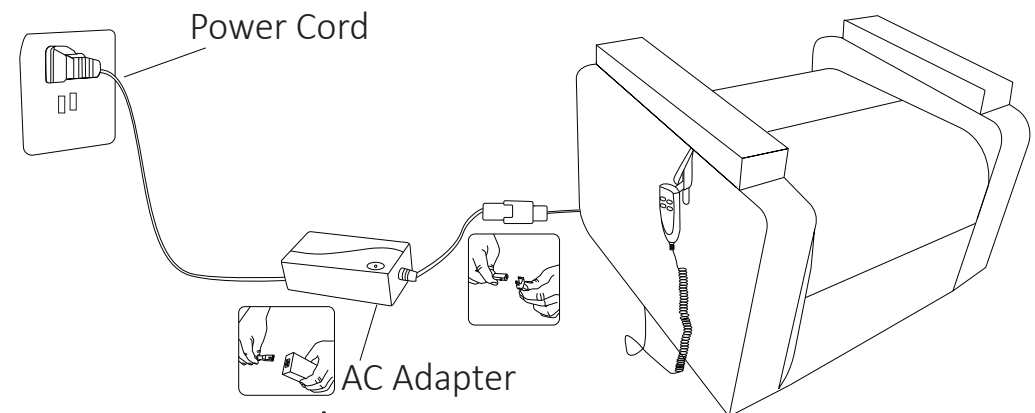


(The cord and AC adapter are in the accessory package which is tied inside the chair base.)


Step 5. Arrange the position of controller.

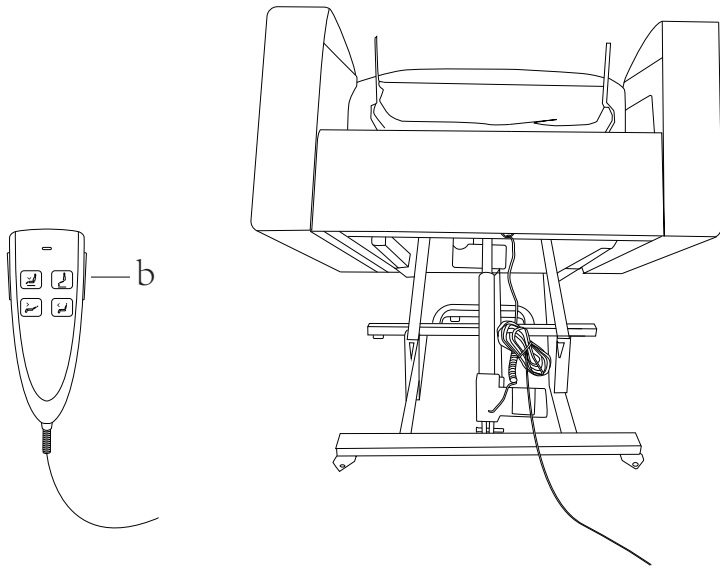

- A. Press this “b” button to lift chair to the highest position as shown above.
- B. Customers can choose to place the controller on the left or right side of the chair according to their preference as shown below.

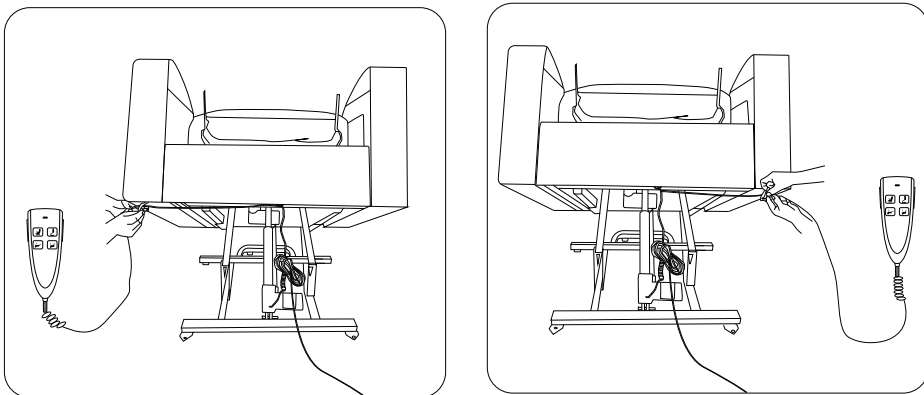


Step 6 .Adjust the backrest's locking clip.

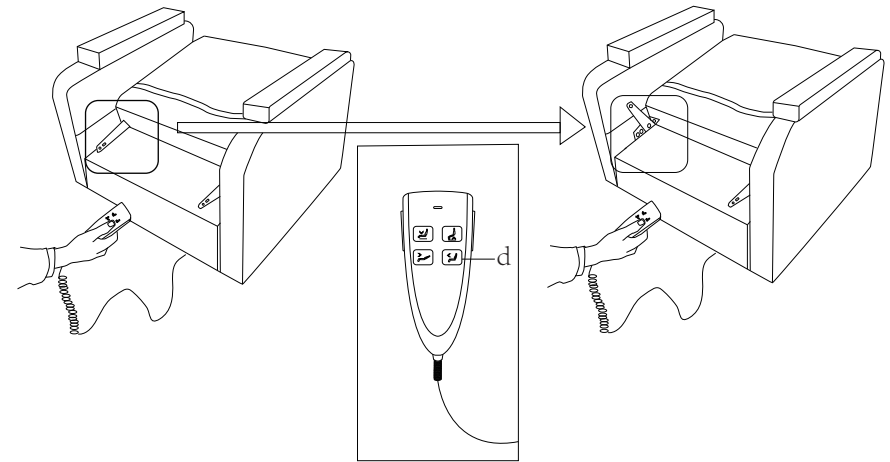

Press this “d ”button to fully open the backrest's locking clip.

Step 7 . Assemble the chair back.

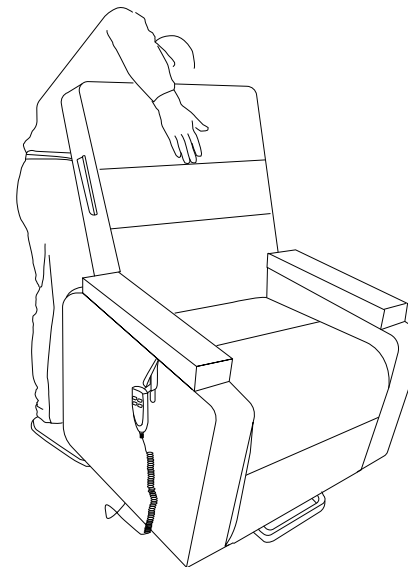

- A. Align the locking clips.
- B. Slide the backrest into the chair base.