Simone

Quick Start Guide


Simone - X6 Innovations - 128 rue la Boetie, 75008 Paris, France

TF-IFU-PF-v20230207

Step 1 - Get to know Simone

Washable components:

1. Diaphragm (anti-backflow)
2. Connector
3. Duckbill valve
4. Flange
5. Bottle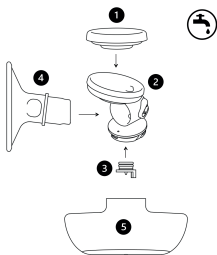


Non-washable components:

6. Hub

Accessories

(not pictured):

- Bra extender strap
- USB charging cable
- Extra O-ring
- Extra duckbill valve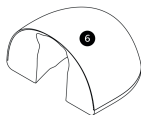

Step 2 - Before you begin: cleaning and sterilizing

Make sure all pump parts are dry before reassembly. Wet parts may affect pump performance or cause damage to the Hub.

Non-Washable Parts:

The Hub contains electronics and is not washable. The Hub can be wiped down with a disinfectant wipe (such as isopropyl alcohol wipes).

Washable Parts:

Before First Use: Sterilizing

Sterilize the bottle, diaphragm, flange, duckbill valve, and connector by placing them into a pot of boiling water for 5 minutes. Do not exceed 5 minutes in boiling water. Keep a gentle boil, do not use a lid, and make sure the parts are not resting on the bottom of the pot. Do not use a steam or microwave sterilizer.

Before Use and Between Uses: Cleaning

- Wash all Milk Assembly parts (Diaphragm, Connector, Duckbill valve, Bottle, Flange) with warm water and mild dish soap. Rinse well under running water and air dry completely before assembling.

Step 3 - Charge your Simone and download the app


Plug USB charging cable (included) into the Pump, then into a USB charger (not included). The LED will blink to indicate charging.

Download the "Simone Pump" app for your smartphone, available for iOS (App Store) or Android (Google Play Store).

Step 4 - Assembling the Simone pump


Get your Simone ready for pumping

1. Check that the duckbill valve is not sealed shut by pressing on the sides with your fingers. It's completely normal for it to look 'closed' at rest, but if it is sealed shut even when you press on the sides as shown, contact us or try the spare that is in your Simone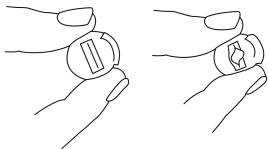



2. Fit the silicone diaphragm onto the top of the connector.

3. Insert the duckbill valve into the hole at the bottom of the connector. Make sure it is fully inserted.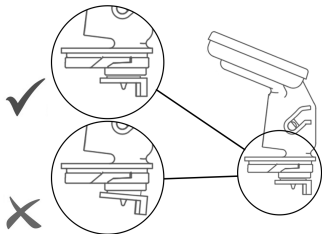


4. Insert the flange into the connector. Make sure to insert it fully (the thicker ring on the flange should be pressing against the connector without leaving any gap). Check the alignment of the flange in the connector. The small hole should face towards the top of the connector (towards the diaphragm), and the

large hole should face the bottom of the connector (towards where the milk bottle will screw on), to allow for milk to flow

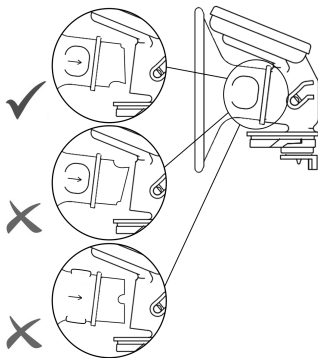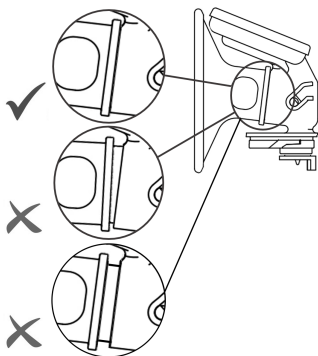
5. Make sure the O-ring is positioned around the base of the connector and that it is securely in place. If you lose it, there is a spare in the accessory bag.
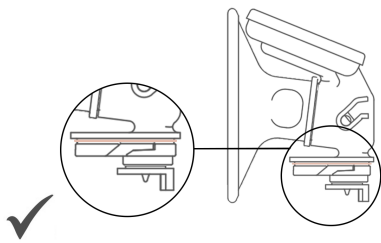


6. Screw the bottle onto the bottom of the connector.

7. Slide your connector assembly into the shell. Start by lining up the circular part at the top of the connector, with the ring inside the shell. Rotate the bottom of the connector slightly towards the front of the hub. You will hear a small click!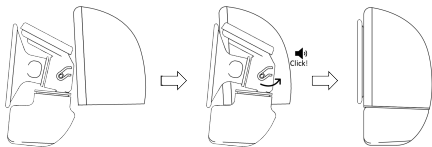

8. Check that the diaphragm is tightly pressed against the shell wall.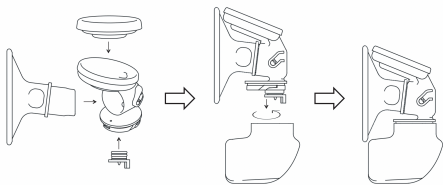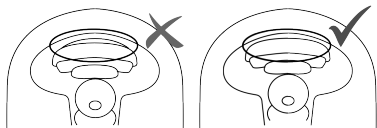


Step 5 - Turn on your Simone and start pumping

Make sure your hands and pump parts are clean and dry before pumping.

Position the Simone pump in your nursing bra so that your nipple is centered in the flange tunnel. Use the included bra extender strap to accommodate the Simone pump in your nursing bra, if needed. The pump needs to be pressed firmly against the breast with no gap.

Long press on the Power button to power on the Simone. Once powered on, your Simone can be controlled using the interface on the breast pump, or through the Simone app.

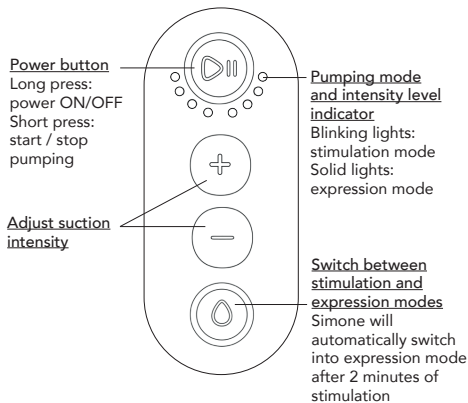If you have any problems, reset your breast pump by holding down the power button for 15-20 seconds.

Step 6 - Get more information or get help

We love feedback! If you have questions, comments, or need support: hello@simonepump.com

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