User Guide

TM2000

Truvitals™

Non-Touch 2-in-1 Thermometer

Thermomètre 2 en 1 Sans Contact | Termometro 2 in 1 Non Touch |
Berührungsloses 2-in-1-Thermometer | Termómetro 2 en 1 No Táctil

Table of Contents

Introduction.....	1
Safety Precautions.....	2-3
Features.....	3
Package Contents.....	3
Product Overview.....	4
Operation Instructions.....	5-9
Maintenance & Cleaning.....	9-10
Error Codes.....	10-11
Specifications.....	11-12
Warranty.....	12-14
Contact Us.....	14

FR

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To see this user guide in Spanish, German, or Italian, please visit www.truvitals.com or scan the QR code below:

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Non-Touch 2-in-1 Thermometer

User Guide

Thank you for purchasing the Truvitals Non-Touch 2-in-1 Thermometer. This User Guide is intended to provide you with guidelines to ensure that operation of this product is safe and does not pose risk to the user. Any use that does not conform to the guidelines described in this User Guide may void the limited warranty.

Please read all directions before using the product and retain this guide for reference. This product is intended for household use only. It is not intended for commercial use.

This product is covered by a limited one-year warranty. Coverage is subject to limits and exclusions. See warranty for details.

Safety Precautions

- This thermometer is not meant to diagnose illness or take the place of your doctor. If you are experiencing symptoms of illness, contact your physician for further guidance.
- This thermometer is designed to measure your temperature and should not replace consulting your physician.
- Dispose of this thermometer responsibly and in accordance with local regulations.
- Remove the batteries from the thermometer if it will not be used for three (3) months or longer.
- Dispose of old batteries responsibly and in accordance with local regulations.
- When replacing the batteries, change all of the batteries at the same time.
- Do not mix types or brands of batteries.
- Batteries may leak harmful liquids, ignitable materials, or may explode potentially causing personal injury or product damage.
- Do not use this thermometer where flammable gases are present.
- Inspect the thermometer before each use to make sure parts are not broken or missing.
- Do not use the thermometer outside of the proper temperature conditions to ensure accurate readings.
- Handle the thermometer with care and avoid dropping the thermometer to prevent inaccurate readings.
- Clean the thermometer sensor before each use to ensure accurate readings.
- Do not put any protective sleeves on the thermometer as this may impact the accuracy of temperature readings.
- Wait at least 30 seconds between readings to ensure accuracy.
- Keep young children in the same environment for a few minutes before taking their temperature as their bodies react slowly to environment changes.
- For the most accurate reading on children ages 3 and under, it is recommended to take three (3) temperature readings and use the highest measurement.
- There are no user-serviceable parts inside the thermometer.
- Do not attempt to repair the thermometer on your own.

TruVitals Non-Touch 2-in-1 Thermometer

- Do not attempt to disassemble, overhaul, or modify this thermometer.
- Keep this thermometer out of reach of children.
- Do not dip or submerge the thermometer in water or other liquids.
- Clean the thermometer using a soft, dry cloth only. Do not use harsh detergents or liquids.
- Keep the thermometer out of direct sunlight in a dry, dust-free environment.

Intended Use

The TruVitals Non-Touch 2-in-1 Thermometer is a battery-operated electronic device with intended use of measuring body temperature through touchless contact by means of an infrared sensor coupled with electronic signal amplification, conditioning, and a digital LED display. The device is reusable and intended for home use on people at all ages.

Features


- Accurate readings in one second
- Easy-to-read LED display
- Audible & color-coded fever alerts
- Reads body/surface temperatures
- Contactless, no-touch readings
- Memory feature records up to 32 measurements
- Read the temperature of objects like baby bottles and more
- Pre-calibrated thermometer is ready to use out of the box
- 15-second auto shutoff to preserve battery life

Package Contents

- ① Non-Touch 2-in-1 Thermometer
- ② AAA 1.5 V Batteries (x2)
- ③ User Guide


Product Overview

Thermometer Overview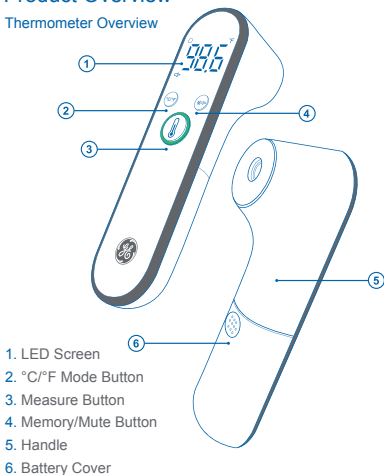

Screen Overview

1. Body Mode Indicator
2. Surface Mode Indicator
3. °C/°F Mode
4. Temperature
5. Low Battery Icon
6. Mute Mode

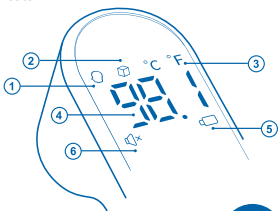Operation Instructions

Body Mode Temperature Readings

1. Power on the thermometer by pressing the measure button. Check that the body mode indicator is lit on the screen to confirm the thermometer is in body mode.


2. Hold the thermometer 1-2 in. (3-5 cm) from the forehead and press the measure button.

3. Hold the thermometer still during reading. A beep will indicate that the measurement is complete unless the device is in mute mode.

4. Once a temperature is taken, the temperature will display for five (5) seconds. After the temperature has finished displaying, the thermometer will revert to standby mode indicated by three (3) dashes on the LED screen. Simply press the measure button again to take a new temperature.

5. To power off the thermometer, hold the measure button for five (5) seconds while the device is in standby mode or wait 15 seconds for the auto shutoff feature to power off the thermometer.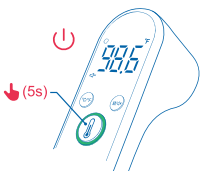

NOTES: Environment changes can affect the accuracy of temperature readings. Before use, allow the thermometer to remain in one location without significant air temperature changes for 30 minutes.

Wait at least 30 minutes after strenuous exercise before taking a temperature reading.

Do not wear a hat and make sure there is no hair, sweat, or makeup on your forehead while taking temperature readings.

To avoid human error, take multiple measurements and use the average temperature.


Fever Alerts

If a temperature reading is above the normal range, the thermometer will give you a fever alert. The table below outlines how the thermometer indicates fever alerts.

One Short Beep	Four Short Beeps	Six Short Beeps
<p>A thermometer with a green backlight and a single short beep icon above the display. The display shows "99.4" with a degree Fahrenheit symbol.</p>	<p>A thermometer with a yellow backlight and four short beeps icon above the display. The display shows "100.2" with a degree Fahrenheit symbol.</p>	<p>A thermometer with a red backlight and six short beeps icon above the display. The display shows "101.3" with a degree Fahrenheit symbol.</p>
<p>Green Backlight Less than 99.5 °F (37.5 °C)</p>	<p>Yellow Backlight 99.5 °F - 101.3 °F (37.5 °C - 38.5 °C)</p>	<p>Red Backlight Greater than 101.3 °F (38.5 °C)</p>

Memory Feature

1. This thermometer automatically stores the last 32 body temperature measurements.
2. To view previous readings, press the memory button.
3. Press the memory button to scroll to the next stored measurement. Before displaying the measurement, a number will display between 1 and 32 to indicate which measurement you are viewing.
4. Remove the batteries from the thermometer to clear the memory.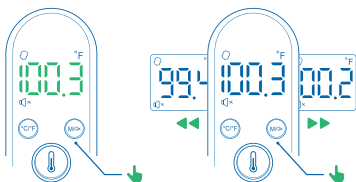


Switching Between Fahrenheit and Celsius

This thermometer can display temperatures in either degrees Fahrenheit (°F) or degrees Celsius (°C). To switch the unit of temperature displayed, simply press the °C/°F button while in standby mode.

Mute Feature

To mute the thermometer, hold the memory/mute button for two (2) seconds. This will silence all alerts from the thermometer.


To unmute the thermometer, simply repeat the process. An audio alert will indicate that the thermometer is unmuted.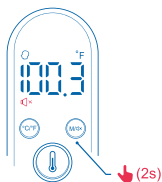

NOTE: When the thermometer is powered on, it will remain in the last used setting.


Surface Mode Temperature Readings

1. With the thermometer powered off, press and hold the measure and memory buttons at the same time for three (3) seconds. Check that the surface mode indicator is lit on the screen to confirm the thermometer is in surface mode.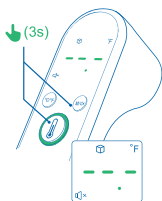


2. Hold the thermometer 1-2 in. (3-5 cm) from the surface you wish to measure and press the measure button.

3. Hold the thermometer still during reading. A beep will indicate that the measurement is complete unless the device is in mute mode.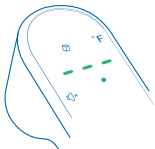

4. Once a temperature is taken, the temperature will display for five (5) seconds. After the temperature has finished displaying, the thermometer will revert to standby mode indicated by three (3) dashes on the LED screen. Simply press the measure button again to take a new temperature.

- To power off the thermometer, hold the measure button for five (5) seconds or wait 15 seconds for the auto shutoff feature to power off the thermometer.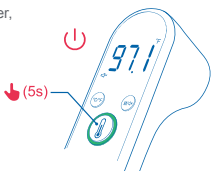

NOTE: The thermometer will automatically default back to body mode each time it powers off.



Maintenance & Cleaning

Battery Replacement

When the batteries need to be replaced, a low battery icon will display on the screen. To change the batteries, follow the steps below.

- Slide the battery cover down to open the battery compartment.
- Remove the old batteries and dispose of them properly.
- Insert two (2) new batteries aligning them with the polarities shown in the battery compartment.
- Slide the battery cover back into place before powering on the thermometer.

NOTE: Remove the batteries if the thermometer will not be in use for more than three (3) months.

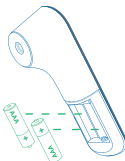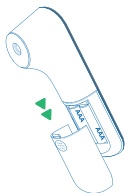Cleaning

To clean the thermometer, wipe it with a dry, soft cloth. If the thermometer may be contaminated, use a cloth moistened with water and a mild detergent or clean it with 75% alcohol.

Take care not to use an excessive amount of liquid when cleaning the thermometer as liquids can seep into the electronic parts and cause damage to the thermometer.

Storage

Store in a safe, dry place free from dust and direct sunlight. Keep this thermometer out of reach of children.

If you are storing the thermometer for more than three (3) months, remove the batteries and keep the thermometer in the original packaging.

Error Codes

Error Code	Code Meaning	Resolution
	The batteries are low.	Replace the batteries.
H	Temperature reading is too high.	Make sure the thermometer is properly aligned with the surface you are trying to read.
L	Temperature reading is too low.	Make sure the thermometer is properly aligned with the surface you are trying to read.
EH	Environment temperature is too high.	Move the thermometer to an appropriate environment and keep it there for 30 minutes before attempting to read a temperature.
EL	Environment temperature is too low.	Move the thermometer to an appropriate environment and keep it there for 30 minutes before attempting to read a temperature.

Error Code	Code Meaning	Resolution
Err	System error.	Move the thermometer to an appropriate environment before trying again. If the problem persists, contact customer service.
No Display	The batteries are dead.	Replace the batteries.

Specifications

Operating Specifications

Body Mode Range: 89.6 °F - 109.4 °F (32 °C - 43 °C)

Surface Mode Range: 0 °F - 199.9 °F (0 °C - 93.2 °C)

Accuracy: ± 0.4 °F (± 0.2 °C) between 96.8 °F - 102.2 °F (36 °C - 39 °C)

Resolution: 0.2 °F (0.1 °C)

Response Time: 1 sec

Time Displayed on Screen: 5 sec

Memory Capacity: 32 readings

Thermometer Specifications

Dimensions (L x W x D): 5.5" x 1.4" x 1.7" (140 x 36 x 42 mm)

Weight (with Batteries): 2.9 oz. (81 g)

Power Supply: 2x AAA 1.5 V batteries

Display Type: LED

Operating Environment

Temperature: 50 °F - 104 °F (10 °C - 40 °C)

Humidity: ≤ 85% RH

Atmospheric Pressure: 70 kPa ~ 106 kPa

Storage Environment

Temperature: -4 °F - 131 °F (-20 °C - 55 °C)

Humidity: ≤ 93% RH

Atmospheric Pressure: 70 kPa ~ 106 kPa

Warranty

LIMITED WARRANTY TO ORIGINAL CONSUMER

This TruVitals Non-Touch 2-in-1 Thermometer ("Product"), including any accessories included in the original packaging, as supplied and distributed new by an authorized retailer is warranted by C&A Marketing, Inc. (the "Company") to the original consumer purchaser only, against certain defects in material and workmanship ("Warranty") as follows:

To receive Warranty service, the original consumer purchaser must contact the Company or its authorized service provider for problem determination and service procedures. Proof of purchase in the form of a bill of sale or receipted invoice, evidencing that the Product is within the applicable Warranty period(s), MUST be presented to the Company or its authorized service provider in order to obtain the requested service.

Service options, parts availability, and response times may vary and may change at any time. In accordance with applicable law, the Company may require that you furnish additional documents and/or comply with registration requirements before receiving warranty service. Please contact our customer service for details on obtaining warranty service:

866-883-0022
info@truvitals.com

Shipping expenses to the Company's Return Facility are not covered by this warranty, and must be paid by the consumer. The consumer likewise bears all risk of loss or further damage to the Product until delivery to said facility.

EXCLUSIONS AND LIMITATIONS The Company warrants the Product against defects in materials and workmanship under normal use for a period of ONE (1) YEAR from the date of retail purchase by the original end-user purchaser ("Warranty Period"). If a hardware defect arises and a valid claim is received within the Warranty Period, the Company, at its sole option and to the extent permitted by law, will either (1) repair the Product defect at no charge, using new or refurbished replacement parts, (2) exchange the Product with a Product that is new or which has been manufactured from new or serviceable used parts and is at least functionally equivalent to the original device, or (3) refund the purchase price of the Product.

A replacement Product or part thereof shall enjoy the warranty of the original Product

TruVitals Non-Touch 2-in-1 Thermometer

for the remainder of the Warranty Period, or ninety (90) days from the date of replacement or repair, whichever provides you longer protection. When a Product or part is exchanged, any replacement item becomes your property, while the replaced item becomes the Company's property. Refunds can only be given if the original Product is returned.

This Warranty does not apply to:

- (a) Any non-TruVitals Non-Touch 2-in-1 Thermometer product, hardware, or software, even if packaged or sold with the Product;
- (b) Damage caused by use with non-TruVitals Non-Touch 2-in-1 Thermometer products;
- (c) Damage caused by accident, abuse, misuse, flood, fire, earthquake, or other external causes;
- (d) Damage caused by operating the Product outside the permitted or intended uses described by the Company;
- (e) Damage caused by third party services;
- (f) A Product or part that has been modified to alter functionality or capability without the written permission of the Company;
- (g) Consumable parts, such as batteries, fuses, and bulbs;
- (h) Cosmetic damage; or
- (i) If any TruVitals Non-Touch 2-in-1 Thermometer serial number has been removed or defaced.

This Warranty is valid only in the country where the consumer purchased the Product, and only applies to Products purchased and serviced in that country.

The Company does not warrant that the operation of the Product will be uninterrupted or error-free. The Company is not responsible for damage arising from your failure to follow instructions relating to its use.

NOTWITHSTANDING ANYTHING TO THE CONTRARY AND TO THE MAXIMUM EXTENT PERMITTED BY APPLICABLE LAW, THE COMPANY PROVIDES THE PRODUCT "AS-IS" AND "AS-AVAILABLE" FOR YOUR CONVENIENCE AND THE COMPANY AND ITS LICENSORS AND SUPPLIERS EXPRESSLY DISCLAIM ALL WARRANTIES AND CONDITIONS, WHETHER EXPRESSED, IMPLIED, OR STATUTORY, INCLUDING THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE, TITLE, QUIET ENJOYMENT, ACCURACY, AND NON-INFRINGEMENT OF THIRD-PARTY RIGHTS. THE COMPANY DOES NOT GUARANTEE ANY SPECIFIC RESULTS FROM THE USE OF THE PRODUCT, OR THAT THE COMPANY WILL CONTINUE TO OFFER OR MAKE AVAILABLE THE PRODUCT FOR ANY PARTICULAR LENGTH OF TIME. THE COMPANY FURTHER DISCLAIMS ALL WARRANTIES AFTER THE EXPRESS WARRANTY PERIOD STATED ABOVE.

YOU USE THE PRODUCT AT YOUR OWN DISCRETION AND RISK. YOU WILL BE SOLELY RESPONSIBLE FOR (AND THE COMPANY DISCLAIMS) ANY AND ALL LOSS, LIABILITY, OR DAMAGES RESULTING FROM YOUR USE OF THE PRODUCT.

NO ADVICE OR INFORMATION, WHETHER ORAL OR WRITTEN, OBTAINED BY YOU FROM THE COMPANY OR THROUGH ITS AUTHORIZED SERVICE PROVIDERS SHALL CREATE ANY WARRANTY.

IN NO EVENT WILL THE COMPANY'S TOTAL CUMULATIVE LIABILITY ARISING FROM OR RELATED TO THE PRODUCT, WHETHER IN CONTRACT OR TORT OR OTHERWISE EXCEED THE FEES ACTUALLY PAID BY YOU TO THE COMPANY OR ANY OF ITS AUTHORIZED RESELLERS FOR THE PRODUCT AT ISSUE IN THE LAST YEAR FROM YOUR PURCHASE. THIS LIMITATION IS CUMULATIVE AND WILL NOT BE INCREASED BY THE EXISTENCE OF MORE THAN ONE INCIDENT

OR CLAIM. THE COMPANY DISCLAIMS ALL LIABILITY OF ANY KIND OF ITS LICENSORS AND SUPPLIERS. IN NO EVENT WILL THE COMPANY OR ITS LICENSORS, MANUFACTURERS AND SUPPLIERS BE LIABLE FOR ANY INCIDENTAL, DIRECT, INDIRECT, SPECIAL, PUNITIVE, OR CONSEQUENTIAL DAMAGES (SUCH AS, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS, BUSINESS, SAVINGS, DATA, OR RECORDS) CAUSED BY THE USE, MISUSE, OR INABILITY TO USE THE PRODUCT.

Nothing in these terms shall attempt to exclude liability that cannot be excluded under applicable law. Some countries, states, or provinces do not allow the exclusion or limitation of incidental or consequential damages or allow limitations on warranties, so certain limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may have other rights that vary from state to state or province to province. Contact your authorized retailer to determine if another warranty applies.

Contact Us

truvitals.com

866-883-0022

info@truvitals.com

Symbol Description			
 WARNING	WARNING indicates a hazardous situation which, if not avoided, could result in death or serious injury	 CAUTION	CAUTION indicates a hazardous situation which, if not avoided, may result in minor or moderate injury
 NOTICE	NOTICE addresses practices not related to personal injuries, such as product and/or property damage		Type BF
	Do not dispose of this product as unsorted municipal waste. Collection of such waste separately for special treatment is necessary		This item is compliant with the Medical Device Directive 93/42/EEC of June 14, 1993, a directive of the European Economic community
IP22	Waterproof degree		Refer to instruction manual/booklet
	Manufacturer		Serial number
	Keep dry		Fragile
	Upside		Maximum stacking layer
	EU Representative		UK Representative

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info@truvitals.com

Model No.: TE-79

Shenzhen Belter Health Measurement and Analysis Technology Co. LTD.
702/704, Block C, Tsinghua Unis Science Park No. 13 Lan-ghan Rd
Nanshan District Shenzhen 518057 China


Shanghai International Holding Corp. GMBH (Europe) Eiffestrasse
80 20537, Hamburg Germany

Wellkang Ltd, 16 Castle St, Dover, CT16 1PW, UK

CE 0123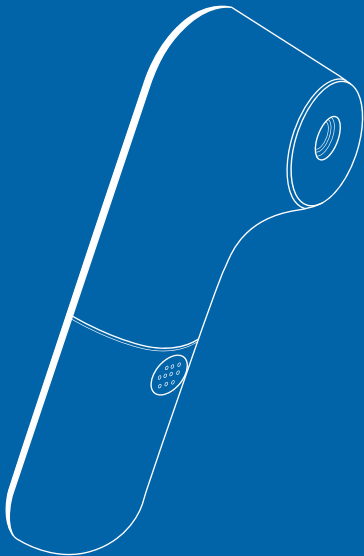

TruVitals is a trademark of C&A IP Holdings LLC in the US.

GE is a trademark of General Electric Company. Used under license.

All other products, brand names, company names, and logos are trademarks of their respective owners, used merely to identify their respective products, and are not meant to connote any sponsorship, endorsement, or approval.

Manufactured and distributed by C&A Marketing, Inc.,
114 Tived Lane East Edison, NJ 08837.

Made in China.

© 2022. C&A IP Holdings LLC. All Rights Reserved.