



STEALTH CLARITY TRANSPARENT SURGICAL FACE MASK WITH ANTI-FOG PANEL - TYPE IIR

Product description

4-layer medical face mask with ear loops and nose clip and Anti-fog viewing window at mouth area. The mask ensures high protection, is characterized by very good comfort and allows easy breathing. The mask is latex-free and biologically safe.

The Stealth Clarity Transparent Surgical mask meets a high-quality standard. The filtering efficiency for bacteria (BFE) is >98%. The material used has been tested for microbiological purity and is biologically safe.

The Stealth Clarity Transparent Surgical mask with Anti-fog Window is designed to facilitate communication with people who rely on facial expressions. This includes people who are deaf, have learning disability, autism or dementia.

Protection

Breathing System for protection from dusts, mists, fumes and micro-organisms e.g. bacteria & viruses.



Technical data

Item number	F13.1.001
Article No	Unit 50
Material	Following EN 14683 Type IIR Polypropylene Filter Material (4 layers) Biocompatibility acc. to ISO 10993-1 Bioburden acc. to ISO 11737-1: ≥ 30 CFU Anti-fog clear panel – tested in accordance to EN166 clause 7.3.2.
Bacterial filter efficiency	>98%
Pressure Gradient	< 60 Pa/cm ²
Latex-free	Yes
Fiberglass free	Yes
Shelf Life	2 years
Product weight (50 box)	0.285 kg
Product weight (single mask)	0.004 kg
Size	17.5cm x 9cm



The product has passed the complete conformity assessment procedure according to MDR 2017/745.

Ordering Information



Code	Description
F13.1.001	Stealth Clarity Transparent Surgical Face Mask – 50 box

Application advice: Fitting Instructions

- 1» To put on the mask, bend the nose bridge forward and place the mask on your hand so that the headgears hang down.
- 2» Put the mask on under the chin - the nose bridge up.
- 3» Pull the straps behind the ears.
- 4» Use both hands to adjust the noseband to the bridge of your nose.

