DIRECTIONS FOR USE

- 1) Wash your hands with hot water and soap, rinse with clean water and dry or use the supplied cleansing gauze.
2) Lay out the material needed as follows: open the aluminium bag, take out only the test strip case and dispose of the desiccant packet.
3) With care and without pulling, perform a 360° rotation of the lancing device's cap.
4) Extract and discard the released cap.
5) Press the lancet, on the side the cap was extracted from, against the fingertip (the tip of the ring-finger is recommended).
6) Keeping your hand pointing downwards, massage the tip of the finger until a large drop of blood forms.
7) Take the pipette without pressing the bulb and place it in contact with the drop of blood.
8) Deposit the blood collected with the pipette into the previously opened bottle.
9) Unscrew the blue cap from the dropper bottle (leave the white cap tightly screwed).
10) Read the test results at 5 minutes.

RESULTS INTERPRETATION

READ THE RESULT AT EXACTLY 5 MINUTES. DO NOT READ THE RESULT AFTER 6 MINUTES. The result must be interpreted independently of the colour intensity of the bands that may appear in the reading window.

NEGATIVE, C-REACTIVE PROTEIN BELOW 10 MG/L

A coloured band appears only under the C (Control) sign, as shown in the figure on the right. This result indicates that the concentration of CRP is below 10 mg/l and suggests the absence of infection and inflammation.

POSITIVE, C-REACTIVE PROTEIN BETWEEN 10 AND 40 MG/L

Two coloured bands appear in the reading window, one by the C (Control) sign, the other by the T1 sign, as shown in the figure on the right. It is recommended to consult a physician, as CRP values between 10 and 40 mg/l may be associated with a viral infection or the onset of a bacterial infection.

POSITIVE, C-REACTIVE PROTEIN BETWEEN 40 AND 80 MG/L

In the reading window, three coloured bands appear, one by the C sign and two others by the T1 and T2 signs, as shown in the figure on the right. Consult a physician, as this result may be associated with a viral or bacterial infection or physical trauma.

POSITIVE, C-REACTIVE PROTEIN OVER 80 MG/L

Four coloured bands appear in the reading window, one by the C sign and three others by the T1, T2 and T3 signs, as shown in the figure on the right. Consult a doctor, as this result may be associated with a bacterial infection or severe inflammation.

NOTE: The colour intensity in the control and test line regions may vary (as represented in the images). The intensity of the bands is not relevant; only their presence must be considered.

NON VALID RESULT

The control band does not appear (no lines appear or one or more T bands appear without the C band). Insufficient sample volume or incorrect procedural techniques are the most likely reasons for the absence of the control line. Review the procedure and repeat the test with a new device and a new blood sample.

FAQ - QUESTIONS AND ANSWERS

HOW DOES THE C-REACTIVE PROTEIN TEST WORK? The test detects the presence of CRP in human whole blood samples using specific antibodies and colloidal gold particles embedded in the test strip.

WHEN CAN THE TEST BE USED? The C-REACTIVE PROTEIN TEST may be used in the case of infection or inflammation symptoms such as fever, headache or weakness, or to monitor the healing process following surgery and minor injuries. The test can also be used to monitor the effectiveness of a therapeutic treatment and can be performed at any time of the day.

CAN THE RESULT BE INCORRECT? The result is correct as long as the instructions are carefully followed and the test is stored under the conditions listed in the 'Precautions' section. However, the result may not be correct if: the device comes into contact with other liquids before being used, if the amount of blood and/or diluent is not enough, if the number of drops dispensed in the well is incorrect or if the reading times are not strictly followed.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the lines are not important for the interpretation of the result.


sult. Refer to the 'Interpretation of results' section and images to correctly evaluate the result. IS THE RESULT RELIABLE IF READ AFTER 6 MINUTES? No. The test must be read 5 minutes after the procedure is completed and no later than 6 minutes from this time. Results read after 6 minutes may not be correct.

WHAT SHOULD I DO IF THE RESULT IS POSITIVE? If the result is positive and the CRP is higher than 10 mg/l, refer to the 'Result interpretation' section and contact your doctor as soon as possible, because antibiotic treatment may be required.

WHAT SHOULD I DO IF THE RESULT IS NEGATIVE? A negative result means that the concentration of CRP is below 10 mg/l, which means that there is neither viral nor bacterial infection in progress. If symptoms persist, seek medical advice.

HOW ACCURATE IS THE C-REACTIVE PROTEIN TEST? The C-REACTIVE PROTEIN TEST showed high concordance with analytical reference methods, with 97% accuracy in distinguishing between normal (< 10 mg/l) and abnormal values.

HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT? The colour and the intensity of the lines are not important for the interpretation of the result.

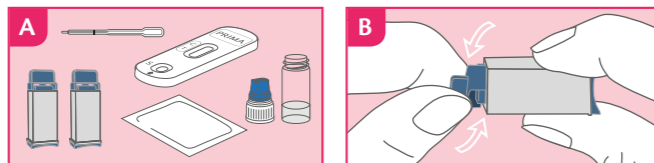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