PRIMA[®] HOME TEST

RESULT/RESULTADO/RESULTADO
 P N **Cut-off: 20 U/mL**
 Date / fecha / data

CELIAC TEST

Self-test to check for the presence of anti deamidated gliadin (anti-DGP) antibodies, IgA and IgG, in the blood, for celiac disease screening.

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GLIADIN AND CELIAC DISEASE REFERENCES
 Celiac disease is an autoimmune disease that causes chronic inflammation of the small intestine. Disease symptoms include diarrhoea, weight loss, abdominal bloating, malnutrition and skin reactions. This inflammation is caused by gliadin, the protein component of gluten, contained in wheat, barley, rye, spelt and many other cereals. Recent studies have shown that subjects affected by celiac disease produce specific antibodies (class IgA and IgG) against deamidated gliadin peptides (anti-DGP). These antibodies have high specificity and sensitivity in celiac disease, which permits very accurate diagnosis. Early diagnosis can reduce the progression of intestinal damage and prevent more serious complications.
 The Celiac Test is a rapid immunochromatographic test to reveal the presence of class IgA and IgG anti-DGP antibodies in whole blood for celiac disease screening.

THE BOX CONTAINS 1 KIT FOR PERFORMING THE CELIAC TEST:

- 1 sealed aluminium bag containing:
 - 1 plastic pipette
 - 2 pain-free sterile lancet devices for fingers, to be used to draw a blood sample
- 1 desiccant packet
- 1 device for the Celiac Test

* Material required but not supplied: cotton wool, a device to measure time (i.e. timer, watch).
 * Do not open the sealed aluminium bag until just before performing the test. Take care to open it as marked.
 * The desiccant packet must not be used. Dispose of it with household waste without opening it.

- PRECAUTIONS**
- 1) Read these instructions for use carefully before performing the test. The Test is reliable only if all the instructions are followed correctly.
 - 2) Keep the Test out of the reach of children.
 - 3) Do not use the Test after the expiry date or if the package has been damaged.
 - 4) Follow the procedure exactly for the specified quantities of blood and diluent.
 - 5) Store the Test components at a temperature between +4 °C and +30 °C. Do not freeze.
 - 6) Use the test and lancet device once only.
 - 7) Test for external use only. DO NOT SWALLOW.
 - 8) In vitro diagnostic device for individual use.
 - 9) Not recommended for people who take anti-coagulant medications (blood thinners) or people suffering from haemophilia.
 - 10) After using, please dispose of all components according to your local waste disposal laws, ask your pharmacist for advice.

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