

Digital Pregnancy Test

Step 1:

When you are ready, remove the test from the foil pouch and remove the cap.

Hold the test by the anti-slip grip with the absorbent tip pointing downwards.

Step 2:

Replace the cap and place the test on a flat surface with the display screen facing upwards.

The clock symbol will appear and blink on the display screen after urine is applied, indicating the test is working.

Step 3:

In approximately 3 minutes, the result "YES+" or "NO-" will appear on the display screen.

Test de grossesse numérique

Étape 1:

Lorsque vous êtes prêt, retirez le test de la pochette en aluminium et retirez le capuchon.

Tenez le test par la poignée antidérapante avec la pointe absorbante vers le bas.

Étape 2:

Remplacez le capuchon et placez le test sur une surface plane avec l'écran d'affichage vers le haut.

Le symbole de l'horloge apparaîtra et clignotera sur l'écran d'affichage après l'application de l'urine, indiquant que le test fonctionne.

Étape 3:

Au bout de 3 minutes environ, le résultat "OUI+" ou "NON-" apparaîtra sur l'écran d'affichage.